

INTOXILYZER® 8000 INSPECTION

Intoxilyzer® 8000 Serial Number: 80-005361

Location: TOXL

A. General Setup and Checks:

1. Diagnostic Tests Pass and Instrument in Ready Mode
2. Breath tube heated
3. Date, time and location code (Level 2,E). Re-set if necessary.
4. Print test (Level 1,P). Sign and attach test record.
5. Tank monitor (Level 3,D,G). Display and Regulator within 50 psi.
 - a. Display: 411 psi Regulator: 400 psi

B. Tests (Sign and attach test records):

Configure simulator for the following tests (Level 1,S).

1. Wet Calibration Check

- a. Low AC (Level 1,C): Known Value ≤ 0.03 AC: 0.015 AC
Sim. Ser #: MP3064 Lot #: 201805C Exp. Date: 5.30.20
- b. High AC (Level 1,C): Known Value ≥ 0.25 AC: 0.300 AC
Sim. Ser #: MP3069 Lot #: 201902D Exp. Date: 2.20.21

Configure dry gas standard for the following tests (Level 1,S).

2. Interferent Check (Level 1,B): Known Value: 0.10 AC + 0.05% Acetone
Sim. Ser #: DR3451 Lot #: ICS 6 Exp. Date: NA
 Display reads "Interferent Detect"
3. RFI Check (CMS Mode) Display reads "RFI Detect"
4. Dry Calibration Check (Level 1,C): Known Value 0.080 AC
Gas Cyl Lot #: 05620080A1 Cyl #: 47 Exp. Date: 4.5.22

Test 1	<u>0.080</u>	Test 4	<u>0.081</u>	Test 7	<u>0.080</u>
Test 2	<u>0.080</u>	Test 5	<u>0.080</u>	Test 8	<u>0.080</u>
Test 3	<u>0.081</u>	Test 6	<u>0.081</u>	Test 9	<u>0.080</u>
Average <u>0.080</u>					

C. Remarks/Maintenance: ANNUAL INSPECTION

Instrument is acceptable to be used in the field.

Charles E. Ed
Breath Analyst Signature

5.15.20
Date

NA
Reviewed by

NA
Date

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005361
Location = TOXL 8164.14.00 09/16
05/15/2020 12:50

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	12:50
02 Std. Sol.	0.013	12:51
03 Room Air	0.000	12:52
04 Std. Sol.	0.014	12:52
05 Room Air	0.000	12:53
06 Std. Sol.	0.014	12:54
07 Room Air	0.000	12:54


08 Sim Temp = 34.0°C

Simul Ser No = MP3064

Std Sol No = 201805C

County = 08

Oper No. = 666666

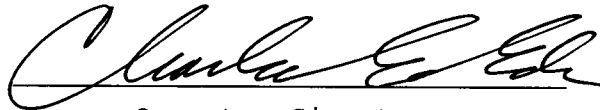

Operator Signature

CHARLES EDER

Remarks:

Low AC
0.015 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005361
Location = TOXL 8164.14.00 09/16
05/15/2020 12:57

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	12:57
02 Std. Sol.	0.298	12:58
03 Room Air	0.000	12:59
04 Std. Sol.	0.298	12:59
05 Room Air	0.000	13:00
06 Std. Sol.	0.298	13:01
07 Room Air	0.000	13:01

08 Sim Temp = 34.0°C

Simul Ser No = MP3069
Std Sol No = 201902D
County = 08 Oper No. = 666666

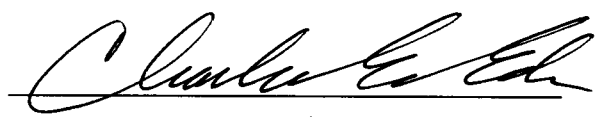

Operator Signature
CHARLES EDER

Remarks: *HIGH AC*
~~0.298 AC~~ 0.300 AC
CEE

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005361
Location = TOXL 8164.14.00 09/16
05/15/2020 13:03

Test	AC	Time
01 Room Air	0.000	13:03
02 *Subject Test	INT*	13:04
03 Room Air	0.000	13:05

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 02/01/1992
Sub Sex = Female Weight = 150
Test = DUI Cit = INTERFERENT
Dr. Lic. = ND/DIS921456
Lot No = 05620080A1
Cyl No = 47
Expiration Date = 04/05/2022
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: *Interferent Check*
0.10 AC + 0.05% ACETONE

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005361
Location = TOXL 8164.14.00 09/16
05/15/2020 13:13

Test	AC	Time
01 Diagnostic	OK	13:13
02 Room Air	0.000	13:14
03 *Subject Test	RFI*	13:14
04 Room Air	0.000	13:15

*Invalid Test
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 02/01/1992
Sub Sex = Female Weight = 150
Test = DUI Cit = RFI CHECK
Dr. Lic. = ND/DIS921456
Lot No = 05620080A1
Cyl No = 47
Expiration Date = 04/05/2022
County = 08 Oper No. = 666666

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature
CHARLES EDER

Remarks:

RFI CHECK

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005361
Location = TOXL 8164.14.00 09/16
05/15/2020 13:16

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	13:16
02 Std. Gas	0.080	13:16
03 Room Air	0.000	13:17
04 Std. Gas	0.080	13:17
05 Room Air	0.000	13:18
06 Std. Gas	0.081	13:18
07 Room Air	0.000	13:19

Lot No = 05620080A1
Cyl No = 47
Exp Date = 04/05/2022
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: *CALIBRATION CHECK*
0.080 AC

Form 106-I8000


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005361
Location = TOXL 8164.14.00 09/16
05/15/2020 13:19

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	13:20
02 Std. Gas	0.081	13:20
03 Room Air	0.000	13:21
04 Std. Gas	0.080	13:21
05 Room Air	0.000	13:21
06 Std. Gas	0.081	13:22
07 Room Air	0.000	13:22

Lot No = 05620080A1
Cyl No = 47
Exp Date = 04/05/2022
County = 08 Oper No. = 666666

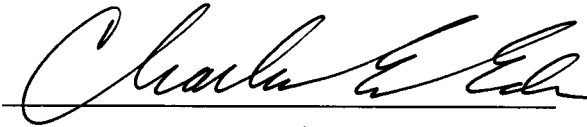

Operator Signature
CHARLES EDER

Remarks: CALIBRATION CHECK
0.080 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005361
Location = TOXL 8164.14.00 09/16
05/15/2020 13:23

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	13:23
02 Std. Gas	0.080	13:24
03 Room Air	0.000	13:24
04 Std. Gas	0.080	13:24
05 Room Air	0.000	13:25
06 Std. Gas	0.080	13:25
07 Room Air	0.000	13:26

Lot No = 05620080A1
Cyl No = 47
Exp Date = 04/05/2022
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: CALIBRATION CHECK
0.080 AC

Form 106-I8000