

INTOXILYZER® 8000 INSPECTION

Intoxilyzer® 8000 Serial Number: 80-005944

Location: TOXL

A. General Setup and Checks:

1.  Diagnostic Tests Pass and Instrument in Ready Mode
2.  Breath tube heated
3.  Date, time and location code (Level 2,E). Re-set if necessary.
4.  Print test (Level 1,P). Sign and attach test record.
5.  Tank monitor (Level 3,D,G). Display and Regulator within 50 psi.
  - a. Display: 342 psi Regulator: 375 psi

B. Tests (Sign and attach test records):

Configure simulator for the following tests (Level 1,S).

1. Wet Calibration Check

- a.  Low AC (Level 1,C): Known Value  $\leq 0.03$  AC: 0.015 AC  
Sim. Ser #: MP3064 Lot #: 201805C Exp. Date: 5.30.20
- b.  High AC (Level 1,C): Known Value  $\geq 0.25$  AC: 0.300 AC  
Sim. Ser #: MP3069 Lot #: 201902D Exp. Date: 2.20.21

Configure dry gas standard for the following tests (Level 1,S).

2.  Interferent Check (Level 1,B): Known Value: 0.10 AC + 0.05% Acetone  
Sim. Ser #: DR3451 Lot #: ICS6 Exp. Date: NA  
 Display reads "Interferent Detect"
3.  RFI Check (CMS Mode)  Display reads "RFI Detect"
4.  Dry Calibration Check (Level 1,C): Known Value 0.080 AC  
Gas Cyl Lot #: 05620080A1 Cyl #: 47 Exp. Date: 4.5.22

Test 1	<u>0.080</u>	Test 4	<u>0.081</u>	Test 7	<u>0.081</u>
Test 2	<u>0.082</u>	Test 5	<u>0.081</u>	Test 8	<u>0.081</u>
Test 3	<u>0.082</u>	Test 6	<u>0.081</u>	Test 9	<u>0.081</u>
Average	<u>0.081</u>				

C. Remarks/Maintenance: ANNUAL INSPECTION

Instrument is acceptable to be used in the field.

Charles E. Ed  
Breath Analyst Signature

5.22.20  
Date

NA

NA

Reviewed by \_\_\_\_\_

Date \_\_\_\_\_

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-005944  
Location = TOXL                 8164.14.00 09/16  
05/22/2020                       12:59

\*\*\*\*\* Printer Test \*\*\*\*\*

abcdefghijklmnopqrstuvwxy1234567890-|=|  
ABCDEFGHIJKLMNopqrstuvwxyz!@#%\$^&\*()\_+?

abcdefghijklmnopqrstuvwxy1234567890-|=|  
ABCDEFGHIJKLMNopqrstuvwxyz!@#%\$^&\*()\_+?

Current Instrument Setup


Data Entry Mode:                Enabled  
Start Test Sequence:            DABACABA  
Display Prelim Rslt?            Yes  
Display Third Digit?            Yes  
Inhib Printer(Y/N)?            No  
Display Volume?                No  
Disable On Memfull?            Yes  
# of Print Copies?              1  
Select Std (D/W/I)?            Dry  
Standard Value?                0.080  
Standard Lot #?                 05620080A1  
Standard Cyl #?                 47  
Standard Expiration?            04/05/2022  
Oper No?                        666666

Flow Cal. Date:                 06/20/2013  
Slope                            677  
Intercept                        -702620

IR Calibration Date:            06/09/2017  
                                    3um                            9um

-----  
0th Coef(\*100):                -16158                        -17815  
1st Coef(\*100):                266780                        136962  
2nd Coef(\*100):                1117                            1089  
H2O adj(mg/l\*10k):            317                             364

\*\*\*\*\* Printer Test End \*\*\*\*\*

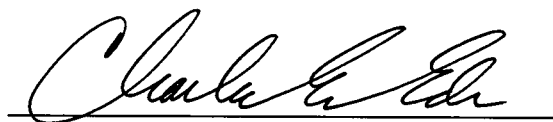

Operator Signature  
CHARLES EDER

Remarks:

PRINT TEST

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-005944  
Location = TOXL      8164.14.00 09/16  
05/22/2020      13:04

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	13:05
02 Std. Sol.	0.013	13:05
03 Room Air	0.000	13:06
04 Std. Sol.	0.014	13:07
05 Room Air	0.000	13:07
06 Std. Sol.	0.013	13:08
07 Room Air	0.000	13:08

08 Sim Temp = 34.0°C

Simul Ser No = MP3064  
Std Sol No = 201805C  
County = 08      Oper No. = 666666



Operator Signature  
CHARLES EDER

Remarks:

LOW AC  
0.015 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-005944  
Location = TOXL      8164.14.00 09/16  
05/22/2020      13:15

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	13:15
02 Std. Sol.	0.306	13:16
03 Room Air	0.000	13:17
04 Std. Sol.	0.308	13:17
05 Room Air	0.000	13:18
06 Std. Sol.	0.307	13:19
07 Room Air	0.000	13:19

08 Sim Temp = 34.0°C

Simul Ser No = MP3069

Std Sol No = 201902D

County = 08

Oper No. = 666666



Operator Signature

CHARLES EDER

Remarks:

HIGH AC  
0.300 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-005944  
Location = TOXL      8164.14.00 09/16  
05/22/2020      13:21

Test	AC	Time
01 Room Air	0.000	13:22
02 *Subject Test	INT*	13:22
03 Room Air	0.000	13:23

\*Invalid Test  
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 02/01/1992  
Sub Sex = Female      Weight = 150  
Test = DUI      Cit = INTERFERENT  
Dr. Lic. = ND/DIS921456  
Lot No = 05620080A1  
Cyl No = 47  
Expiration Date = 04/05/2022  
County = 08      Oper No. = 666666



Operator Signature  
CHARLES EDER

Remarks:

*Interferent Check*  
*0.10AC + 0.05% ACETONE*

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-005944  
Location = TOXL      8164.14.00 09/16  
05/22/2020      13:23

Test	AC	Time
01 Diagnostic	OK	13:24
02 Room Air	0.000	13:25
03 *Subject Test	RFI*	13:25
04 Room Air	0.000	13:25

\*Invalid Test  
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 02/01/1992  
Sub Sex = Female      Weight = 150  
Test = DUI      Cit = RFI CHECK  
Dr. Lic. = ND/DIS921456  
Lot No = 05620080A1  
Cyl No = 47  
Expiration Date = 04/05/2022  
County = 08      Oper No. = 666666

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature  
CHARLES EDER

Remarks: ~~CALIBRATION CHECK~~ CEE  
~~0.080AC~~ CEE

Form 106-18000

RFI CHECK


Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-005944  
Location = TOXL      8164.14.00 09/16  
05/22/2020      13:29

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	13:29
02 Std. Gas	0.080	13:30
03 Room Air	0.000	13:30
04 Std. Gas	0.082	13:31
05 Room Air	0.000	13:31
06 Std. Gas	0.082	13:32
07 Room Air	0.000	13:32

Lot No = 05620080A1  
Cyl No = 47  
Exp Date = 04/05/2022  
County = 08      Oper No. = 666666



Operator Signature  
CHARLES EDER

Remarks:

CALIBRATION CHECK  
0.080 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-005944  
Location = TOXL      8164.14.00 09/16  
05/22/2020      13:33

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	13:34
02 Std. Gas	0.081	13:34
03 Room Air	0.000	13:34
04 Std. Gas	0.081	13:35
05 Room Air	0.000	13:35
06 Std. Gas	0.081	13:36
07 Room Air	0.000	13:36

Lot No = 05620080A1  
Cyl No = 47  
Exp Date = 04/05/2022  
County = 08      Oper No. = 666666



Operator Signature  
CHARLES EDER

Remarks: CALIBRATION CHECK  
0.080 AC

Form 106-I8000




Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-005944  
Location = TOXL      8164.14.00 09/16  
05/22/2020      13:36

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	13:37
02 Std. Gas	0.081	13:37
03 Room Air	0.000	13:38
04 Std. Gas	0.081	13:38
05 Room Air	0.000	13:39
06 Std. Gas	0.081	13:39
07 Room Air	0.000	13:40

Lot No = 05620080A1  
Cyl No = 47  
Exp Date = 04/05/2022  
County = 08      Oper No. = 666666

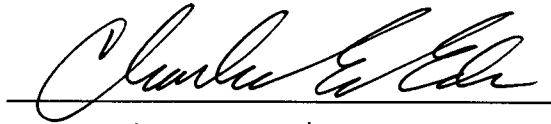

Operator Signature  
CHARLES EDER

Remarks:

*CALIBRATION CHECK*  
*0.080 AC*

Form 106-I8000