

INTOXILYZER® 8000 INSPECTION

Intoxilyzer® 8000 Serial Number: 80-003057

Location: TOXL

A. General Setup and Checks:

1. Diagnostic Tests Pass and Instrument in Ready Mode
2. Breath tube heated
3. Date, time and location code (Level 2,E). Re-set if necessary.
4. Print test (Level 1,P). Sign and attach test record.
5. Tank monitor (Level 3,D,G). Display and Regulator within 50 psi.
 - a. Display: 849 psi Regulator: 960 psi
850 LH

B. Tests (Sign and attach test records):

Configure simulator for the following tests (Level 1,S).

1. Wet Calibration Check

- a. Low AC (Level 1,C): Known Value ≤ 0.03 AC: 0.020 AC
Sim. Ser #: MP5321 Lot #: 2018100 Exp. Date: 10.24.20
- b. High AC (Level 1,C): Known Value ≥ 0.25 AC: 0.300 AC
Sim. Ser #: MP5317 Lot #: 2019020 Exp. Date: 2.20.21

Configure dry gas standard for the following tests (Level 1,S).

2. Interferent Check (Level 1,B): Known Value: 0.10 AC + 0.05% Acetone
Sim. Ser #: DR7348 Lot #: 1055 Exp. Date: NA
 Display reads "Interferent Detect"
3. RFI Check (CMS Mode) Display reads "RFI Detect"
4. Dry Calibration Check (Level 1,C): Known Value 0.080 AC
Gas Cyl Lot #: 24119080A1 Cyl #: 006 Exp. Date: 11-5-21

Test 1 <u>0.079</u>	Test 4 <u>0.080</u>	Test 7 <u>0.080</u>
Test 20 <u>0.080</u>	Test 5 <u>0.079</u>	Test 8 <u>0.079</u>
Test 3 <u>0.080</u>	Test 6 <u>0.079</u>	Test 9 <u>0.079</u>
Average <u>0.079</u>		

C. Remarks/Maintenance: Annual Inspection

TRAINING FOR KALITHES. CEE 5-22-20

Instrument is acceptable to be used in the field
Kalin L. Hilt / Charles E. Ehr
Breath Analyst Signature

21 May 20 / 22 MAY 20
Date

Charles E. Ehr
Reviewed by

22 MAY 20
Date

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003057
Location = TOXL 8164.14.00 09/16
05/22/2020 09:31

***** Printer Test *****

abcdefghijklmnopqrstuvwxyz1234567890-=_|
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&^*()_+?

abcdefghijklmnopqrstuvwxyz1234567890-=_|
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&^*()_+?

Current Instrument Setup


Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 05620080A1
Standard Cyl #? 48
Standard Expiration? 04/05/2022
Oper No? 123456

Flow Cal. Date: 04/04/2018
Slope 549
Intercept 197785

IR Calibration Date: 04/15/2015

	3um	9um
0th Coef(*100):	-23846	-15606
1st Coef(*100):	256102	135820
2nd Coef(*100):	2997	1528
H2O adj(mg/l*10k):	379	300

***** Printer Test End *****

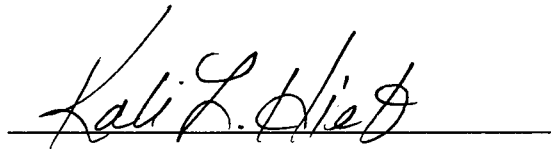

Operator Signature
N/A STUDENT

Remarks:

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003057
Location = TOXL 8164.14.00 09/16
05/22/2020 09:43

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	09:43
02 Std. Sol.	0.018	09:44
03 Room Air	0.000	09:45
04 Std. Sol.	0.018	09:45
05 Room Air	0.000	09:46
06 Std. Sol.	0.019	09:47
07 Room Air	0.000	09:47

08 Sim Temp = 33.9°C

Simul Ser No = MP5321
Std Sol No = 201810D
County = 08 Oper No. = 123456

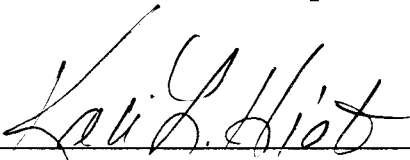

Operator Signature
N/A STUDENT

Remarks: *cal check*
low AC 0.020 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003057
Location = TOXL 8164.14.00 09/16
05/22/2020 09:56

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	09:57
02 Std. Sol.	0.296	09:58
03 Room Air	0.000	09:58
04 Std. Sol.	0.298	09:59
05 Room Air	0.000	10:00
06 Std. Sol.	0.299	10:00
07 Room Air	0.000	10:01


08 Sim Temp = 34.0°C

Simul Ser No = MP5317

Std Sol No = 201902D

County = 08

Oper No. = 123456

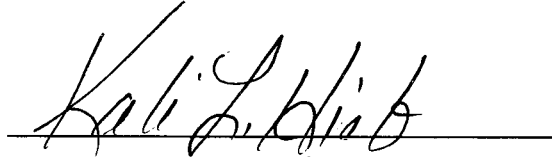

Operator Signature
N/A STUDENT

Remarks:

High AC 0.300 AC
cal check

Form 106-I8000


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003057
Location = TOXL 8164.14.00 09/16
05/22/2020 10:09

Test	AC	Time
01 Room Air	0.000	10:10
02 *Subject Test	INT*	10:11
03 Room Air	0.000	10:11

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS821456
Lot No = 24119080A1
Cyl No = 006
Expiration Date = 11/05/2021
County = 08 Oper No. = 123456

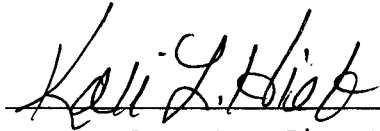

Operator Signature
N/A STUDENT

Remarks: *Interferent 0.10AC + 0.05% Acetone*

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003057
Location = TOXL 8164.14.00 09/16
05/22/2020 10:12

Test	AC	Time
01 Diagnostic	OK	10:13
02 Room Air	0.000	10:13
03 *Subject Test	RFI*	10:13
04 Room Air	0.000	10:14

*Invalid Test
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS821456
Lot No = 24119080A1
Cyl No = 006
Expiration Date = 11/05/2021
County = 08 Oper No. = 123456

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

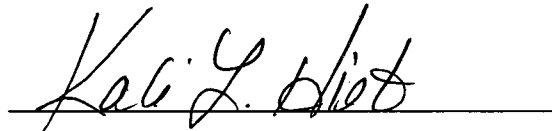
Operator Signature
N/A STUDENT

Remarks: RFI

Form 106-I8000


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003057
Location = TOXL 8164.14.00 09/16
05/22/2020 10:16

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:16
02 Std. Gas	0.079	10:17
03 Room Air	0.000	10:17
04 Std. Gas	0.080	10:18
05 Room Air	0.000	10:18
06 Std. Gas	0.080	10:18
07 Room Air	0.000	10:19

Lot No = 24119080A1
Cyl No = 006
Exp Date = 11/05/2021
County = 08 Oper No. = 123456

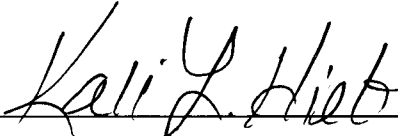

Operator Signature
N/A STUDENT

Remarks: *cal check 0.080 AC*

Form 106-I8000


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003057
Location = TOXL 8164.14.00 09/16
05/22/2020 10:19

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:20
02 Std. Gas	0.080	10:20
03 Room Air	0.000	10:21
04 Std. Gas	0.079	10:21
05 Room Air	0.000	10:21
06 Std. Gas	0.079	10:22
07 Room Air	0.000	10:22

Lot No = 24119080A1
Cyl No = 006
Exp Date = 11/05/2021
County = 08 Oper No. = 123456

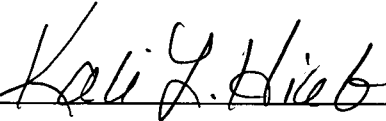

Operator Signature
N/A STUDENT

Remarks: *cal check 0.020 AC*

Form 106-I8000


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003057
Location = TOXL 8164.14.00 09/16
05/22/2020 10:24

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:25
02 Std. Gas	0.080	10:25
03 Room Air	0.000	10:25
04 Std. Gas	0.079	10:26
05 Room Air	0.000	10:26
06 Std. Gas	0.079	10:27
07 Room Air	0.000	10:27

Lot No = 24119080A1
Cyl No = 006
Exp Date = 11/05/2021
County = 08 Oper No. = 123456

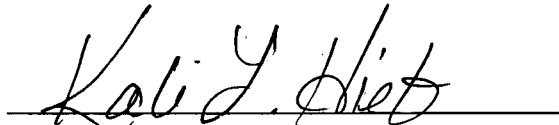
Operator Signature
N/A STUDENT

Remarks: *cal check 0.000 AC*

Form 106-I8000