



**INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT**  
 NORTH DAKOTA OFFICE OF ATTORNEY GENERAL  
 CRIME LABORATORY DIVISION-TOXICOLOGY SECTION/BREATH ALCOHOL PROGRAM  
 SFN 59281 (06/2018)

Serial Number 80-005361	Instrument Location Fort Totten Bureau of Indian Affairs - Location Code 62
Reason for Install/Repair	
<input checked="" type="checkbox"/> Install After Receiving From Crime Laboratory <input type="checkbox"/> Install After Location Change <input type="checkbox"/> Other (Specify) _____	

Check When Done:

- 1. Surge Protector Installed/Property Grounded.
- 2. Telephone Line Connected to Intoxilyzer® 8000.
- 3. Breath Tube Heated.
- 4. Enter Preliminary Data (i.e. Date, Time, DST (Y), and Location; Level 2, Function E).
- 5. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 6. Run Tests:
  - A. Print Test (Level1, Function P).
  - B. ACA Test (Level 1, Function C).
  - C. Radio Frequency Interference (RFI) Test (CMS Mode or Level 1, Function B or C; Key Radio During Test).
- 7. Repair and/or Maintenance Performed (if any): N/A
- 8. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 9. File Previous Intoxilyzer® Record (SFN504096, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 10. Send the Following to the Crime Laboratory: Completed Intoxilyzer® 8000 Installation and Repair Checkout (SFN59281, Form 104-G), Print Test, ACA Test, and RFI Test.

Field Inspector Signature <i>Jeffrey Bauake</i>	Date 5/22/2020
--	-------------------

Crime Laboratory Use Only

This installation has been reviewed and the instrument is approved to be used for the analysis of breath to determine alcohol concentration from the date the Field Inspector performed the installation. This record on file at the Office of Attorney General, Crime Laboratory Division, in the County of Burleigh, North Dakota, is certified to be a true and correct copy of the documents received.

Reviewed/Certified By <i>Charles E. Ed</i>	Certified Date 5.26.2020
---	-----------------------------

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-005361  
Location = FTOT                8164.14.00 09/16  
05/22/2020                      15:13

\*\*\*\*\* Printer Test \*\*\*\*\*

abcdefghijklmnopqrstuvwxyz1234567890-=  
|  
ABCDEFGHIJKLMNQPQRSTUVWXYZ!@#\$\$%^&\*()\_+?

abcdefghijklmnopqrstuvwxyz1234567890-=  
|  
ABCDEFGHIJKLMNQPQRSTUVWXYZ!@#\$\$%^&\*()\_+?

Current Instrument Setup


Data Entry Mode:            Enabled  
Start Test Sequence:        DABACABA  
Display Prelim Rslt?        Yes  
Display Third Digit?        Yes  
Inhib Printer(Y/N)?        No  
Display Volume?            No  
Disable On Memfull?        Yes  
# of Print Copies?         1  
Select Std (D/W/I)?        Dry  
Standard Value?            0.080  
Standard Lot #?            34418080A2  
Standard Cyl #?            56  
Standard Expiration?       02/05/2021  
Oper No?                    120501

Flow Cal. Date:            05/15/2020  
Slope                      634  
Intercept                  -488053

IR Calibration Date:       05/15/2020

	3um	9um
0th Coef(*100):	-25383	-27323
1st Coef(*100):	288661	141732
2nd Coef(*100):	458	822
H2O adj(mg/l*10k):	335	405

\*\*\*\*\* Printer Test End \*\*\*\*\*

  
\_\_\_\_\_  
Operator Signature  
JEFFREY BAUSKE

Remarks: PRINT TEST

Form 106-I8000


Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-005361  
Location = FTOT                    8164.14.00 09/16  
05/22/2020                            15:32

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	15:33
02 Std. Gas	0.079	15:33
03 Room Air	0.000	15:34
04 Std. Gas	0.076	15:34
05 Room Air	0.000	15:34
06 Std. Gas	0.073	15:35
07 Room Air	0.000	15:35

Lot No = 34418080A2  
Cyl No = 56  
Exp Date = 02/05/2021  
County = 62                            Oper No. = 120501

  
\_\_\_\_\_  
Operator Signature  
JEFFREY BAUSKE

Remarks:

INVALID ACA TEST

Form 106-I8000

I'VE SEEN THIS HAPPEN A FEW TIMES  
WITH THE INSTALL OF A GAS CYLINDER. THE  
PIN/SPRING IN THE GAS CYLINDER MAY BE  
SLIGHTLY MIS-ALIGNED. THE CONTINUED USE  
OF THE GAS REGULATOR SWITCH AUTO-CORRECTS  
THE MIS-ALIGNMENT AFTER A FEW USES.

 5/26/20


Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-005361  
Location = FTOT                 8164.14.00 09/16  
05/22/2020                        15:42

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	15:43
02 Std. Gas	0.079	15:43
03 Room Air	0.000	15:44
04 Std. Gas	0.079	15:44
05 Room Air	0.000	15:44
06 Std. Gas	0.079	15:45
07 Room Air	0.000	15:45

Lot No = 34418080A2  
Cyl No = 56  
Exp Date = 02/05/2021  
County = 62                        Oper No. = 120501

  
Operator Signature  
JEFFREY BAUSKE

Remarks: VALID ACA TEST

Form 106-I8000

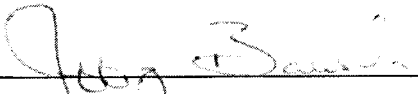
Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-005361  
Location = FTOT                    8164.14.00 09/16  
05/22/2020                            15:36

Test	AC	Time
01 Room Air	0.000	15:40
02 *Subject Test	RFI*	15:41
03 Room Air	0.000	15:41

\*Invalid Test  
Inhibited - RFI

Sub Name = RFI, TEST NA  
Sub DOB = 01/01/2000  
Sub Sex = Unknown                    Weight = NA  
Test = OTH                            Cit = NA  
Dr. Lic. = ND/NA  
Lot No = 34418080A2  
Cyl No = 56  
Expiration Date = 02/05/2021  
County = 62                            Oper No. = 120501

  
\_\_\_\_\_  
Operator Signature  
JEFFREY BAUSKE

Remarks: RFI TEST

Form 106-I8000