

**INTOXILYZER® 8000 INSPECTION**

Intoxilyzer® 8000 Serial Number: 80-003069

Location: TOXL

A. General Setup and Checks:

1.  Diagnostic Tests Pass and Instrument in Ready Mode
2.  Breath tube heated
3.  Date, time and location code (Level 2,E). Re-set if necessary.
4.  Print test (Level 1,P). Sign and attach test record.
5.  Tank monitor (Level 3,D,G). Display and Regulator within 50 psi.
  - a. Display: 845 psi Regulator: 850 psi

B. Tests (Sign and attach test records):

- Configure simulator for the following tests (Level 1,S).

1. Wet Calibration Check

- a.  Low AC (Level 1,C): Known Value  $\leq 0.03$  AC: 0.010 AC  
Sim. Ser #: MP5321 Lot #: 201810D Exp. Date: 10.24.20
- b.  High AC (Level 1,C): Known Value  $\geq 0.25$  AC: 0.300 AC  
Sim. Ser #: MP5317 Lot #: 201902D Exp. Date: 2.20.21

- Configure dry gas standard for the following tests (Level 1,S).

2.  Interferent Check (Level 1,B): Known Value: 0.10 AC + 0.05% Acetone  
Sim. Ser #: DR7348 Lot #: ICS5 Exp. Date: NA  
 Display reads "Interferent Detect"
3.  RFI Check (CMS Mode)  Display reads "RFI Detect"
4.  Dry Calibration Check (Level 1,C): Known Value 0.080 AC  
Gas Cyl Lot #: 051,20080A Cyl #: 04B Exp. Date: 4.5.22

Test 1 <u>0.080</u>	Test 4 <u>0.080</u>	Test 7 <u>0.081</u>
Test 2 <u>0.080</u>	Test 5 <u>0.080</u>	Test 8 <u>0.080</u>
Test 3 <u>0.080</u>	Test 6 <u>0.080</u>	Test 9 <u>0.081</u>
Average <u>0.080</u>		

C. Remarks/Maintenance: Annual Inspection  
CLASS ROOM ONLY. TRAINING FOR KALI HEB.

- Instrument is acceptable to be used in the field.

Kali G. Hest/Chad E. Eds  
Breath Analyst Signature

29 MAY 20  
29 May 20  
Date

Reviewed by

Chad E. Eds

29 MAY 20  
Date

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer           Alcohol Analyzer  
North Dakota Model 8000       SN 80-003069  
Location = TOXL                 8164.14.00 09/16  
05/22/2020                       14:32

\*\*\*\*\* Printer Test \*\*\*\*\*

abcdefghijklmnopqrstuvwxyz1234567890-=  
ABCDEFGHIJKLMNopqrstuvwxyz!@#%^&\*()\_+?

abcdefghijklmnopqrstuvwxyz1234567890-=  
ABCDEFGHIJKLMNopqrstuvwxyz!@#%^&\*()\_+?

Current Instrument Setup


Data Entry Mode:           Enabled  
Start Test Sequence:       DABACABA  
Display Prelim Rslt?       Yes  
Display Third Digit?       Yes  
Inhib Printer(Y/N)?       No  
Display Volume?            No  
Disable On Memfull?       Yes  
# of Print Copies?         1  
Select Std (D/W/I)?       Dry  
Standard Value?            0.080  
Standard Lot #?            24119080A1  
Standard Cyl #?            9  
Standard Expiration?       11/05/2021  
Oper No?                    123456

Flow Cal. Date:            11/16/2018  
Slope                       624  
Intercept                  -587323

IR Calibration Date:       11/16/2018  
                              3um                   9um

-----  
0th Coef(\*100):       -22190           -14281  
1st Coef(\*100):       275678           132600  
2nd Coef(\*100):       2091             1201  
H2O adj(mg/l\*10k):   559               338

\*\*\*\*\* Printer Test End \*\*\*\*\*

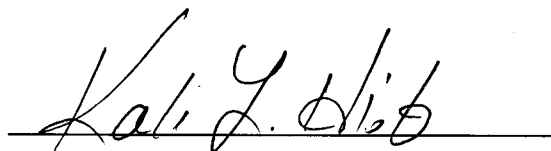

Operator Signature  
N/A STUDENT

Remarks: *print test*

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-003069  
Location = TOXL      8164.14.00 09/16  
05/22/2020      14:39

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:39
02 Std. Sol.	0.019	14:40
03 Room Air	0.000	14:40
04 Std. Sol.	0.020	14:41
05 Room Air	0.000	14:42
06 Std. Sol.	0.019	14:42
07 Room Air	0.000	14:43

08 Sim Temp = 33.9°C

Simul Ser No = MP5321  
Std Sol No = 201810D  
County = 08      Oper No. = 123456

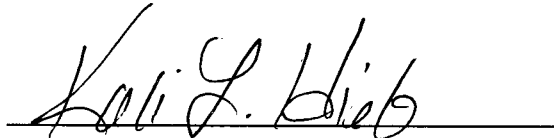

Operator Signature  
N/A STUDENT

Remarks:

*cal check  
low AC 0.020 AC*

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-003069  
Location = TOXL      8164.14.00 09/16  
05/29/2020      09:05

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	09:05
02 Std. Sol.	0.293	09:06
03 Room Air	0.000	09:07
04 Std. Sol.	0.293	09:07
05 Room Air	0.000	09:08
06 Std. Sol.	0.293	09:09
07 Room Air	0.000	09:09

08 Sim Temp = 34.0°C

Simul Ser No = MP5317  
Std Sol No = 201902D  
County = 08      Oper No. = 123456

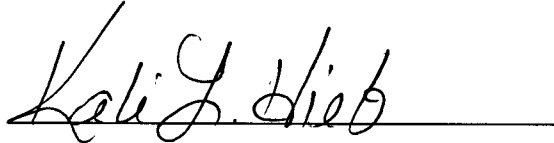

Operator Signature  
N/A STUDENT

Remarks:      *cal check*  
                  *High AC 0.300 AC*

Form 106-I8000


Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-003069  
Location = TOXL                8164.14.00 09/16  
05/29/2020                        09:15

Test	AC	Time
01 Room Air	0.000	09:17
02 *Subject Test	INT*	09:18
03 Room Air	0.000	09:18

\*Invalid Test  
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 01/01/1982  
Sub Sex = Female                    Weight = 150  
Test = DUI                            Cit = NA  
Dr. Lic. = ND/DIS821456  
Lot No = 05620080A1  
Cyl No = 48  
Expiration Date = 04/05/2022  
County = 08                          Oper No. = 123456

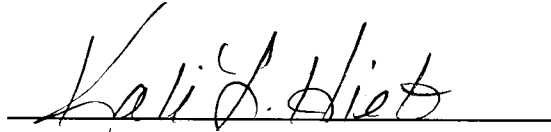

Operator Signature  
N/A STUDENT

Remarks: Interferent check  
0.10AC + 0.05% Acetone

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-003069  
Location = TOXL                8164.14.00 09/16  
05/29/2020                        09:29

Test	AC	Time
01 Diagnostic	OK	09:30
02 Room Air	0.000	09:31
03 Reference	RFI*	09:31
04 Room Air	0.000	09:32

\*Invalid Test  
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 01/01/1982  
Sub Sex = Female                    Weight = 150  
Test = DUI                            Cit = NA  
Dr. Lic. = ND/DIS821456  
Lot No = 05620080A1  
Cyl No = 48  
Expiration Date = 04/05/2022  
County = 08                          Oper No. = 123456

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

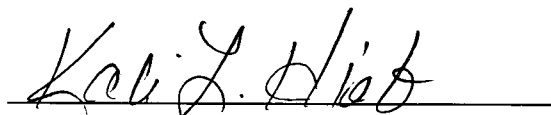

Operator Signature  
N/A STUDENT

Remarks:    RFI check

Form 106-I8000


Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-003069  
Location = TOXL      8164.14.00 09/16  
05/29/2020      09:32

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	09:33
02 Std. Gas	0.080	09:33
03 Room Air	0.000	09:34
04 Std. Gas	0.080	09:34
05 Room Air	0.000	09:35
06 Std. Gas	0.080	09:35
07 Room Air	0.000	09:36

Lot No = 05620080A1  
Cyl No = 48  
Exp Date = 04/05/2022  
County = 08      Oper No. = 123456

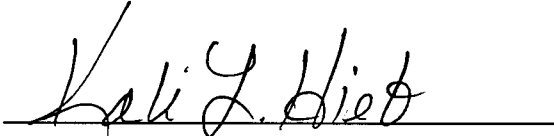

Operator Signature  
N/A STUDENT

Remarks:      *cal check*  
                  *0.080 AC*

Form 106-I8000


Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-003069  
Location = TOXL      8164.14.00 09/16  
05/29/2020      09:36

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	09:37
02 Std. Gas	0.080	09:37
03 Room Air	0.000	09:38
04 Std. Gas	0.080	09:38
05 Room Air	0.000	09:39
06 Std. Gas	0.080	09:39
07 Room Air	0.000	09:40

Lot No = 05620080A1  
Cyl No = 48  
Exp Date = 04/05/2022  
County = 08      Oper No. = 123456

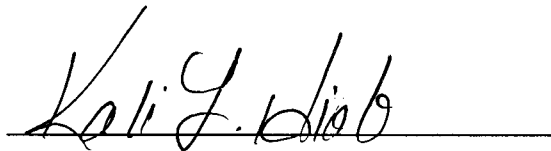

Operator Signature  
N/A STUDENT

Remarks:

cal check  
D.DBOAC

Form 106-I8000




Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-003069  
Location = TOXL      8164.14.00 09/16  
05/29/2020      09:40

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	09:40
02 Std. Gas	0.081	09:41
03 Room Air	0.000	09:41
04 Std. Gas	0.080	09:42
05 Room Air	0.000	09:42
06 Std. Gas	0.081	09:43
07 Room Air	0.000	09:43

Lot No = 05620080A1  
Cyl No = 48  
Exp Date = 04/05/2022  
County = 08      Oper No. = 123456

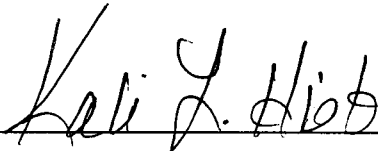  
\_\_\_\_\_  
Operator Signature  
N/A STUDENT

Remarks:      *cal check*  
                      *D.DBO AC*

Form 106-I8000