



INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT
 NORTH DAKOTA OFFICE OF ATTORNEY GENERAL
 CRIME LABORATORY DIVISION-TOXICOLOGY SECTION/BREATH ALCOHOL PROGRAM
 SFN 59281 (06/2018)

Serial Number: 00-004956	Instrument Location: McKenzie County Correctional Facility - WCPD
Reason for Install/Repair	
<input checked="" type="checkbox"/> Install After Receiving From Crime Laboratory <input type="checkbox"/> Install After Location Change <input type="checkbox"/> Other (Specify) _____	

Check When Done:

- 1. Surge Protector Installed/Property Grounded.
- 2. Telephone Line Connected to Intoxilyzer® 8000.
- 3. Breath Tube Heated.
- 4. Enter Preliminary Data (i.e. Date, Time, DST (Y), and Location; Level 2, Function E).
- 5. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 6. Run Tests:
 - A. Print Test (Level 1, Function P).
 - B. ACA Test (Level 1, Function C).
 - C. Radio Frequency Interference (RFI) Test (CMS Mode or Level 1, Function B or C; Key Radio During Test).
- 7. Repair and/or Maintenance Performed (if any): _____
- 8. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 9. File Previous Intoxilyzer® Record (SFN504096, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 10. Send the Following to the Crime Laboratory: Completed Intoxilyzer® 8000 Installation and Repair Checkout (SFN59281, Form 104-G), Print Test, ACA Test, and RFI Test.

Field Inspector Signature: [Signature] #34114, MCSO	Date: 5/12/20
--	----------------------

Crime Laboratory Use Only

This installation has been reviewed and the instrument is approved to be used for the analysis of breath to determine alcohol concentration from the date the Field Inspector performed the installation. This record on file at the Office of Attorney General, Crime Laboratory Division, in the County of Burleigh, North Dakota, is certified to be a true and correct copy of the documents received.

Reviewed/Certified By: [Signature]	Certified Date: 5/29/2020
---	----------------------------------

NOTE: INSTALL DATE FOR THIS INTOXILYZER IS 5/29/2020. THAT IS THE DATE THE RFI TEST COMPLETED THE INSTRUMENT INSTALLATION. **CEE**



INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT
 NORTH DAKOTA OFFICE OF ATTORNEY GENERAL
 CRIME LABORATORY DIVISION-TOXICOLOGY SECTION/BREATH ALCOHOL PROGRAM
 SFN 59281 (06/2018)

Serial Number 80-004956	Instrument Location 1201 12th St. SE, Watford City, ND 58054
Reason for Install/Repair	
<input checked="" type="checkbox"/> Install After Receiving From Crime Laboratory <input type="checkbox"/> Install After Location Change <input checked="" type="checkbox"/> Other (Specify) _____	

Check When Done:

- 1. Surge Protector Installed/Property Grounded.
- 2. Telephone Line Connected to Intoxilyzer® 8000.
- 3. Breath Tube Heated.
- 4. Enter Preliminary Data (i.e. Date, Time, DST (Y), and Location; Level 2, Function E).
- 5. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 6. Run Tests:
 - A. Print Test (Level 1, Function P).
 - B. ACA Test (Level 1, Function C).
 - C. Radio Frequency Interference (RFI) Test (CMS Mode or Level 1, Function B or C; Key Radio During Test).
- 7. Repair and/or Maintenance Performed (if any): _____
- 8. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 9. File Previous Intoxilyzer® Record (SFN504096, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 10. Send the Following to the Crime Laboratory: Completed Intoxilyzer® 8000 Installation and Repair Checkout (SFN59281, Form 104-G), Print Test, ACA Test, and RFI Test.

Field Inspector Signature [Signature]	Date 5/29/20
---	------------------------

Crime Laboratory Use Only

This installation has been reviewed and the instrument is approved to be used for the analysis of breath to determine alcohol concentration from the date the Field Inspector performed the installation. This record on file at the Office of Attorney General, Crime Laboratory Division, in the County of Burleigh, North Dakota, is certified to be a true and correct copy of the documents received.

Reviewed/Certified By [Signature]	Certified Date 5/29/20
---	----------------------------------

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004956
Location = WCPD 8164.14.00 09/16
05/12/2020 14:59

***** Printer Test *****

abcdefghijklmnopqrstuvwxyz1234567890- = |
ABCDEFGHIJKLMNPOQRSTUVWXYZ!@#\$\$%^&*()_+?

abcdefghijklmnopqrstuvwxyz1234567890- = |
ABCDEFGHIJKLMNPOQRSTUVWXYZ!@#\$\$%^&*()_+?

Current Instrument Setup

Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 17618080A1
Standard Cyl #? 29
Standard Expiration? 08/05/2020
Oper No? 132007

Flow Cal. Date: 06/13/2019
Slope 606
Intercept -586035

IR Calibration Date: 06/13/2019
3um 9um

0th Coef(*100): -19986 -21891
1st Coef(*100): 263289 136002
2nd Coef(*100): 2309 1332
H2O adj(mg/l*10k): 544 502

***** Printer Test End *****

Kari Stuart

Operator Signature
KARI STUART

Remarks:

print test

Form 106-I8000

12
Kari Stuart *May*

20

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004956
Location = WCPD 8164.14.00 09/16
05/12/2020 15:00

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	15:00
02 Std. Gas	0.081	15:00
03 Room Air	0.000	15:01
04 Std. Gas	0.081	15:01
05 Room Air	0.000	15:02
06 Std. Gas	0.081	15:02
07 Room Air	0.000	15:03

Lot No = 17618080A1
Cyl No = 29
Exp Date = 08/05/2020
County = 27 Oper No. = 132007

Kari Stuart

Operator Signature
KARI STUART

Remarks:

ACA test

Form 106-18000

STATE OF NORTH DAKOTA } 93
COUNTY OF WOODEN
I, the undersigned, being a true and correct
owner of the original document on file in this office,
do hereby certify that the above is a true and correct
copy of the original document on file in this office.
12 May 2020
Kari Stuart
Wooden City Police Department

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004956
Location = WCPD 8164.14.00 09/16
05/12/2020 15:07

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	15:07
02 Std. Gas	0.081	15:08
03 Room Air	0.000	15:08
04 Std. Gas	0.081	15:08
05 Room Air	0.000	15:09
06 Std. Gas	0.081	15:09
07 Room Air	0.000	15:10

Lot No = 17618080A1
Cyl No = 29
Exp Date = 08/05/2020
County = 27 Oper No. = 132007

Kari Stuart 2410
Operator Signature
KARI STUART

Remarks: ACA test

Form 106-18000

12 May 20
Kari Stuart

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004956
Location = WCPD 8164.14.00 09/16
05/12/2020 15:20

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	15:20
02 Std. Gas	INT*	15:21
03 Room Air	0.000	15:21

*Invalid Test
Interferent Detected

Lot No = 17618080A1
Cyl No = 29
Exp Date = 08/05/2020
County = 27

Oper No. = 132007

Kari Stuart 3410

Operator Signature
KARI STUART

Remarks:

RFI test

Form 106-18000

*CEE
NOT AN
RFI
TEST.*

*REQUESTED
RFI TEST*

5.18.20 CEE

STATE OF NORTH DAKOTA } 85
COUNTY OF WOODBURGH }
I, *Kari Stuart* *12* *May* *20*
do hereby certify that the above is a true and correct copy of the original as shown to me.

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004956
Location = WCPD 8164.14.00 09/16
05/29/2020 09:32

DRY CAL CHECK

Test	AC	Time
01 Room Air	RFI*	09:32
02 Room Air	0.000	09:32

*Invalid Test
Inhibited - RFI

Lot No = 17618080A1
Cyl No = 29
Exp Date = 08/05/2020
County = 27 Oper No. = 132007

Kari Stuart #3416

Operator Signature
KARI STUART

Remarks:

RFI

Form 106-I8000

STATE OF NORTH DAKOTA
COUNTY OF BEND
251 MAY 20
Kari Stuart #3416