

INTOXILYZER® 8000 INSPECTION

Intoxilyzer® 8000 Serial Number: 80-004197

Location: TOYL

A. General Setup and Checks:

1. Diagnostic Tests Pass and Instrument in Ready Mode
2. Breath tube heated
3. Date, time and location code (Level 2,E). Re-set if necessary.
4. Print test (Level 1,P). Sign and attach test record.
5. Tank monitor (Level 3,D,G). Display and Regulator within 50 psi.
 - a. Display: 742 psi Regulator: 740 psi

B. Tests (Sign and attach test records):

Configure simulator for the following tests (Level 1,S).

1. Wet Calibration Check

- a. Low AC (Level 1,C): Known Value \leq 0.03 AC: 0.010 AC
Sim. Ser #: MP5320 Lot #: 201805B Exp. Date: 5.30.20
- b. High AC (Level 1,C): Known Value \geq 0.25 AC: 0.250 AC
Sim. Ser #: MP5290 Lot #: 201911B Exp. Date: 11.5.21

Configure dry gas standard for the following tests (Level 1,S).

2. Interferent Check (Level 1,B): Known Value: 0.10 AC + 0.05% Acetone
Sim. Ser #: DR734B Lot #: 1C55 Exp. Date: N/A
 Display reads "Interferent Detect"
3. RFI Check (CMS Mode) Display reads "RFI Detect"
4. Dry Calibration Check (Level 1,C): Known Value 0.080 AC
Gas Cyl Lot #: 05620080A1 Cyl #: 04B Exp. Date: 4.5.2022

Test 1 <u>0.078</u>	Test 4 <u>0.080</u>	Test 7 <u>0.079</u>
Test 2 <u>0.078</u>	Test 5 <u>0.078</u>	Test 8 <u>0.078</u>
Test 3 <u>0.078</u>	Test 6 <u>0.078</u>	Test 9 <u>0.078</u>
Average <u>0.078</u>		

C. Remarks/Maintenance: Annual Inspection

Classroom only, flow sensor issue.

TRAINING FOR KALI HEB. CEE

Instrument is acceptable to be used in the field.

Breath Analyst Signature

Reviewed by

Date

Date

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004197
Location = TOXL 8164.14.00 09/16
06/03/2020 15:34

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	15:34
02 Std. Sol.	0.009	15:36
03 Room Air	0.000	15:36
04 Std. Sol.	0.010	15:37
05 Room Air	0.000	15:37
06 Std. Sol.	0.010	15:38
07 Room Air	0.000	15:38

08 Sim Temp = 34.0°C

Simul Ser No = MP5320
Std Sol No = 201805B
County = 08 Oper No. = 123456

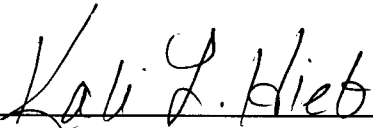

Operator Signature
N/A STUDENT

Remarks: cal check 0.010AC
 low AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004197
Location = TOXL 8164.14.00 09/16
06/03/2020 15:41

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	15:41
02 Std. Sol.	0.249	15:42
03 Room Air	0.000	15:43
04 Std. Sol.	0.250	15:44
05 Room Air	0.000	15:44
06 Std. Sol.	0.249	15:45
07 Room Air	0.000	15:46

08 Sim Temp = 34.0°C

Simul Ser No = MP5290
Std Sol No = 201911B
County = 08 Oper No. = 123456

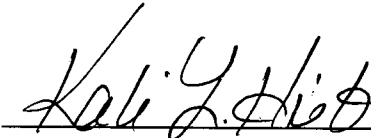

Operator Signature
N/A STUDENT

Remarks: *High AC 0.250 AC*
Cal check

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004197
Location = TOXL 8164.14.00 09/16
06/03/2020 15:52

Test	AC	Time
01 Diagnostic	OK	15:53
02 Room Air	RFI*	15:54
03 Room Air	0.000	15:54

*Invalid Test
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS821456
Lot No = 05620080A1
Cyl No = 048
Expiration Date = 04/05/2022
County = 08 Oper No. = 123456

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

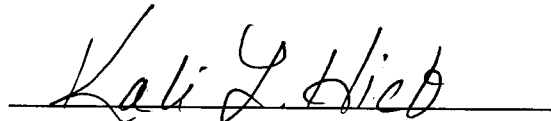

Operator Signature
N/A STUDENT

Remarks:

RFI Check

Form 106-I8000


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004197
Location = TOXL 8164.14.00 09/16
06/03/2020 15:55

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	15:56
02 Std. Gas	0.078	15:56
03 Room Air	0.000	15:56
04 Std. Gas	0.078	15:57
05 Room Air	0.000	15:57
06 Std. Gas	0.078	15:58
07 Room Air	0.000	15:58

Lot No = 05620080A1
Cyl No = 048
Exp Date = 04/05/2022
County = 08 Oper No. = 123456

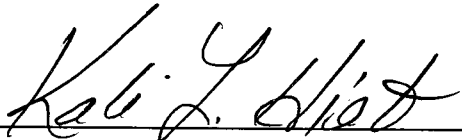
Operator Signature
N/A STUDENT

Remarks: *Cal check 0.000 AC*

Form 106-I8000


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004197
Location = TOXL 8164.14.00 09/16
06/03/2020 15:59

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	15:59
02 Std. Gas	0.080	15:59
03 Room Air	0.000	16:00
04 Std. Gas	0.078	16:00
05 Room Air	0.000	16:01
06 Std. Gas	0.078	16:01
07 Room Air	0.000	16:02

Lot No = 05620080A1
Cyl No = 048
Exp Date = 04/05/2022
County = 08 Oper No. = 123456

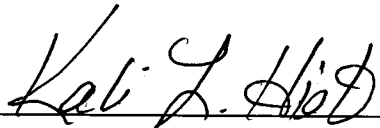
Operator Signature
N/A STUDENT

Remarks: *Cal check 0.000 AC*

Form 106-I8000


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004197
Location = TOXL 8164.14.00 09/16
06/03/2020 16:03

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	16:03
02 Std. Gas	0.079	16:04
03 Room Air	0.000	16:04
04 Std. Gas	0.078	16:05
05 Room Air	0.000	16:05
06 Std. Gas	0.078	16:05
07 Room Air	0.000	16:06

Lot No = 05620080A1
Cyl No = 048
Exp Date = 04/05/2022
County = 08 Oper No. = 123456

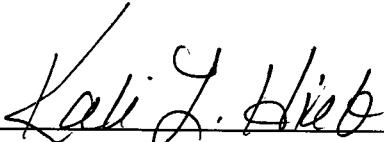

Operator Signature
N/A STUDENT

Remarks: *Cal check 0.000 AC*

Form 106-18000