Toxicology Section/Breath Alcohol Program Intoxilyzer® 8000 Inspection

INTOXILYZER® 8000 INSPECTION

Intoxilyzer[®] 8000 Serial Number: 80-006499

Location: TOXL

BrW-004

- General Setup and Checks: Α.
 - X Diagnostic Tests Pass and Instrument in Ready Mode 1
 - 2. Breath tube heated
 - ZDate, time and location code (Level 2,E). Re-set if necessary. 3.
 - Print test (Level 1, P). Sign and attach test record. 4.
 - Tank monitor (Level 3,D,G). Display and Regulator within 50 psi. 5
 - Display: 340 psi Regulator: 350psi a
- Β. Tests (Sign and attach test records):

X Configure simulator for the following tests (Level 1,S).

- Wet Calibration Check
 - Δ Low AC (Level 1,C): Known Value ≤ 0.03 AC: 0.020 AC a. Sim. Ser #: M73061 Lot #: 201810D Exp. Date: 10.24.20
 - High AC (Level 1,C): Known Value ≥ 0.25 AC: 0.300 AC b. Sim. Ser #: MP3069 Lot #: 201902D Exp. Date: 2.20.21

 \square Configure dry gas standard for the following tests (Level 1,S).

- SInterferent Check (Level 1,B): Known Value: 0.10 AC + 0.05% Acetone 2. Sim. Ser #: DR3451 Lot #: ICS 6 Exp. Date: NA SDisplay reads "Interferent Detect"
- Display reads "RFI Detect" RFI Check (CMS Mode) 3.
- X Dry Calibration Check (Level 1,C): Known Value 0.080 AC 4. Gas Cyl Lot #: 05620 080A1 Cyl #: 34 Exp. Date: 4.5.22 Test 1 0.081 Test 4 0.080 Test 7 0.081Test 2 0.080 Test 5 0.081 Test 8 0.080

Test 2 0.000	Test 5 0.007	Test 0 0.000
Test 3 0-080	Test 6 0.08 /	Test 90.081
Average 0.081		

Remarks/Maintenance: <u>ANNUAL</u> INSPECTION NEW BREATH HOSE O-RING C.

Anstrument is acceptable to be used in the field.

Breath Analyst Signature

Reviewed by

6.27.20 e Date

Date

Intoxilyzer Test Record and Checklist NDOAG Crime Lab. Div., Bismarck, ND 58501 CMI, Inc. Intoxilyzer Alcohol Analyzer North Dakota Model 8000 SN 80-006494 Location = TOXL 8164.14.00 09/16 06/29/2020 11:22 abcdefghijklmnopqrstuvwxyz1234567890-=| ABCDEFGHIJKLMNOPQRSTUVWXYZ!@#\$%^&*() +? abcdefghijklmnopqrstuvwxyz1234567890-=| ABCDEFGHIJKLMNOPQRSTUVWXYZ!@#\$%^&*() +? Current Instrument Setup Data Entry Mode: Enabled Start Test Sequence: Display Prelim Rslt? DABACABA Yes Display Third Digit? Yes Inhib Printer(Y/N)? No Display Volume? No Disable On Memfull? Yes # of Print Copies? 1 Select Std (D/W/I)? Dry Standard Value? 0.080 Standard Lot #? 05620080A1 Standard Cyl #? 34 04/05/2022 Standard Expiration? Oper No? 666666 Flow Cal. Date: 08/18/2015 Slope 655 Intercept -624238 IR Calibration Date: 05/10/2017 3um 9um -------0th Coef(*100):-289691st Coef(*100):2565892nd Coef(*100):2527 -21478140860 1330 H2O adj(mg/l*10k): 655 473 ********** Printer Test End **********

alle

Operator Signature CHARLES EDER

Remarks:

PIALT

Form 106-18000

Intoxilyzer Test Record and Checklist NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. IntoxilyzerAlcohol AnalyzerNorth Dakota Model 8000SN 80-006494Location = TOXL8164.14.00 09/1606/29/202011:35

	WET CAL CHECK	
Test	AC	Time
01 Room Air 02 Std. Sol. 03 Room Air 04 Std. Sol. 05 Room Air 06 Std. Sol. 07 Room Air	0.000 0.017 0.000 0.017 0.000 0.018 0.000	11:36 11:36 11:37 11:38 11:38 11:39 11:40

08 Sim Temp = 34.0°C

in

Operator Signature CHARLES EDER

Low AC

0.020 AC

Form 106-18000

Intoxilyzer Test Record and Checklist NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. IntoxilyzerAlcohol AnalyzerNorth Dakota Model 8000SN 80-006494Location = TOXL8164.14.00 09/1606/29/202011:41

_		
	WET CAL CHECK	
Test	AC	Time
01 Room Air 02 Std. Sol. 03 Room Air 04 Std. Sol. 05 Room Air 06 Std. Sol. 07 Room Air	0.000 0.300 0.000 0.299 0.000 0.298 0.000	11:41 11:42 11:43 11:43 11:44 11:44 11:45 11:45

08 Sim Temp = 34.0°C

Simul Ser No = MP3069 Std Sol No = 201902D County = 08 Oper No. = 666666

Ama Ele

Operator Signature CHARLES EDER

HIGH AC

0.300AC

Form 106-18000

Intoxilyzer Test Record and Checklist NDOAG Crime Lab. Div., Bismarck, ND 58501 CMI, Inc. Intoxilyzer Alcohol Analyzer North Dakota Model 8000 SN 80-006494 Location = TOXL 8164.14.00 09/16 06/29/2020 11:46 Test AC Time 01 Room Air 0.000 11:47 02 *Subject Test INT* 11:48 03 Room Air 0.000 11:48 *Invalid Test Interferent Detected Sub Name = DISCOVER, THE SPIRIT Sub DOB = 02/01/1992Sub Sex = Female Weight = 150Test = DUI Cit = INTERFERENT Dr. Lic. = ND/DIS921456Lot No = 05620080A1Cyl No = 34Expiration Date = 04/05/2022

Operator Signature CHARLES EDER

Remarks:

County = 08

A Check .05% ACETONE 0.10AC +

Oper No. = 666666

Form 106-18000

Intoxilyzer Test Record and Checklist NDOAG Crime Lab. Div., Bismarck, ND 58501 CMI, Inc. Intoxilyzer Alcohol Analyzer North Dakota Model 8000 SN 80-006494 Location = TOXL 8164.14.00 09/16 06/29/2020 11:49 Test AC Time 01 Diagnostic OK 11:50 02 Room Air 0.000 11:50 03 *Subject Test RFI* 11:51 04 Room Air 0.000 11:51 *Invalid Test Inhibited - RFI Sub Name = DISCOVER, THE SPIRIT Sub DOB = 02/01/1992Sub Sex = Female Weight = 150Test = DUI Cit = RFI CHECK Dr. Lic. = ND/DIS921456Lot No = 05620080A1Cyl No = 34Expiration Date = 04/05/2022County = 08Oper No. = 666666

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

Operator Signature CHARLES EDER

Remarks:

Form 106-18000

Intoxilyzer Test Record and Checklist NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer North Dakota Model 8000 SN 80-006494 Location = TOXL 8164.14.00 09/16 06/29/2020 11:51

	DRY CAL CHECK	
Test	AC	Time
01 Room Air 02 Std. Gas 03 Room Air 04 Std. Gas 05 Room Air 06 Std. Gas 07 Room Air	0.000 0.081 0.000 0.080 0.000 0.080 0.080	11:52 11:52 11:53 11:53 11:54 11:54 11:55

Lot No = 05620080A1Cyl No = 34Exp Date = 04/05/2022County = 08Oper No. = 666666

Operator Signature CHARLES EDER ALIBRATION CHECK 0.080 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer North Dakota Model 8000 SN 80-006494 Location = TOXL 8164.14.00 09/16 06/29/2020 11:55

	DRY CAL CHECK	
Test	AC	Time
01 Room Air 02 Std. Gas 03 Room Air 04 Std. Gas 05 Room Air 06 Std. Gas 07 Room Air	0.000 0.080 0.000 0.081 0.000 0.081 0.000	11:55 11:56 11:56 11:57 11:57 11:58 11:58

Lot No = 05620080A1 Cyl No = 34Exp Date = 04/05/2022County = 08

Oper No. = 666666

in

Operator Signature

erator Signature CHARLES EDER HIBCATTON CHECK 0-080AC

Form 106-18000

Intoxilyzer Test Record and Checklist NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. IntoxilyzerAlcohol AnalyzerNorth Dakota Model 8000SN 80-006494Location = TOXL8164.14.00 09/1606/29/202011:58

[DRY CAL CHECK	
Test	AC	Time
01 Room Air 02 Std. Gas 03 Room Air 04 Std. Gas 05 Room Air 06 Std. Gas 07 Room Air	0.000 0.081 0.000 0.080 0.000 0.081 0.000	11:59 11:59 12:00 12:00 12:01 12:01 12:02

Lot No = 05620080A1Cyl No = 34 Exp Date = 04/05/2022County = 08

Oper No. = 666666

ul le 2

Operator Signature CHARLES EDER

NECK 0.0BOAC

Form 106-18000