



ETHANOL GAS STANDARD CYLINDER REPORT
 NORTH DAKOTA OFFICE OF ATTORNEY GENERAL
 CRIME LABORATORY DIVISION-TOXICOLOGY SECTION/BREATH ALCOHOL PROGRAM
 SFN 59282 (08/2018)

Chemical Test Operator Name (Print) <i>Joshua G. Fastnacht</i>		
Location <i>Lauda County Sheriff's Office (LAMO)</i>	Intoxilyzer® Serial Number <i>80-006491</i>	
Gas Lot Number <i>05620080A1</i>	Gas Cylinder Number <i>012</i>	Gas Expiration Date <i>4/5/2022</i>

Check When Done:

- 1. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 2. Perform an ACA Test (Level 1, Function C).
 Write in the ACA Test Results (Reported to 3 Digits, Ex. 0.081).
 A. *0.028* AC
 B. *0.027* AC
 C. *0.028* AC
- 3. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 4. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 5. Send the Following to the Crime Laboratory:
 - A. Ethanol Gas Standard Cylinder Report (SFN59282, Form 105-G).
 - B. ACA Test Record.

Chemical Test Operator Signature <i>JG Fastnacht 4372</i>	Date <i>7-2-2020</i>
Reviewed By (Crime Laboratory Use Only) <i>Charles E. Ehl</i>	Date <i>7.14.2020</i>

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006491
Location = LAMO 8164.14.00 09/16
07/02/2020 18:17

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	18:17
02 Std. Gas	0.078	18:18
03 Room Air	0.000	18:18
04 Std. Gas	0.077	18:19
05 Room Air	0.000	18:19
06 Std. Gas	0.078	18:20
07 Room Air	0.000	18:20

Lot No = 05620080A1
Cyl No = 12
Exp Date = 04/05/2022
County = 23 Oper No. = 132539


Operator Signature
JOSHUA FASTNACHT

Remarks:

Form 106-I8000