

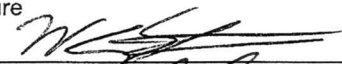
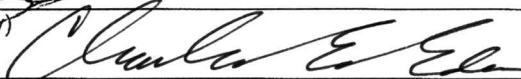


ETHANOL GAS STANDARD CYLINDER REPORT
 NORTH DAKOTA OFFICE OF ATTORNEY GENERAL
 CRIME LABORATORY DIVISION-TOXICOLOGY SECTION/BREATH ALCOHOL PROGRAM
 SFN 59282 (08/2018)

Chemical Test Operator Name (Print) Wesley Straight		
Location Cooperstown		Intoxilyzer® Serial Number 80-004952
Gas Lot Number 24119080A1	Gas Cylinder Number 4	Gas Expiration Date 11/5/21

Check When Done:

- 1. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 2. Perform an ACA Test (Level 1, Function C).
 Write in the ACA Test Results (Reported to 3 Digits, Ex. 0.081).
 - A. 0.079 AC
 - B. 0.078 AC
 - C. 0.080 AC
- 3. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 4. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 5. Send the Following to the Crime Laboratory:
 - A. Ethanol Gas Standard Cylinder Report (SFN59282, Form 105-G).
 - B. ACA Test Record.

Chemical Test Operator Signature 	Date 7/10/20
Reviewed By (Crime Laboratory Use Only) 	Date 7.15.2020

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004952
Location = COOP 8164.14.00 09/16
07/10/2020 16:02

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	16:03
02 Std. Gas	0.079	16:03
03 Room Air	0.000	16:04
04 Std. Gas	0.078	16:04
05 Room Air	0.000	16:05
06 Std. Gas	0.080	16:05
07 Room Air	0.000	16:06

Lot No = 24119080A1
Cyl No = 4
Exp Date = 11/05/2021
County = 20 Oper No. = 061123



Operator Signature
WESLEY STRAIGHT

Remarks:

Form 106-I8000



ETHANOL GAS STANDARD CYLINDER REPORT
 NORTH DAKOTA OFFICE OF ATTORNEY GENERAL
 CRIME LABORATORY DIVISION-TOXICOLOGY SECTION/BREATH ALCOHOL PROGRAM
 SFN 59282 (08/2018)

Chemical Test Operator Name (Print) <i>Wasley Straight</i>		
Location <i>Coop</i>	Intoxilyzer® Serial Number <i>80-004952</i>	
Gas Lot Number <i>24119080A1</i>	Gas Cylinder Number <i>4</i>	Gas Expiration Date <i>11/5/21</i>

Check When Done:

- 1. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 2. Perform an ACA Test (Level 1, Function C).
 Write in the ACA Test Results (Reported to 3 Digits, Ex. 0.081).
 A. *0.079* AC
 B. *0.078* AC
 C. *0.080* AC
- 3. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 4. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 5. Send the Following to the Crime Laboratory:
 A. Ethanol Gas Standard Cylinder Report (SFN59282, Form 105-G).
 B. ACA Test Record.

Chemical Test Operator Signature <i>[Signature]</i>	Date <i>7/9/21</i>
Reviewed By (Crime Laboratory Use Only)	Date

NOTE:

EMAILED WES STRAIGHT ON 7.14.20
 TO CORRECT DATE ON FORM 105-G.

CEE 7.15.20