



**ETHANOL GAS STANDARD CYLINDER REPORT**  
 NORTH DAKOTA OFFICE OF ATTORNEY GENERAL  
 CRIME LABORATORY DIVISION-TOXICOLOGY SECTION/BREATH ALCOHOL PROGRAM  
 SFN 59282 (08/2018)

Chemical Test Operator Name (Print) <i>Beaver, Michael P</i>		
Location <i>WCSO</i>	Intoxilyzer® Serial Number <i>80-006504</i>	
Gas Lot Number <i>24119080A1</i>	Gas Cylinder Number <i>12</i>	Gas Expiration Date <i>11-05-21</i>

Check When Done:

- 1. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 2. Perform an ACA Test (Level 1, Function C).  
 Write in the ACA Test Results (Reported to 3 Digits, Ex. 0.081).  
 A. *0.079* AC  
 B. *0.080* AC  
 C. *0.079* AC
- 3. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 4. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 5. Send the Following to the Crime Laboratory:  
 A. Ethanol Gas Standard Cylinder Report (SFN59282, Form 105-G).  
 B. ACA Test Record.

Chemical Test Operator Signature <i>[Signature]</i>	Date <i>14-Jul-2020</i>
Reviewed By (Crime Laboratory Use Only) <i>[Signature]</i>	Date <i>7-17-2020</i>

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-006504  
Location = WESO      8164.14.00 09/16  
07/14/2020      17:09

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	17:09
02 Std. Gas	0.079	17:10
03 Room Air	0.000	17:10
04 Std. Gas	0.080	17:11
05 Room Air	0.000	17:11
06 Std. Gas	0.079	17:12
07 Room Air	0.000	17:12

Lot No = 24119080A1  
Cyl No = 12  
Exp Date = 11/05/2021  
County = 52      Oper No. = 131495



Operator Signature  
MICHAEL BEAVER

Remarks:

Form 106-I8000