



ETHANOL GAS STANDARD CYLINDER REPORT
 NORTH DAKOTA OFFICE OF ATTORNEY GENERAL
 CRIME LABORATORY DIVISION-TOXICOLOGY SECTION/BREATH ALCOHOL PROGRAM
 SFN 59282 (08/2018)

Chemical Test Operator Name (Print) <i>Allen, Kathryn M.</i>		
Location <i>Cass County Jail-Sheriff's Office</i>	Intoxilyzer® Serial Number <i>80-006496</i>	
Gas Lot Number <i>34418080 A2</i>	Gas Cylinder Number <i>1</i>	Gas Expiration Date <i>02-05-2021</i>

Check When Done:

- 1. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 2. Perform an ACA Test (Level 1, Function C).
 Write in the ACA Test Results (Reported to 3 Digits, Ex. 0.081).
 A. 0.081 AC
 B. 0.081 AC
 C. 0.081 AC
- 3. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 4. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 5. Send the Following to the Crime Laboratory:
 A. Ethanol Gas Standard Cylinder Report (SFN59282, Form 105-G).
 B. ACA Test Record.

Chemical Test Operator Signature <i>Kathryn M. Allen #3671</i>	Date <i>7/30/20</i>
Reviewed By (Crime Laboratory Use Only) <i>Charles E. Ehl</i>	Date <i>7/31/20</i>


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006496
Location = CASS 8164.13.00 06/09
07/30/2020 17:31

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	17:31
02 Std. Gas	0.081	17:31
03 Room Air	0.000	17:32
04 Std. Gas	0.081	17:32
05 Room Air	0.000	17:33
06 Std. Gas	0.081	17:33
07 Room Air	0.000	17:34

Lot No = 34418080A2
Cyl No = 1
Exp Date = 02/05/2021
County = 09 Oper No. = 090801


Operator Signature
KATHRYN ALLEN

Remarks:

Good Inst of new canister

Form 106-I8000