



**ETHANOL GAS STANDARD CYLINDER REPORT**  
 NORTH DAKOTA OFFICE OF ATTORNEY GENERAL  
 CRIME LABORATORY DIVISION-TOXICOLOGY SECTION/BREATH ALCOHOL PROGRAM  
 SFN 59282 (08/2018)

Chemical Test Operator Name (Print) <b>TRONSET, JOHN F</b>		
Location <b>Stanley Municipal City Corrections</b>	Intoxilyzer® Serial Number <b>80-005357</b>	
Gas Lot Number <b>05620080A1</b>	Gas Cylinder Number <b>5</b>	Gas Expiration Date <b>04/05/2022</b>

Check When Done:

- 1. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 2. Perform an ACA Test (Level 1, Function C).  
 Write in the ACA Test Results (Reported to 3 Digits, Ex. 0.081).  
 A. **0.080** AC  
 B. **0.082** AC  
 C. **0.080** AC
- 3. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 4. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 5. Send the Following to the Crime Laboratory:  
 A. Ethanol Gas Standard Cylinder Report (SFN59282, Form 105-G).  
 B. ACA Test Record.

Chemical Test Operator Signature <i>John F. Tronset</i> SPD 1235	Date <b>08-22-2020</b>
Reviewed By (Crime Laboratory Use Only) <i>Charles E. Ehl</i>	Date <b>10.19.2020</b>

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-005357  
Location = MOSO      8164.14.00 09/16  
08/22/2020      04:41

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	04:42
02 Std. Gas	0.080	04:42
03 Room Air	0.000	04:42
04 Std. Gas	0.082	04:43
05 Room Air	0.000	04:43
06 Std. Gas	0.080	04:44
07 Room Air	0.000	04:44

Lot No = 05620080A1  
Cyl No = 5  
Exp Date = 04/05/2022  
County = 31      Oper No. = 031123

  
Operator Signature      SAD 1235  
JOHN TRONSET

Remarks:

Form 106-I8000