

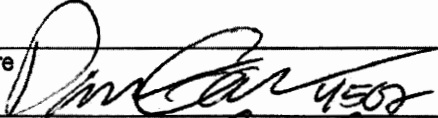
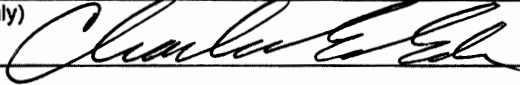


ETHANOL GAS STANDARD CYLINDER REPORT
 NORTH DAKOTA OFFICE OF ATTORNEY GENERAL
 CRIME LABORATORY DIVISION-TOXICOLOGY SECTION/BREATH ALCOHOL PROGRAM
 SFN 59282 (08/2018)

Chemical Test Operator Name (Print) Garvens, Daniel C		
Location CAVA	Intoxilyzer® Serial Number 80-005937	
Gas Lot Number 05620080A1	Gas Cylinder Number 2	Gas Expiration Date 04/05/2022

Check When Done:

- 1. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 2. Perform an ACA Test (Level 1, Function C).
 Write in the ACA Test Results (Reported to 3 Digits, Ex. 0.081).
 A. 0.0 80 AC
 B. 0.0 79 AC
 C. 0.0 80 AC
- 3. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 4. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 5. Send the Following to the Crime Laboratory:
 - A. Ethanol Gas Standard Cylinder Report (SFN59282, Form 105-G).
 - B. ACA Test Record.

Chemical Test Operator Signature 	Date 1/4/2021
Reviewed By (Crime Laboratory Use Only) 	Date 1/5/2021

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005937
Location = CAVA 8164.14.00 09/16
01/04/2021 10:13

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:14
02 Std. Gas	0.080	10:14
03 Room Air	0.000	10:14
04 Std. Gas	0.079	10:15
05 Room Air	0.000	10:15
06 Std. Gas	0.080	10:16
07 Room Air	0.000	10:16

Lot No = 05620080A1
Cyl No = 2
Exp Date = 04/05/2022
County = 34 Oper No. = 060849



Operator Signature
DANIEL GARVENS

Remarks:

New Cylinder Install - ACA Test

Form 106-I8000