OFFICE OF ATTORNEY GENERAL CRIME LABORATORY DIVISION

Toxicology Section/Breath Alcohol Program Intoxilyzer® 8000 Inspection

INTOXILYZER® 8000 INSPECTION

ntoxilyzer® 8000 Serial Number: <u>80-00594</u> 4	Location: TOXL
A. General Setup and Checks: 1. Diagnostic Tests Pass and Instrument in Ro 2. Breath tube heated 3. Date, time and location code (Level 2,E). F 4. Print test (Level 1,P). Sign and attach test 5. Tank monitor (Level 3,D,G). Display and R a. Display: 772 psi Regulator: /// ps	Re-set if necessary.
B. Tests (Sign and attach test records): Configure simulator for the following tests (Level 1. Wet Calibration Check a. Low AC (Level 1,C): Known Value Sim. Ser #: MF5319 Lot #: 201911 b. High AC (Level 1,C): Known Value Sim. Ser #: MF 3062 Lot #: 2021 63	≤ 0.03 AC:
2. Interferent Check (Level 1,B): Known Value Sim. Ser #: DR345/ Lot #: FCS 6 Explain Explain Explain Explain Explain Explain Sim. Ser #: DR345/ Lot #: FCS 6 Explain	te: 0.10 AC + 0.05% Acetone of Date: 1.4 Peads "RFI Detect" Value 1.0 AC AC Exp. Date: 1.5 AC Test 7 1.0 AC 1.0 AC Test 8 1.0 AC 1.0
C. Remarks/Maintenance: <u>ANNUAL INSPECTA</u> External Battery	ON, REPLACED
Instrument is acceptable to be used in the field. Breath Aralyst Signature Reviewed by	4.6.21 Date 4-6-21 Date

02/18

Issuing Authority - CEE

Page 1 of 1

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005944
Location = TOXL 8164.14.00 09/16
04/06/2021 15:42

********* Printer Test ********

abcdefghijklmnopqrstuvwxyz1234567890-=|
ABCDEFGHIJKLMNOPQRSTUVWXYZ!@#\$%^&*() +?

abcdefghijklmnopqrstuvwxyz1234567890-=|
ABCDEFGHIJKLMNOPQRSTUVWXYZ!@#\$%^&*()_+?

Current Instrument Setup

Data Entry Mode: Enabled Start Test Sequence: DABACABA Display Prelim Rslt? Yes Display Third Digit? Yes Inhib Printer(Y/N)? No Display Volume? No Disable On Memfull? Yes # of Print Copies? Select Std (D/W/I)? Dry Standard Value? 0.080 Standard Lot #? 24119080A1 Standard Cyl #? 11/05/2021 Standard Expiration? Oper No? 666666

Flow Cal. Date: 06/20/2013 Slope 677 Intercept -702620

IR Calibration Date: 06/09/2017
3um 9um

Oth Coef(*100): -16158 -17815
1st Coef(*100): 266780 136962
2nd Coef(*100): 1117 1089
H2O adj(mg/l*10k): 317 364

****** Printer Test End *******

Operator Signature CHARLES EDER

Remarks:

CMI, Inc. Intoxilyzer
North Dakota Model 8000
Location = TOXL
04/06/2021

SN 80-005944 8164.14.00 09/16 15:42

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	15:43
02 Std. Sol.	0.016	15:44
03 Room Air	0.000	15:44
04 Std. Sol.	0.016	15:45
05 Room Air	0.000	15:46
06 Std. Sol.	0.015	15:46
07 Room Air	0.000	15:47

 $08 \text{ Sim Temp} = 34.0^{\circ}\text{C}$

Simul Ser No = MP5319 Std Sol No = 201911C

County = 08

Oper No. = 666666

Operator Signature CHARLES EDER

Remarks:

CMI, Inc. Intoxilyzer
North Dakota Model 8000
Location = TOXL
04/06/2021

SN 80-005944 8164.14.00 09/16 15:50

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	15:51
02 Std. Sol.	0.407	15:51
03 Room Air	0.000	15:52
04 Std. Sol.	0.409	15:53
05 Room Air	0.000	15:53
06 Std. Sol.	0.408	15:54
07 Room Air	0.000	15:54

08 Sim Temp = 34.0°C

Simul Ser No = MP3062 Std Sol No = 202103E

County = 08

Oper No. = 666666

Operator Signature

CHARLES EDER

Remarks:

CMI, Inc. Intoxilyzer North Dakota Model 8000 Location = TOXL 04/06/2021

Alcohol Analyzer SN 80-005944 8164.14.00 09/16 15:56

Test	AC	Time
01 Room Air	0.000	15:57
02 *Subject Test	INT*	15:57
03 Room Air	0.000	15:58

*Invalid Test

Interferent Detected

Sub Name = DISCOVER, THE SPIRIT

Sub DOB = 02/01/1992

Sub Sex = FemaleWeight = 150

Test = DUI Cit = INTERFERENT

Dr. Lic. = ND/DIS921456

Lot No = 24119080A1

Cyl No = 5

Remarks:

Expiration Date = 11/05/2021

Oper No. = 666666 County = 08

Operator Signature

event Check + 0.05% ACETONE CHARLES EDER

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005944
Location = TOXL 8164.14.00 09/16
04/06/2021 15:59

Test	AC	Time
01 Diagnostic	OK	15:59
02 Room Air	0.000	16:00
03 *Subject Test	RFI*	16:00
04 Room Air	0.000	16:00

*Invalid Test Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT

Sub DOB = 02/01/1992

Sub Sex = Female Weight = 150
Test = DUI Cit = RFI CHECK

Dr. Lic. = ND/DIS921456

Lot No = 24119080A1

Cyl No = 5

Expiration Date = 11/05/2021

County = 08 Oper No. = 666666

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

Operator Signature CHARLES EDER

Remarks:

CMI, Inc. Intoxilyzer
North Dakota Model 8000
Location = TOXL
04/06/2021

Alcohol Analyzer SN 80-005944 8164.14.00 09/16 16:01

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	16:01
02 Std. Gas	0.082	16:02
03 Room Air	0.000	16:02
04 Std. Gas	0.082	16:02
05 Room Air	0.000	16:03
06 Std. Gas	0.082	16:03
07 Room Air	0.000	16:04

Lot No = 24119080A1

Cyl No = 5

Exp Date = 11/05/2021

County = 08

Oper No. = 666666

Operator Signature CHARLES EDER

Remarks:

Form 106-I8000

0.080 A

CMI, Inc. Intoxilyzer North Dakota Model 8000 Location = TOXL 04/06/2021

Alcohol Analyzer SN 80-005944 8164.14.00 09/16 16:04

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	16:05
02 Std. Gas	0.083	16:05
03 Room Air	0.000	16:06
04 Std. Gas	0.083	16:06
05 Room Air	0.000	16:06
06 Std. Gas	0.083	16:07
07 Room Air	0.000	16:07

Lot No = 24119080A1

Cyl No = 5

Exp Date = 11/05/2021

County = 08

Oper No. = 666666

Operator Signature CHARLES EDER

Remarks:

ALIBRATION CHECK

CMI, Inc. Intoxilyzer North Dakota Model 8000 Location = TOXL 04/06/2021

Alcohol Analyzer SN 80-005944 8164.14.00 09/16 16:08

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	16:09
02 Std. Gas	0.082	16:09
03 Room Air	0.000	16:10
04 Std. Gas	0.083	16:10
05 Room Air	0.000	16:11
06 Std. Gas	0.083	16:11
07 Room Air	0.000	16:12

Lot No = 24119080A1

Cyl No = 5

Exp Date = 11/05/2021

County = 08

Oper No. = 666666

Operator Signature CHARLES EDER