

INTOXILYZER® 8000 INSPECTION

Intoxilyzer® 8000 Serial Number: 80-004205

Location: TOXL

A. General Setup and Checks:

1.  Diagnostic Tests Pass and Instrument in Ready Mode
2.  Breath tube heated
3.  Date, time and location code (Level 2,E). Re-set if necessary.
4.  Print test (Level 1,P). Sign and attach test record.
5.  Tank monitor (Level 3,D,G). Display and Regulator within 50 psi.
  - a. Display: 407 psi Regulator: 425 psi

B. Tests (Sign and attach test records):

Configure simulator for the following tests (Level 1,S).

1. Wet Calibration Check

- a.  Low AC (Level 1,C): Known Value  $\leq 0.03$  AC: 0.020 AC  
Sim. Ser #: MP3064 Lot #: 20070 Exp. Date: 2.13.22
- b.  High AC (Level 1,C): Known Value  $\geq 0.25$  AC: 0.300 AC  
Sim. Ser #: MP5317 Lot #: 202012A Exp. Date: 12.8.22

Configure dry gas standard for the following tests (Level 1,S).

2.  Interferent Check (Level 1,B): Known Value: 0.10 AC + 0.05% Acetone  
Sim. Ser #: DR3451 Lot #: ICS6 Exp. Date: NA  
 Display reads "Interferent Detect"
3.  RFI Check (CMS Mode)  Display reads "RFI Detect"
4.  Dry Calibration Check (Level 1,C): Known Value 0.080 AC  
Gas Cyl Lot #: 24119080A1 Cyl #: 6 Exp. Date: 11.5.21

Test 1 <u>0.080</u>	Test 4 <u>0.081</u>	Test 7 <u>0.081</u>
Test 2 <u>0.080</u>	Test 5 <u>0.080</u>	Test 8 <u>0.080</u>
Test 3 <u>0.079</u>	Test 6 <u>0.080</u>	Test 9 <u>0.080</u>
Average <u>0.080</u>		

C. Remarks/Maintenance: ANNUAL INSPECTION

CLASSROOM ONLY INSTRUMENT - FLOW SENSOR ISSUE

Instrument is acceptable to be used in the field. NO

Breath Analyst Signature

Date

Reviewed by

Date

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000            SN 80-004205  
Location = TOXL                    8164.14.00 09/16  
04/12/2021                            15:14

\*\*\*\*\* Printer Test \*\*\*\*\*

abcdefghijklmnopqrstuvwxyz1234567890-=  
ABCDEFGHIJKLMNPOQRSTUVWXYZ!@#%^&\*()\_+?

abcdefghijklmnopqrstuvwxyz1234567890-=  
ABCDEFGHIJKLMNPOQRSTUVWXYZ!@#%^&\*()\_+?

Current Instrument Setup

Data Entry Mode:            Enabled  
Start Test Sequence:        DABACABA  
Display Prelim Rslt?        Yes  
Display Third Digit?        Yes  
Inhib Printer(Y/N)?        No  
Display Volume?            No  
Disable On Memfull?        Yes  
# of Print Copies?           1  
Select Std (D/W/I)?        Dry  
Standard Value?            0.080  
Standard Lot #?            24119080A1  
Standard Cyl #?            6  
Standard Expiration?       11/05/2021  
Oper No?                    666666

Flow Cal. Date:            06/14/2019  
Slope                        487  
Intercept                    627463

IR Calibration Date:        06/14/2019

	3um	9um
0th Coef(*100):	-18036	-22537
1st Coef(*100):	272422	133312
2nd Coef(*100):	1354	1047
H2O adj(mg/l*10k):	318	449

\*\*\*\*\* Printer Test End \*\*\*\*\*



Operator Signature  
CHARLES EDER

Remarks:

PRINT TEST

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-004205  
Location = TOXL      8164.14.00 09/16  
04/12/2021      15:16

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	15:16
02 Std. Sol.	0.021	15:17
03 Room Air	0.000	15:18
04 Std. Sol.	0.021	15:18
05 Room Air	0.000	15:19
06 Std. Sol.	0.021	15:20
07 Room Air	0.000	15:20

08 Sim Temp = 34.0°C

Simul Ser No = MP3064  
Std Sol No = 20070  
County = 08      Oper No. = 666666

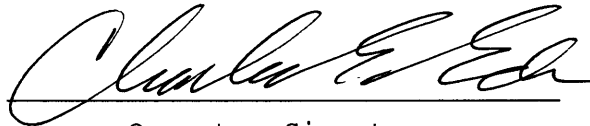

Operator Signature  
CHARLES EDER

Remarks: *Low AC*  
*0.020 AC*

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-004205  
Location = TOXL      8164.14.00 09/16  
04/12/2021      15:22

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	15:22
02 Std. Sol.	0.300	15:23
03 Room Air	0.000	15:24
04 Std. Sol.	0.301	15:24
05 Room Air	0.000	15:25
06 Std. Sol.	0.302	15:26
07 Room Air	0.000	15:26

08 Sim Temp = 34.0°C

Simul Ser No = MP5317  
Std Sol No = 202012A  
County = 08      Oper No. = 666666

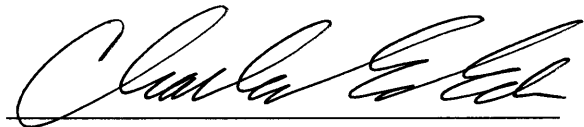

Operator Signature  
CHARLES EDER

Remarks: *HIGH AC*  
*0.300 AC*

Form 106-I8000


Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-004205  
Location = TOXL                8164.14.00 09/16  
04/12/2021                        15:28

Test	AC	Time
01 Room Air	0.000	15:29
02 *Subject Test	INT*	15:29
03 Room Air	0.000	15:30

\*Invalid Test  
Interferent Detected

Sub Name = BILLS, THE BUFFALO  
Sub DOB = 06/14/1993  
Sub Sex = Male                    Weight = NA  
Test = DUI                        Cit = INTERFERENT  
Dr. Lic. = ND/BIL933277  
Lot No = 24119080A1  
Cyl No = 6  
Expiration Date = 11/05/2021  
County = 08                        Oper No. = 666666

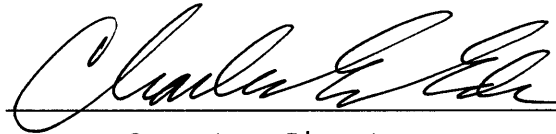

Operator Signature  
CHARLES EDER

Remarks: *Interferent Check*  
*0.10AC + 0.05% ACETONE*

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer                      Alcohol Analyzer  
North Dakota Model 8000                      SN 80-004205  
Location = TOXL                              8164.14.00 09/16  
04/12/2021    15:30

Test	AC	Time
01 Diagnostic	OK	15:31
02 Room Air	0.000	15:32
03 *Subject Test	RFI*	15:32
04 Room Air	0.000	15:32

\*Invalid Test  
Inhibited - RFI

Sub Name = BILLS, THE BUFFALO  
Sub DOB = 06/14/1993  
Sub Sex = Male                              Weight = NA  
Test = DUI                                      Cit = RFI CHECK  
Dr. Lic. = ND/BIL933277  
Lot No = 24119080A1  
Cyl No = 6  
Expiration Date = 11/05/2021  
County = 08                              Oper No. = 666666

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

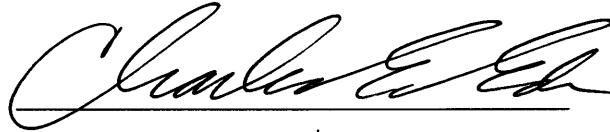

Operator Signature  
CHARLES EDER

Remarks: *RFI CHECK*

Form 106-I8000


Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-004205  
Location = TOXL      8164.14.00 09/16  
04/12/2021      15:32

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	15:33
02 Std. Gas	0.080	15:33
03 Room Air	0.000	15:34
04 Std. Gas	0.080	15:34
05 Room Air	0.000	15:35
06 Std. Gas	0.079	15:35
07 Room Air	0.000	15:36

Lot No = 24119080A1  
Cyl No = 6  
Exp Date = 11/05/2021  
County = 08      Oper No. = 666666

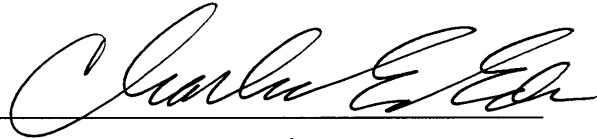

Operator Signature  
CHARLES EDER

Remarks: CALIBRATION CHECK  
0.080 AC

Form 106-I8000


Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-004205  
Location = TOXL      8164.14.00 09/16  
04/12/2021      15:36

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	15:36
02 Std. Gas	0.081	15:37
03 Room Air	0.000	15:37
04 Std. Gas	0.080	15:38
05 Room Air	0.000	15:38
06 Std. Gas	0.080	15:38
07 Room Air	0.000	15:39

Lot No = 24119080A1  
Cyl No = 6  
Exp Date = 11/05/2021  
County = 08      Oper No. = 666666

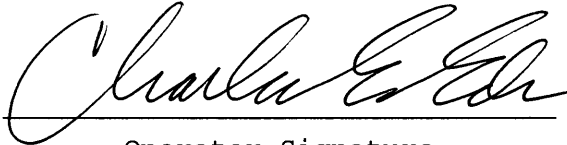

Operator Signature  
CHARLES EDER

Remarks: CALIBRATION CHECK  
0.080 AC

Form 106-I8000



Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-004205  
Location = TOXL      8164.14.00 09/16  
04/12/2021      15:39

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	15:40
02 Std. Gas	0.081	15:40
03 Room Air	0.000	15:41
04 Std. Gas	0.080	15:41
05 Room Air	0.000	15:41
06 Std. Gas	0.080	15:42
07 Room Air	0.000	15:42

Lot No = 24119080A1  
Cyl No = 6  
Exp Date = 11/05/2021  
County = 08      Oper No. = 666666



Operator Signature  
CHARLES EDER

Remarks: CALIBRATION CHECK  
0.080 AC

Form 106-I8000