

INTOXILYZER® 8000 INSPECTION

Intoxilyzer® 8000 Serial Number: 80-006503

Location: ToxL

A. General Setup and Checks:

1. Diagnostic Tests Pass and Instrument in Ready Mode
2. Breath tube heated
3. Date, time and location code (Level 2,E). Re-set if necessary.
4. Print test (Level 1,P). Sign and attach test record.
5. Tank monitor (Level 3,D,G). Display and Regulator within 50 psi.
 - a. Display: 327 psi Regulator: 300 psi

B. Tests (Sign and attach test records):

Configure simulator for the following tests (Level 1,S).

1. Wet Calibration Check 20070
 - a. Low AC (Level 1,C): Known Value ≤ 0.03 AC: 0.020 AC
Sim. Ser #: MP3064 Lot #: FCS6 Exp. Date: 2.13.22
 - b. High AC (Level 1,C): Known Value ≥ 0.25 AC: 0.300 AC
Sim. Ser #: MP5317 Lot #: 202012A Exp. Date: 12.8.22

Configure dry gas standard for the following tests (Level 1,S).

2. Interferent Check (Level 1,B): Known Value: 0.10 AC + 0.05% Acetone
Sim. Ser #: DR3451 Lot #: FCS6 Exp. Date: NA
 Display reads "Interferent Detect"
3. RFI Check (CMS Mode) Display reads "RFI Detect"
4. Dry Calibration Check (Level 1,C): Known Value 0.080 AC
Gas Cyl Lot #: 24119080A1 Cyl #: 6 Exp. Date: 11.5.21

Test 1	<u>0.082</u>	Test 4	<u>0.082</u>	Test 7	<u>0.082</u>
Test 2	<u>0.082</u>	Test 5	<u>0.082</u>	Test 8	<u>0.081</u>
Test 3	<u>0.083</u>	Test 6	<u>0.083</u>	Test 9	<u>0.082</u>
Average <u>0.082</u>					

C. Remarks/Maintenance: ANNUAL INSPECTION, REPLACED SIMULATOR RETURN O-RING,

Instrument is acceptable to be used in the field.

Charles E. Ed
Breath Analyst Signature

4.13.2021
Date

Kari J. Hib
Reviewed by

4-13-2021
Date

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006503
Location = TOXL 8164.14.00 09/16
04/13/2021 12:18

***** Printer Test *****

abcdefghijklmnopqrstuvwxy1234567890-|=|
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&^*()_+?

abcdefghijklmnopqrstuvwxy1234567890-|=|
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&^*()_+?

Current Instrument Setup

Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 24119080A1
Standard Cyl #? 6
Standard Expiration? 11/05/2021
Oper No? 666666

Flow Cal. Date: 08/20/2015
Slope 663
Intercept -633300

IR Calibration Date: 06/12/2017
 3um 9um

0th Coef(*100): -6741 -15632
1st Coef(*100): 256583 132194
2nd Coef(*100): 3028 1131
H2O adj(mg/l*10k): 354 358

***** Printer Test End *****



Operator Signature
CHARLES EDER

Remarks:

PRINT TEST

Form 106-I8000


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006503
Location = TOXL 8164.14.00 09/16
04/13/2021 12:32

Test	AC	Time
01 Room Air	0.000	12:33
02 *Subject Test	INT*	12:34
03 Room Air	0.000	12:35

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 02/01/1992
Sub Sex = Female Weight = 150
Test = DUI Cit = INTERFERENT
Dr. Lic. = ND/DIS921456
Lot No = 24119080A1
Cyl No = 6
Expiration Date = 11/05/2021
County = 08 Oper No. = 666666

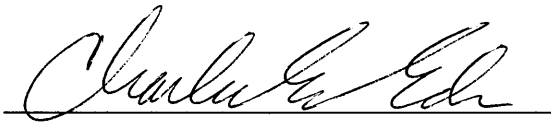

Operator Signature
CHARLES EDER

Remarks: *Interferent Check*
0.10AE + 0.05% ACETONE
Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006503
Location = TOXL 8164.14.00 09/16
04/13/2021 12:36

Test	AC	Time
01 Diagnostic	OK	12:37
02 Room Air	0.000	12:37
03 *Subject Test	RFI*	12:38
04 Room Air	0.000	12:38

*Invalid Test
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 02/01/1992
Sub Sex = Female Weight = 150
Test = DUI Cit = RFI CHECK
Dr. Lic. = ND/DIS921456
Lot No = 24119080A1
Cyl No = 6
Expiration Date = 11/05/2021
County = 08 Oper No. = 666666

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

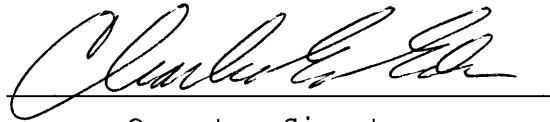

Operator Signature
CHARLES EDER

Remarks: RFI CHECK

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006503
Location = TOXL 8164.14.00 09/16
04/13/2021 12:39

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	12:39
02 Std. Gas	0.082	12:40
03 Room Air	0.000	12:40
04 Std. Gas	0.082	12:40
05 Room Air	0.000	12:41
06 Std. Gas	0.083	12:41
07 Room Air	0.000	12:42

Lot No = 24119080A1
Cyl No = 6
Exp Date = 11/05/2021
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: CALIBRATION CHECK
0.080 AC

Form 106-I8000


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006503
Location = TOXL 8164.14.00 09/16
04/13/2021 12:42

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	12:43
02 Std. Gas	0.082	12:43
03 Room Air	0.000	12:44
04 Std. Gas	0.082	12:44
05 Room Air	0.000	12:45
06 Std. Gas	0.083	12:45
07 Room Air	0.000	12:45

Lot No = 24119080A1
Cyl No = 6
Exp Date = 11/05/2021
County = 08 Oper No. = 666666

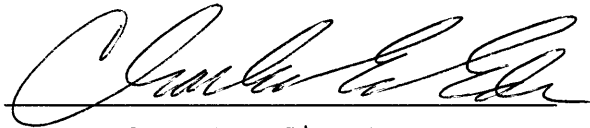

Operator Signature
CHARLES EDER

Remarks: CALIBRATION CHECK
0.080 AC

Form 106-I8000

