

INTOXILYZER® 8000 INSPECTION

Intoxilyzer® 8000 Serial Number: 80-004953

Location: TOXL

A. General Setup and Checks:

1.  Diagnostic Tests Pass and Instrument in Ready Mode
2.  Breath tube heated
3.  Date, time and location code (Level 2,E). Re-set if necessary.
4.  Print test (Level 1,P). Sign and attach test record.
5.  Tank monitor (Level 3,D,G). Display and Regulator within 50 psi.
  - a. Display 264 psi Regulator: 250 psi

B. Tests (Sign and attach test records):

Configure simulator for the following tests (Level 1,S).

1. Wet Calibration Check

- a.  Low AC (Level 1,C): Known Value  $\leq 0.03$  AC: 0.020 AC  
Sim. Ser #: MP3064 Lot #: 20070 Exp. Date: 2.13.22
- b.  High AC (Level 1,C): Known Value  $\geq 0.25$  AC: 0.300 AC  
Sim. Ser #: MP5317 Lot #: 202012A Exp. Date: 12.8.22

Configure dry gas standard for the following tests (Level 1,S).

2.  Interferent Check (Level 1,B): Known Value: 0.10 AC + 0.05% Acetone  
Sim. Ser #: DR3451 Lot #: FCS 6 Exp. Date: NA  
 Display reads "Interferent Detect"
3.  RFI Check (CMS Mode)  Display reads "RFI Detect"
4.  Dry Calibration Check (Level 1,C): Known Value 0.080 AC  
Gas Cyl Lot #: 24119080A1 Cyl #: 6 Exp. Date: 11.5.21

Test 1 <u>0.080</u>	Test 4 <u>0.081</u>	Test 7 <u>0.082</u>
Test 2 <u>0.081</u>	Test 5 <u>0.081</u>	Test 8 <u>0.081</u>
Test 3 <u>0.081</u>	Test 6 <u>0.081</u>	Test 9 <u>0.081</u>
Average <u>0.081</u>		

C. Remarks/Maintenance: ANNUAL INSPECTION, REPLACED SIMULATOR RETURN O-RING.

Instrument is acceptable to be used in the field.

[Signature]  
Breath Analyst Signature

4.13.2021  
Date

[Signature]  
Reviewed by

4-13-2021  
Date

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-004953  
Location = TOXL                8164.14.00 09/16  
04/13/2021                      14:27

\*\*\*\*\* Printer Test \*\*\*\*\*

abcdefghijklmnopqrstuvwxyz1234567890-|=|  
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&^\*()\_+?

abcdefghijklmnopqrstuvwxyz1234567890-|=|  
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&^\*()\_+?

Current Instrument Setup

Data Entry Mode:            Enabled  
Start Test Sequence:        DABACABA  
Display Prelim Rslt?        Yes  
Display Third Digit?        Yes  
Inhib Printer(Y/N)?        No  
Display Volume?            No  
Disable On Memfull?        Yes  
# of Print Copies?         1  
Select Std (D/W/I)?        Dry  
Standard Value?            0.080  
Standard Lot #?            24119080A1  
Standard Cyl #?            6  
Standard Expiration?       11/05/2021  
Oper No?                    666666

Flow Cal. Date:            04/20/2018  
Slope                      609  
Intercept                 -462839

IR Calibration Date:       04/20/2018

	3um	9um
0th Coef(*100):	-22916	-27195
1st Coef(*100):	261768	134580
2nd Coef(*100):	1749	1216
H2O adj(mg/l*10k):	496	540

\*\*\*\*\* Printer Test End \*\*\*\*\*



Operator Signature  
CHARLES EDER

Remarks:            PRINT TEST

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-004953  
Location = TOXL                    8164.14.00 09/16  
04/13/2021                            14:29

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:29
02 Std. Sol.	0.018	14:30
03 Room Air	0.000	14:31
04 Std. Sol.	0.018	14:31
05 Room Air	0.000	14:32
06 Std. Sol.	0.019	14:33
07 Room Air	0.000	14:33

08 Sim Temp = 34.0°C

Simul Ser No = MP3064  
Std Sol No = 20070  
County = 08                            Oper No. = 666666

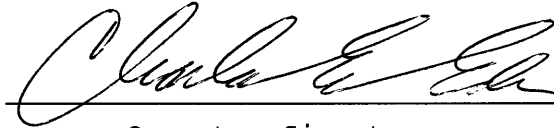

Operator Signature  
CHARLES EDER

Remarks: *Low AC*  
*0.020 AC*

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-004953  
Location = TOXL                 8164.14.00 09/16  
04/13/2021                        14:35

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:36
02 Std. Sol.	0.302	14:36
03 Room Air	0.000	14:37
04 Std. Sol.	0.306	14:38
05 Room Air	0.000	14:38
06 Std. Sol.	0.305	14:39
07 Room Air	0.000	14:39


08 Sim Temp = 34.0°C

Simul Ser No = MP5317

Std Sol No = 202012A

County = 08

Oper No. = 666666

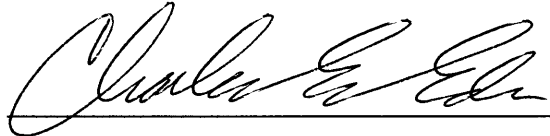

Operator Signature

CHARLES EDER

Remarks:

HIGH AC  
0.300 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer                      Alcohol Analyzer  
North Dakota Model 8000                      SN 80-004953  
Location = TOXL                              8164.14.00 09/16  
04/13/2021    14:41

Test	AC	Time
01 Room Air	0.000	14:42
02 *Subject Test	INT*	14:43
03 Room Air	0.000	14:44

\*Invalid Test  
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 02/01/1992  
Sub Sex = Female                              Weight = 150  
Test = DUI                                      Cit = INTERFERENT  
Dr. Lic. = ND/DIS921456  
Lot No = 24119080A1  
Cyl No = 6  
Expiration Date = 11/05/2021  
County = 08                                      Oper No. = 666666



Operator Signature  
CHARLES EDER

Remarks: *Interferent Check*  
*0.10 AC + 0.05% ACETONE*

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer                      Alcohol Analyzer  
North Dakota Model 8000                      SN 80-004953  
Location = TOXL                              8164.14.00 09/16  
04/13/2021    14:44

Test	AC	Time
01 Diagnostic	OK	14:46
02 Room Air	0.000	14:47
03 *Subject Test	RFI*	14:47
04 Room Air	0.000	14:47

\*Invalid Test  
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 02/01/1992  
Sub Sex = Female                              Weight = 150  
Test = DUI                                      Cit = RFI CHECK  
Dr. Lic. = ND/DIS921456  
Lot No = 24119080A1  
Cyl No = 6  
Expiration Date = 11/05/2021  
County = 08                                      Oper No. = 666666

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

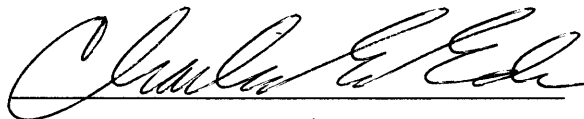

Operator Signature  
CHARLES EDER

Remarks:



Form 106-I8000


Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-004953  
Location = TOXL                 8164.14.00 09/16  
04/13/2021                         14:48

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:48
02 Std. Gas	0.080	14:48
03 Room Air	0.000	14:49
04 Std. Gas	0.081	14:49
05 Room Air	0.000	14:50
06 Std. Gas	0.081	14:50
07 Room Air	0.000	14:51

Lot No = 24119080A1  
Cyl No = 6  
Exp Date = 11/05/2021  
County = 08                            Oper No. = 666666

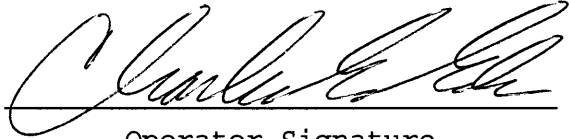

Operator Signature  
CHARLES EDER

Remarks: CALIBRATION CHECK  
0.080 AC

Form 106-I8000


Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer                      Alcohol Analyzer  
North Dakota Model 8000                      SN 80-004953  
Location = TOXL                              8164.14.00 09/16  
04/13/2021    14:51

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:51
02 Std. Gas	0.081	14:52
03 Room Air	0.000	14:52
04 Std. Gas	0.081	14:53
05 Room Air	0.000	14:53
06 Std. Gas	0.081	14:53
07 Room Air	0.000	14:54

Lot No = 24119080A1  
Cyl No = 6  
Exp Date = 11/05/2021  
County = 08                                      Oper No. = 666666

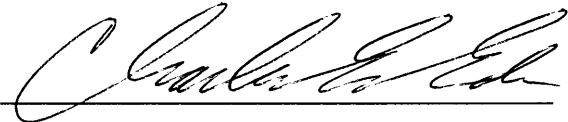

Operator Signature  
CHARLES EDER

Remarks: CALIBRATION CHECK  
0.080AC

Form 106-I8000




Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-004953  
Location = TOXL                 8164.14.00 09/16  
04/13/2021                        14:54

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:55
02 Std. Gas	0.082	14:55
03 Room Air	0.000	14:55
04 Std. Gas	0.081	14:56
05 Room Air	0.000	14:56
06 Std. Gas	0.081	14:57
07 Room Air	0.000	14:57

Lot No = 24119080A1  
Cyl No = 6  
Exp Date = 11/05/2021  
County = 08                            Oper No. = 666666

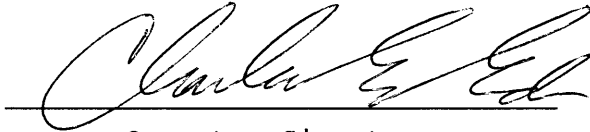

Operator Signature  
CHARLES EDER

Remarks: CALIBRATION CHECK  
0.080 AC

Form 106-I8000