Intoxilyzer® 8000 Inspection INTOXILYZER® 8000 INSPECTION

Intoxilyzer<sup>®</sup> 8000 Serial Number: <u>80-0049</u>39

Location: TOXL

- Α. General Setup and Checks:
  - Diagnostic Tests Pass and Instrument in Ready Mode 1.
  - 2. Breath tube heated
  - Date, time and location code (Level 2,E). Re-set if necessary. 3.
  - 4. Reprint test (Level 1,P). Sign and attach test record.
  - Tank monitor (Level 3,D,G). Display and Regulator within 50 psi. 5.
    - Display: 142 psi Regulator: 150 psi
- Β. Tests (Sign and attach test records):
  - Configure simulator for the following tests (Level 1,S).
    - Wet Calibration Check
      - X Low AC (Level 1,C); Known Value  $\leq 0.03$  AC: O.020 AC а. Sim. Ser #: MP3064 Lot #: 200 70 Exp. Date: 2.13.22
      - A High AC (Level 1,C): Known Value ≥ 0.25 AC: 0.300 AC b. Sim. Ser #: MP3057 Lot #: 20030 Exp. Date: 1.21.22

 $\overset{\frown}{X}$  Configure dry gas standard for the following tests (Level 1,S).

- X Interferent Check (Level 1,B): Known Value: 0.10 AC + 0.05% Acetone 2. Sim. Ser #: <u>DR3847</u> Lot #: <u>ICS 6</u> Exp. Date: <u>DA</u> Display reads "Interferent Detect"
- RFI Check (CMS Mode) 3.
- X Dry Calibration Check (Level 1,C): Known Value \_ <u>0.080</u> AC 4. Gas Cyl Lot #:24119080 A1 Cyl #: 6 Exp. Date: 11.5:21
- Remarks/Maintenance: ANNUAL INSPECTION C.

Average 0.082

X Instrument is acceptable to be used in the field.

Breath Analyst Signature

Réviewed by

 $\frac{4.14.21}{Date}$ 

02/18 Issuing Authority - CEE

Page 1 of 1

Intoxilyzer Test Record and Checklist NDOAG Crime Lab. Div., Bismarck, ND 58501 Alcohol Analyzer CMI, Inc. Intoxilyzer North Dakota Model 8000 SN 80-004939 8164.14.00 09/16 Location = TOXL 04/14/2021 16:08 abcdefghijklmnopgrstuvwxyz1234567890-= ABCDEFGHIJKLMNOPQRSTUVWXYZ!@#\$%^&\*() +? abcdefghijklmnopqrstuvwxyz1234567890-= ABCDEFGHIJKLMNOPQRSTUVWXYZ!@#\$%^&\*() +? Current Instrument Setup Data Entry Mode: Enabled Start Test Sequence: DABACABA Display Prelim Rslt? Yes Display Third Digit? Yes Inhib Printer(Y/N)? No Display Volume? No Disable On Memfull? Yes # of Print Copies? 1 Select Std (D/W/I)? Dry Standard Value? 0.080 Standard Lot #? 24119080A1 Standard Cyl #? 6 Standard Expiration? 11/05/2021 Oper No? 666666 Flow Cal. Date: 01/18/2010 Slope 724 -779308 Intercept IR Calibration Date: 04/05/2017 9um 3um 0th Coef(\*100): -24406 -20805 lst Coef(\*100): 261875 2nd Coef(\*100): 2828 135290 1427 H2O adj(mg/l\*10k): 563393 

Operator Signature CHARLES EDER

Form 106-18000

Intoxilyzer Test Record and Checklist NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. IntoxilyzerAlcohol AnalyzerNorth Dakota Model 8000SN 80-004939Location = TOXL8164.14.00 09/1604/14/202116:09

			WET CAL CHECK	
Te	est		AC	Time
01	Room	Air	0.000	16:10
02	Std.	Sol.	0.020	16:10
03	Room	Air	0.000	16:11
04	Std.	Sol.	0.020	16:12
05	Room	Air	0.000	16:12
06	Std.	Sol.	0.020	16:13
07	Room	Air	0.000	16:14

08 Sim Temp = 34.0°C

Simul Ser No = MP3064 Std Sol No = 20070 County = 08

Oper No. = 666666

M 2 1 1 2

Operator Signature CHARLES EDER

Low AC 0.020, C

Form 106-18000

Intoxilyzer Test Record and Checklist NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. IntoxilyzerAlcohol AnalyzerNorth Dakota Model 8000SN 80-004939Location = TOXL8164.14.00 09/1604/14/202116:15

	WET CAL CHECK	
Test	AC	Time
01 Room Air	0.000	16:15
02 Std. Sol.	0.296	16:16
03 Room Air	0.000	16:17
04 Std. Sol.	0.296	16:17
05 Room Air	0.000	16:18
06 Std. Sol.	0.296	16:19
07 Room Air	0.000	16:19

08 Sim Temp = 34.0°C

Simul Ser No = MP3057 Std Sol No = 20030 County = 08

Oper No. = 666666

ÚU

Operator Signature CHARLES EDER

+IGH

Remarks:

Form 106-18000

0.300 K

Intoxilyzer Test Record and Checklist NDOAG Crime Lab. Div., Bismarck, ND 58501 CMI, Inc. Intoxilyzer Alcohol Analyzer North Dakota Model 8000 SN 80-004939 8164.14.00 09/16 Location = TOXL 04/14/2021 16:22 Test AC Time 01 Room Air 0.000 16:23 16:23 02 \*Subject Test INT\* 03 Room Air 0.000 16:24 \*Invalid Test Interferent Detected Sub Name = DISCOVER, THE SPIRIT Sub DOB = 02/01/1992Sub Sex = Female Weight = 150Test = DUI Cit = INTERFERENT Dr. Lic. = ND/DIS921456Lot No = 24119080A1Cyl No = 6Expiration Date = 11/05/2021Oper No. = 666666County = 08

12

Operator Signature CHARLES, EDER

Check ACETONE Ð1 0,10

Form 106-I8000

Intoxilyzer Test Record and Checklist NDOAG Crime Lab. Div., Bismarck, ND 58501 Alcohol Analyzer CMI, Inc. Intoxilyzer North Dakota Model 8000 SN 80-004939 Location = TOXL 8164.14.00 09/16 04/14/2021 16:24 AC Time Test 01 Diagnostic OK 16:24 16:25 02 Room Air 0.000 16:25 03 \*Subject Test RFI\* 04 Room Air 0.000 16:26 \*Invalid Test Inhibited - RFI Sub Name = DISCOVER, THE SPIRIT Sub DOB = 02/01/1992Sub Sex = Female Weight = 150Cit = RFI CHECK Test = DUI Dr. Lic. = ND/DIS921456Lot No = 24119080A1Cyl No = 6Expiration Date = 11/05/2021County = 08Oper No. = 666666

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

**Operator Signature** CHARLES EDER

Form 106-18000

Intoxilyzer Test Record and Checklist NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer North Dakota Model 8000 SN 80-004939 Location = TOXL 8164.14.00 09/16 04/14/2021 16:26

			DRY CAL CHECK	
Te	est		AC	Time
	Room		0.000	16:26
02	Std.	Gas	0.081	16:27
03	Room	Air	0.000	16:27
04	Std.	Gas	0.081	16:28
05	Room	Air	0.000	16:28
06	Std.	Gas	0.081	16:28
07	Room	Air	0.000	16:29

Lot No = 24119080A1Cyl No = 6Exp Date = 11/05/2021County = 08

Oper No. = 666666

CA 1

Operator Signature CHARLES EDER CHECK

LIBRATION

Remarks:

Form 106-18000

0.080

Intoxilyzer Test Record and Checklist NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. IntoxilyzerAlcohol AnalyzerNorth Dakota Model 8000SN 80-004939Location = TOXL8164.14.00 09/1604/14/202116:29

	DRY CAL CHECK	
Test	AC	Time
01 Room Air 02 Std. Gas 03 Room Air 04 Std. Gas 05 Room Air 06 Std. Gas 07 Room Air	0.000 0.082 0.000 0.081 0.000 0.082 0.000	16:30 16:30 16:31 16:31 16:32 16:32

Lot No = 24119080A1 Cyl No = 6 Exp Date = 11/05/2021 County = 08

Oper No. = 666666

1 U

Operator Signature CHARLES EDER

BRATTON CHECK 0.080AC

Form 106-18000

Intoxilyzer Test Record and Checklist NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. IntoxilyzerAlcohol AnalyzerNorth Dakota Model 8000SN 80-004939Location = TOXL8164.14.00 09/1604/14/202116:32

	DRY CAL CHECK	
Test	AC	Time
01 Room Air 02 Std. Gas 03 Room Air 04 Std. Gas 05 Room Air 06 Std. Gas 07 Room Air	0.000 0.082 0.000 0.081 0.000 0.083 0.000	16:33 16:33 16:34 16:34 16:35 16:35 16:35

Lot No = 24119080A1 Cyl No = 6 Exp Date = 11/05/2021 County = 08

Oper No. = 666666

Operator Signature CHARLES EDER

ALIBRATTON CHECK 0.080 AC

Form 106-18000