OFFICE OF ATTORNEY GENERAL CRIME LABORATORY DIVISION

Toxicology Section/Breath Alcohol Program Intoxilyzer® 8000 Inspection

INTOXILYZER® 8000 INSPECTION

Into	xilyzer® 8000 Serial Number: <u>80-003</u> 059 Loca	ition: $\overline{10}\times$
A.	General Setup and Checks: 1. Diagnostic Tests Pass and Instrument in Ready Model 2. Breath tube heated 3. Date, time and location code (Level 2,E). Re-set 4. Print test (Level 1,P). Sign and attach test record 5. Tank monitor (Level 3,D,G). Display and Regulation and Display: psi Regulator: psi	if necessary.
В.	Tests (Sign and attach test records): Configure simulator for the following tests (Level 1,S). Wet Calibration Check a.	Date: <u>2.13.22</u> AC: <u>6.300</u> AC Date: <u>12.8.22</u> I 1,S). AC + 0.05% Acetone DAC + MA
C.	4. Dry Calibration Check (Level 1,C): Known Value Gas Cyl Lot #: 24/190804 Cyl #: 6 Exp Test 1 0.081 Test 4 0.081 Test Test 2 0.081 Test 5 0.081 Test Test 3 0.081 Test 6 0.081 Test Average 0.081	. Date: <u>//· 5· 2 </u> 7 <u> </u>
	astrument is acceptable to be used in the field. ath Analyst Signature iewed by	4.15-21 Date 4-19-31 Date

02/18

Issuing Authority - CEE

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003059
Location = TOXL 8164.14.00 09/16
04/15/2021 15:32

******** Printer Test ********

abcdefghijklmnopqrstuvwxyz1234567890-=|
ABCDEFGHIJKLMNOPQRSTUVWXYZ!@#\$%^&*() +?

abcdefghijklmnopqrstuvwxyz1234567890-=| ABCDEFGHIJKLMNOPQRSTUVWXYZ!@#\$%^&*() +?

Current Instrument Setup

Data Entry Mode: Enabled Start Test Sequence: DABACABA Display Prelim Rslt? Yes Display Third Digit? Yes Inhib Printer(Y/N)? No Display Volume? No Disable On Memfull? Yes # of Print Copies? 1 Select Std (D/W/I)? Dry Standard Value? 0.080 Standard Lot #? 24119080A1 Standard Cyl #? 11/05/2021 Standard Expiration? Oper No? 666666

Flow Cal. Date: 11/28/2018 Slope 589

Intercept -276335

IR Calibration Date: 12/04/2018
3um 9um

Oth Coef(*100): -25330 -15825 1st Coef(*100): 278286 137135 2nd Coef(*100): 1992 1257 H2O adj(mg/l*10k): 525 321

****** Printer Test End *******

Operator Signature CHARLES EDER

Remarks:

CMI, Inc. Intoxilyzer North Dakota Model 8000 Location = TOXL 04/15/2021 SN 80-003059 8164.14.00 09/16 15:34

WET CAL CHECK

Test			AC	Time
01	Room	Air	0.000	15:35
02	Std.	Sol.	0.018	15:35
03	Room	Air	0.000	15:36
04	Std.	Sol.	0.018	15:37
05	Room	Air	0.000	15:37
06	Std.	Sol.	0.018	15:38
07	Room	Air	0.000	15:38

 $08 \text{ Sim Temp} = 34.0^{\circ}\text{C}$

Simul Ser No = MP3064 Std Sol No = 20070

County = 08

Oper No. = 666666

Operator Signature CHARLES EDER

Remarks:

CMI, Inc. Intoxilyzer North Dakota Model 8000 Location = TOXL 04/15/2021 SN 80-003059 8164.14.00 09/16 15:40

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	15:40
02 Std. Sol.	0.298	15:41
03 Room Air	0.000	15:42
04 Std. Sol.	0.301	15:42
05 Room Air	0.000	15:43
06 Std. Sol.	0.300	15:44
07 Room Air	0.000	15:44

 $08 \text{ Sim Temp} = 34.0^{\circ}\text{C}$

Simul Ser No = MP5317 Std Sol No = 202012A

County = 08

Oper No. = 666666

Operator Signature CHARLES EDER

Remarks:

0, 300 AC Form 106-18000

CMI, Inc. Intoxilyzer
North Dakota Model 8000
Location = TOXL
04/15/2021

SN 80-003059 8164.14.00 09/16 15:46

Test	AC	Time
01 Room Air 02 *Subject Test	0.000 TNT*	15:47 15:48
03 Room Air	0.000	15:48

*Invalid Test Interferent Detected

Sub Name = DISCOVER, THE SPIRIT

Sub DOB = 02/01/1992

Sub Sex = Female

Weight = 150

Test = DUI

Cit = INTERFERENT

Dr. Lic. = ND/DIS921456

Lot No = 24119080A1

Cyl No = 6

Expiration Date = 11/05/2021

County = 08

Oper No. = 666666

rent Check +0.05% ACETONE

Operator Signature CHARLES EDER

_ /

Remarks:

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003059
Location = TOXL 8164.14.00 09/16
04/15/2021 15:49

Test	AC	Time
01 Diagnostic	OK	15:49
02 Room Air	0.000	15:50
03 *Subject Test	RFI*	15:50
04 Room Air	0.000	15:51

*Invalid Test Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT

Sub DOB = 02/01/1992

Sub Sex = Female Weight = 150
Test = DUI Cit = RFI CHECK

Dr. Lic. = ND/DIS921456

Lot No = 24119080A1

Cyl No = 6

Expiration Date = 11/05/2021

County = 08 Oper No. = 666666

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

Operator Signature

CHARLES EDER

Remarks:

CMI, Inc. Intoxilyzer North Dakota Model 8000 Location = TOXL 04/15/2021

Alcohol Analyzer SN 80-003059 8164.14.00 09/16 15:51

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	15:52
02 Std. Gas	0.081	15:52
03 Room Air	0.000	15:53
04 Std. Gas	0.081	15:53
05 Room Air	0.000	15:54
06 Std. Gas	0.081	15:54
07 Room Air	0.000	15:55

Lot No = 24119080A1

Cyl No = 6

Remarks:

Exp Date = 11/05/2021

County = 08

Oper No. = 666666

Operator Signature

CHARLES EDER

'ALIBRATION CHECK O. LOAC +O. OS! ACETONE

CMI, Inc. Intoxilyzer
North Dakota Model 8000
Location = TOXL
04/15/2021

SN 80-003059 8164.14.00 09/16 15:55

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	15:55
02 Std. Gas	0.081	15:56
03 Room Air	0.000	15:56
04 Std. Gas	0.081	15:57
05 Room Air	0.000	15:57
06 Std. Gas	0.081	15:57
07 Room Air	0.000	15:58

Lot No = 24119080A1

Cyl No = 6

Exp Date = 11/05/2021

County = 08

Oper No. = 666666

Operator Signature

CHARLES EDER

A BRA

CMI, Inc. Intoxilyzer
North Dakota Model 8000
Location = TOXL
04/15/2021

SN 80-003059 8164.14.00 09/16 15:59

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	15:59
02 Std. Gas	0.081	16:00
03 Room Air	0.000	16:00
04 Std. Gas	0.081	16:00
05 Room Air	0.000	16:01
06 Std. Gas	0.081	16:01
07 Room Air	0.000	16:02

Lot No = 24119080A1

Cyl No = 6

Exp Date = 11/05/2021

County = 08

Oper No. = 666666

Operator Signature CHARLES EDER

Remarks:

Form 106-I8000

0.088 AC