

INTOXILYZER® 8000 INSPECTION

Intoxilyzer® 8000 Serial Number: 80-005938

Location: TOXL

A. General Setup and Checks:

1. Diagnostic Tests Pass and Instrument in Ready Mode
2. Breath tube heated
3. Date, time and location code (Level 2,E). Re-set if necessary.
4. Print test (Level 1,P). Sign and attach test record.
5. Tank monitor (Level 3,D,G). Display and Regulator within 50 psi.
 - a. Display 247 psi Regulator: 250 psi

B. Tests (Sign and attach test records):

Configure simulator for the following tests (Level 1,S).

1. Wet Calibration Check

- a. Low AC (Level 1,C): Known Value ≤ 0.03 AC: 0.020 AC
Sim. Ser #: MP3064 Lot #: 20070 Exp. Date: 2.13.22
- b. High AC (Level 1,C): Known Value ≥ 0.25 AC: 0.300 AC
Sim. Ser #: MPS317 Lot #: 202012A Exp. Date: 12.8.22

Configure dry gas standard for the following tests (Level 1,S).

2. Interferent Check (Level 1,B): Known Value: 0.10 AC + 0.05% Acetone
Sim. Ser #: DR5114 Lot #: IC56 Exp. Date: NA
 Display reads "Interferent Detect"
3. RFI Check (CMS Mode) Display reads "RFI Detect"
4. Dry Calibration Check (Level 1,C): Known Value 0.080 AC
Gas Cyl Lot # 24119080A1 Cyl #: 5 Exp. Date: 11.5.21

Test 1 <u>0.081</u>	Test 4 <u>0.081</u>	Test 7 <u>0.081</u>
Test 2 <u>0.081</u>	Test 5 <u>0.081</u>	Test 8 <u>0.080</u>
Test 3 <u>0.081</u>	Test 6 <u>0.081</u>	Test 9 <u>0.080</u>
Average <u>0.081</u>		

C. Remarks/Maintenance: ANNUAL INSPECTION, REPLACED SIMULATOR RETURN O-RINGS.

Instrument is acceptable to be used in the field.

Charles E. Ed
Breath Analyst Signature

4.19.21
Date

Kevin L. Smith
Reviewed by

4-19-21
Date

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005938
Location = TOXL 8164.14.00 09/16
04/19/2021 16:36

***** Printer Test *****

abcdefghijklmnopqrstuvwxy1234567890-|=|
ABCDEFGHIJKLMN0PQRSTUVWXYZ!@#%&^&*()_+?

abcdefghijklmnopqrstuvwxy1234567890-|=|
ABCDEFGHIJKLMN0PQRSTUVWXYZ!@#%&^&*()_+?

Current Instrument Setup


Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 24119080A1
Standard Cyl #? 5
Standard Expiration? 11/05/2021
Oper No? 666666

Flow Cal. Date: 07/15/2013
Slope 664
Intercept -578364

IR Calibration Date: 07/27/2015

	3um	9um
0th Coef(*100):	-14232	-19463
1st Coef(*100):	289698	139293
2nd Coef(*100):	2380	1071
H2O adj(mg/l*10k):	253	294

***** Printer Test End *****

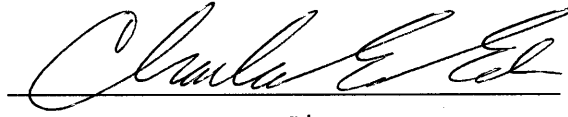

Operator Signature
CHARLES EDER

Remarks: PRINT TEST

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005938
Location = TOXL 8164.14.00 09/16
04/19/2021 16:37

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	16:38
02 Std. Sol.	0.019	16:39
03 Room Air	0.000	16:39
04 Std. Sol.	0.020	16:40
05 Room Air	0.000	16:40
06 Std. Sol.	0.019	16:41
07 Room Air	0.000	16:42

08 Sim Temp = 34.0°C

Simul Ser No = MP3064
Std Sol No = 20070
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: *Low AC*
0.020 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005938
Location = TOXL 8164.14.00 09/16
04/19/2021 16:50

Test	AC	Time
01 Diagnostic	OK	16:51
02 Room Air	0.000	16:51
03 *Subject Test	RFI*	16:51
04 Room Air	0.000	16:52

*Invalid Test
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 02/01/1992
Sub Sex = Female Weight = 150
Test = DUI Cit = RFI CHECK
Dr. Lic. = ND/DIS921456
Lot No = 24119080A1
Cyl No = 5
Expiration Date = 11/05/2021
County = 08 Oper No. = 666666

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

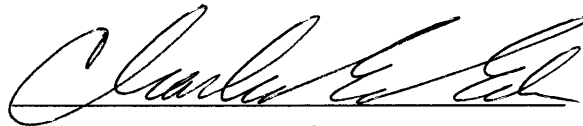

Operator Signature

CHARLES EDER

Remarks:

RFI CHECK

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005938
Location = TOXL 8164.14.00 09/16
04/19/2021 16:52

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	16:52
02 Std. Gas	0.081	16:53
03 Room Air	0.000	16:53
04 Std. Gas	0.081	16:54
05 Room Air	0.000	16:54
06 Std. Gas	0.081	16:55
07 Room Air	0.000	16:55

Lot No = 24119080A1
Cyl No = 5
Exp Date = 11/05/2021
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks:

CALIBRATION CHECK
0.080 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005938
Location = TOXL 8164.14.00 09/16
04/19/2021 16:59

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	16:59
02 Std. Gas	0.081	17:00
03 Room Air	0.000	17:00
04 Std. Gas	0.080	17:01
05 Room Air	0.000	17:01
06 Std. Gas	0.080	17:01
07 Room Air	0.000	17:02

Lot No = 24119080A1
Cyl No = 5
Exp Date = 11/05/2021
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks:

CALIBRATION CHECK
0.080 AC

Form 106-I8000