Toxicology Section/Breath Alcohol Program Intoxilyzer® 8000 Inspection

INTOXILYZER® 8000 INSPECTION

Intoxilyzer® 8000 Serial Number: 80-004942 Location: 70 XL
 A. General Setup and Checks: 1. Diagnostic Tests Pass and Instrument in Ready Mode 2. Breath tube heated 3. Date, time and location code (Level 2,E). Re-set if necessary. 4. Print test (Level 1,P). Sign and attach test record. 5. Zerank monitor (Level 3,D,G). Display and Regulator within 50 psi. a. Display: 959 psi Regulator: 950 psi
 B. Tests (Sign and attach test records): Configure simulator for the following tests (Level 1,S). 1. Wet Calibration Check a. Low AC (Level 1,C): Known Value ≤ 0.03 AC: O·O·30 AC Sim. Ser #: M1/5289 Lot #: 2019 // E Exp. Date: //* 19·2/ b. High AC (Level 1,C): Known Value ≥ 0.25 AC: O·400 AC Sim. Ser #: M2/3062 Lot #: 2021 03 E Exp. Date: 3·24.25
Configure dry gas standard for the following tests (Level 1,S).
2. MInterferent Check (Level 1,B): Known Value: O-IOAC + 0.05% Acetone Sim. Ser #: DR3847 Lot #: FCS Exp. Date: NA-X Display reads "Interferent Detect" 3. MRFI Check (CMS Mode)
C. Remarks/Maintenance: ANNUAL INSPECTION, REPLACED
SIMMLATOR RETURN O-RING
Instrument is acceptable to be used in the field.
(lache tetal 4.26-2)
Breath Analyst Signature Date
Lau L. 4127 4-27-31
Reviewed by Date

02/18

Issuing Authority - CEE

Page 1 of 1

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004942
Location = TOXL 8164.14.00 09/16
04/26/2021 15:19

******* Printer Test ********

abcdefghijklmnopqrstuvwxyz1234567890-=|
ABCDEFGHIJKLMNOPQRSTUVWXYZ!@#\$%^&*() +?

abcdefghijklmnopqrstuvwxyz1234567890-=|
ABCDEFGHIJKLMNOPQRSTUVWXYZ!@#\$%^&*() +?

Current Instrument Setup

Data Entry Mode: Enabled Start Test Sequence: DABACABA Display Prelim Rslt? Yes Display Third Digit? Yes Inhib Printer(Y/N)? No Display Volume? No Disable On Memfull? Yes # of Print Copies? 1 Select Std (D/W/I)? Dry Standard Value? 0.080 Standard Lot #? 05620080A1 Standard Cyl #? 31 Standard Expiration? 04/05/2022 Oper No? 666666

Flow Cal. Date: 05/25/2018 Slope 626 Intercept -548004

IR Calibration Date: 05/25/2018
3um 9um

 0th Coef(*100):
 -10942
 -13713

 1st Coef(*100):
 255654
 132286

 2nd Coef(*100):
 3705
 1469

 H2O adj(mg/l*10k):
 481
 462

****** Printer Test End *******

Operator Signature CHARLES EDER

Remarks: POINT TE

Form 106-I8000

CMI, Inc. Intoxilyzer
North Dakota Model 8000
Location = TOXL
04/26/2021

Alcohol Analyzer SN 80-004942 8164.14.00 09/16 15:22

WET CAL CHECK

Т	est		AC	Time
01	Room	Air	0.000	15:23
02	Std.	Sol.	0.030	15:24
03	Room	Air	0.000	15:24
04	Std.	Sol.	0.031	15:25
05	Room	Air	0.000	15:25
06	Std.	Sol.	0.032	15:26
07	Room	Air	0.000	15:27

 $08 \text{ Sim Temp} = 34.0^{\circ}\text{C}$

Simul Ser No = MP5289 Std Sol No = 201911E

County = 08

Oper No. = 666666

Operator Signature CHARLES EDER

Low A

Remarks:

Form 106-I8000

0.038 AC

CMI, Inc. Intoxilyzer North Dakota Model 8000 Location = TOXL 04/26/2021

Alcohol Analyzer SN 80-004942 8164.14.00 09/16 15:28

WET CAL CHECK

Te	est		AC	Time
01	Room	Air	0.000	15:28
02	Std.	Sol.	0.406	15:29
03	Room	Air	0.000	15:30
04	Std.	Sol.	0.407	15:30
05	Room	Air	0.000	15:31
06	Std.	Sol.	0.408	15:32
07	Room	Air	0.000	15:32

08 Sim Temp = 34.0°C

Simul Ser No = MP3062 Std Sol No = 202103E

County = 08

Oper No. = 666666

Operator Signature CHARLES EDER

Remarks:

Form 106-I8000

0,400 AC

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004942
Location = TOXL 8164.14.00 09/16
04/26/2021 15:34

Test	AC	Time
01 Room Air	0.000	15:35
02 *Subject Test	INT*	15:35
03 Room Air	0.000	15:36

*Invalid Test

Interferent Detected

Sub Name = DISCOVER, THE SPIRIT

Sub DOB = 02/01/1992

Sub Sex = Female Weight = 150
Test = DUI Cit = INTERFERENT

Dr. Lic. = ND/DIS921456

Lot No = 02621080A1

Cyl No = 25

Expiration Date = 03/05/2023

County = 08 Oper No. = 666666

Operator Signature

CHARLES EDER

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004942
Location = TOXL 8164.14.00 09/16
04/26/2021 15:36

Test	AC	Time
01 Diagnostic	OK	15:37
02 Room Air	0.000	15:37
03 *Subject Test	RFI*	15:37
04 Room Air	0.000	15:38

*Invalid Test Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT

Sub DOB = 02/01/1992

Sub Sex = Female Weight = 150
Test = DUI Cit = RFI CHECK

Dr. Lic. = ND/DIS921456

Lot No = 02621080A1

Cyl No = 25

Remarks:

Expiration Date = 03/05/2023

County = 08 Oper No. = 666666

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

Operator Signature

CHARLES EDER

Form 106-I8000

CMI, Inc. Intoxilyzer North Dakota Model 8000 Location = TOXL 04/26/2021 Alcohol Analyzer SN 80-004942 8164.14.00 09/16 15:38

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	15:39
02 Std. Gas	0.083	15:39
03 Room Air	0.000	15:40
04 Std. Gas	0.082	15:40
05 Room Air	0.000	15:41
06 Std. Gas	0.082	15:41
07 Room Air	0.000	15:41

Lot No = 02621080A1

Cyl No = 25

Exp Date = 03/05/2023

County = 08

Oper No. = 666666

Operator Signature CHARLES EDER

Remarks:

Form 106-I8000

0.080AC

CMI, Inc. Intoxilyzer
North Dakota Model 8000
Location = TOXL
04/26/2021

Alcohol Analyzer SN 80-004942 8164.14.00 09/16 15:42

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	15:42
02 Std. Gas	0.082	15:43
03 Room Air	0.000	15:43
04 Std. Gas	0.082	15:43
05 Room Air	0.000	15:44
06 Std. Gas	0.082	15:44
07 Room Air	0.000	15:45

Lot No = 02621080A1

Cyl No = 25

Exp Date = 03/05/2023

County = 08

Oper No. = 666666

Operator Signature CHARLES EDER

Remarks: CALIBRATION CHECK

Form 106-I8000

0.080 AC

CMI, Inc. Intoxilyzer
North Dakota Model 8000
Location = TOXL
04/26/2021

Alcohol Analyzer SN 80-004942 8164.14.00 09/16 15:45

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	15:46
02 Std. Gas	0.082	15:46
03 Room Air	0.000	15:46
04 Std. Gas	0.083	15:47
05 Room Air	0.000	15:47
06 Std. Gas	0.082	15:48
07 Room Air	0.000	15:48

Lot No = 02621080A1

Cyl No = 25

Exp Date = 03/05/2023

County = 08

Oper No. = 666666

Operator Signature CHARLES EDER

Form 106-I8000