Intoxilyzer® 8000 Inspection

## **INTOXILYZER® 8000 INSPECTION**

Intoxilyzer® 8000 Serial Number: 80-005957

Location: TOXL

- Α. General Setup and Checks:
  - Diagnostic Tests Pass and Instrument in Ready Mode 1.
  - 2. Breath tube heated
  - 3. Date, time and location code (Level 2,E). Re-set if necessary.
  - Print test (Level 1,P). Sign and attach test record. 4.
  - Tank monitor (Level 3,D,G). Display and Regulator within 50 psi. 5.
    - Display: 223 psi Regulator: 250 psi
- Tests (Sign and attach test records): Β.

Configure simulator for the following tests (Level 1,S).

- Wet Calibration Check 1.
  - ▲ Low AC (Level 1,C): Known Value  $\leq 0.03$  AC:  $O_{*}020$  AC а. Sim. Ser #: <u>MP3064</u> Lot #: 20070 Exp. Date:  $2 \cdot 13 \cdot 22$ [X High AC (Level 1,C): Known Value  $\ge 0.25$  AC:  $0 \cdot 300$  AC
  - b. Sim. Ser #: MP5317 Lot #: 202012 4 Exp. Date: 12.8.22

Configure dry gas standard for the following tests (Level 1,S).

- K Interferent Check (Level 1,B): Known Value:  $\frac{\partial_{r}}{\partial}AC + 0.05\%$  Acetone 2. Sim. Ser #: DR3847 Lot #: ICS 6 Exp. Date: NA Display reads "Interferent Detect"
- 🗙 RFI Check (CMS Mode) 🛛 🕺 Display reads "RFI Detect" 3.
- X Dry Calibration Check (Level 1,C): Known Value <u>0.080</u> AC 4. Gas Cyl Lot #: 02621080 A1 Cyl #: 25 Exp. Date: 3.5.23 Test 10.079Test 40.079Test 70.079Test 20.079Test 50.079Test 80.079Test 30.079Test 60.080Test 90.079Average 0.079
- Remarks/Maintenance: ANNUAL INSPECTION, TANK SENSOR CAL, C.

Aunstrument is acceptable to be used in the field.

Breath Analyst Signature

Reviewed by

02/18 Issuing Authority - CEE

 $\frac{5.4.2021}{\text{Date}}$ 

Page 1 of 1

Intoxilyzer Test Record and Checklist NDOAG Crime Lab. Div., Bismarck, ND 58501 CMI, Inc. Intoxilyzer Alcohol Analyzer North Dakota Model 8000 SN 80-005957 Location = TOXL8164.14.00 09/16 05/04/2021 09:20 abcdefghijklmnopgrstuvwxyz1234567890-=| ABCDEFGHIJKLMNOPQRSTUVWXYZ!@#\$%^&\*() +? abcdefghijklmnopgrstuvwxyz1234567890-= ABCDEFGHIJKLMNOPQRSTUVWXYZ!@#\$%^&\*() +? Current Instrument Setup Data Entry Mode: Enabled Start Test Sequence: DABACABA Display Prelim Rslt? Yes Display Third Digit? Yes Inhib Printer(Y/N)? No Display Volume? No Yes Disable On Memfull? # of Print Copies? 1 Select Std (D/W/I)? Dry Standard Value? 0.080 Standard Lot #? 02621080A1 Standard Cyl #? 25 03/05/2023 Standard Expiration? Oper No? 666666 Flow Cal. Date: 06/21/2013 Slope 647 Intercept -613317 IR Calibration Date: 05/11/2015 3um 9um \_ \_ \_ \_ \_ \_ \_ \_ \_ \_ \_ \_ \_ \_ \_ \_ 0th Coef(\*100): -16666 -18045 1st Coef(\*100): 278478 2nd Coef(\*100): 3303 132949 1397 H2O adj(mg/l\*10k): 379351

Operator Signature CHARLES EDER

RINT IE

Form 106-I8000

Intoxilyzer Test Record and Checklist NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. IntoxilyzerAlcohol AnalyzerNorth Dakota Model 8000SN 80-005957Location = TOXL8164.14.00 09/1605/04/202109:26

		WET CAL CHECK	
Т	est	AC	Time
01	Room Air	0.000	09:26
02	Std. Sol.	0.019	09:27
03	Room Air	0.000	09:28
04	Std. Sol.	0.018	09:28
05	Room Air	0.000	09:29
06	Std. Sol.	0.019	09:30
07	Room Air	0.000	09:30
00	Cim Tomp	- 24 090	

08 Sim Temp = 34.0°C

Simul Ser No = MP3064 Std Sol No = 20070 County = 08

Oper No. = 666666

Operator Signature CHARLES EDER

Low AC 0.020 AC

Form 106-18000

Intoxilyzer Test Record and Checklist NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. IntoxilyzerAlcohol AnalyzerNorth Dakota Model 8000SN 80-005957Location = TOXL8164.14.00 09/1605/04/202109:31

	WET CAL CHECK				
Test	AC	Time			
01 Room Air	0.000	09:32			
02 Std. Sol.	0.296	09:32			
03 Room Air	0.000	09:33			
04 Std. Sol.	0.297	09:34			
05 Room Air	0.000	09:34			
06 Std. Sol.	0.297	09:35			
07 Room Air	0.000	09:35			
08 Sim Temp = $34.0$ °C Simul Ser No = MP5317					
Std Sol No = $2$					
County = 08		No. = 666666			
Charles Ech					

Operator Signature CHARLES EDER

HIGH AC 0. 300 AC

Form 106-18000

Intoxilyzer Test Record and Checklist NDOAG Crime Lab. Div., Bismarck, ND 58501 CMI, Inc. Intoxilyzer Alcohol Analyzer North Dakota Model 8000 SN 80-005957 Location = TOXL 8164.14.00 09/16 05/04/2021 09:37 Test AC Time 09:38 01 Room Air 0.000 09:39 02 \*Subject Test INT\* 03 Room Air 0.000 09:40 \*Invalid Test Interferent Detected Sub Name = TEST, DONOR2 NONE Sub DOB = 07/25/1998Sub Sex = Male Weight = NA Test = DUI Cit = INTERFERENT Dr. Lic. = ND/TES989643Lot No = 02621080A1Cyl No = 25Expiration Date = 03/05/2023County = 08Oper No. = 666666**Operator Signature** CHARLES EDER

- INTERFECTANT CHECK 0.10AC + 0.05% ACETONE Remarks: CE4

Form 106-18000

Intoxilyzer Test Record and Checklist NDOAG Crime Lab. Div., Bismarck, ND 58501 CMI, Inc. Intoxilyzer Alcohol Analyzer North Dakota Model 8000 SN 80-005957 Location = TOXL 8164.14.00 09/16 05/04/2021 09:40 AC Time Test 09:41 01 Diagnostic OK 09:41 02 Room Air 0.000 03 \*Subject Test RFI\* 09:41 09:42 04 Room Air 0.000 \*Invalid Test Inhibited - RFI Sub Name = TEST, DONOR2 NONE Sub DOB = 07/25/1998Sub Sex = Male Weight = NA Test = DUI Cit = RFI CHECK Dr. Lic. = ND/TES989643Lot No = 02621080A1Cyl No = 25Expiration Date = 03/05/2023County = 08Oper No. = 666666

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

Operator Signature CHARLES EDER

CK

Form 106-18000

Intoxilyzer Test Record and Checklist NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer North Dakota Model 8000 SN 80-005957 Location = TOXL 8164.14.00 09/16 05/04/2021 09:42

	DRY CAL CHECK	
Test	AC	Time
01 Room Air 02 Std. Gas 03 Room Air 04 Std. Gas 05 Room Air 06 Std. Gas 07 Room Air	0.000 0.079 0.000 0.079 0.000 0.079 0.000	09:43 09:43 09:43 09:44 09:44 09:45 09:45

Lot No = 02621080A1Cyl No = 25Exp Date = 03/05/2023County = 08

Oper No. = 666666

Operator Signature CHARLES EDER YON CHECK

ALIBRAT

0.080AC

Form 106-18000

Intoxilyzer Test Record and Checklist NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer North Dakota Model 8000 SN 80-005957 Location = TOXL 8164.14.00 09/16 05/04/2021 09:45

	DRY CAL CHECK	
Test	AC	Time
01 Room Air	0.000	09:46
02 Std. Gas	0.079	09:46
03 Room Air	0.000	09:47
04 Std. Gas	0.079	09:47
05 Room Air	0.000	09:48
06 Std. Gas	0.080	09:48
07 Room Air	0.000	09:49

Lot No = 02621080A1Cyl No = 25Exp Date = 03/05/2023County = 08

Oper No. = 666666

UU

Operator Signature

CHARLES EDER 1 ALIBRATION CAECK

0:080AC Form 106-18000

Intoxilyzer Test Record and Checklist NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. IntoxilyzerAlcohol AnalyzerNorth Dakota Model 8000SN 80-005957Location = TOXL8164.14.00 09/1605/04/202109:49

	DRY CAL CHECK	]
Test	AC	Time
01 Room Air 02 Std. Gas 03 Room Air 04 Std. Gas 05 Room Air 06 Std. Gas 07 Room Air	0.000 0.079 0.000 0.080 0.000 0.079 0.000	09:50 09:50 09:51 09:51 09:52 09:52

Lot No = 02621080A1 Cyl No = 25 Exp Date = 03/05/2023 County = 08

Oper No. = 666666

C/

Operator Signature CHARLES EDER

0.080AC

Form 106-18000