

**INTOXILYZER® 8000 INSPECTION**

Intoxilyzer® 8000 Serial Number: 80-004956

Location: TOXL

A. General Setup and Checks:

1.  Diagnostic Tests Pass and Instrument in Ready Mode
2.  Breath tube heated
3.  Date, time and location code (Level 2,E). Re-set if necessary.
4.  Print test (Level 1,P). Sign and attach test record.
5.  Tank monitor (Level 3,D,G). Display and Regulator within 50 psi.
  - a. Display: 834 psi Regulator: 850 psi

B. Tests (Sign and attach test records):

Configure simulator for the following tests (Level 1,S).

1. Wet Calibration Check
  - a.  Low AC (Level 1,C): Known Value  $\leq 0.03$  AC: 0.020 AC  
Sim. Ser #: MP3064 Lot #: 20070 Exp. Date: 2.13.22
  - b.  High AC (Level 1,C): Known Value  $\geq 0.25$  AC: 0.300 AC  
Sim. Ser #: MP5317 Lot #: 202012A Exp. Date: 12.8.22

Configure dry gas standard for the following tests (Level 1,S).

2.  Interferent Check (Level 1,B): Known Value: 0.10 AC + 0.05% Acetone  
Sim. Ser #: DR3847 Lot #: IC56 Exp. Date: NA  
 Display reads "Interferent Detect"
3.  RFI Check (CMS Mode)  Display reads "RFI Detect"
4.  Dry Calibration Check (Level 1,C): Known Value 0.080 AC  
Gas Cyl Lot #: 02621080A1 Cyl #: 25 Exp. Date: 3.5.23

Test 1	<u>0.080</u>	Test 4	<u>0.082</u>	Test 7	<u>0.082</u>
Test 2	<u>0.082</u>	Test 5	<u>0.081</u>	Test 8	<u>0.081</u>
Test 3	<u>0.081</u>	Test 6	<u>0.081</u>	Test 9	<u>0.081</u>
Average <u>0.081</u>					

C. Remarks/Maintenance: ANNUAL INSPECTION

Instrument is acceptable to be used in the field.

Charles E. Ed  
Breath Analyst Signature

5.5.2021  
Date

Kali L. Dieb  
Reviewed by

5-11-21  
Date

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000            SN 80-004956  
Location = TOXL                    8164.14.00 09/16  
05/05/2021                            15:44

\*\*\*\*\* Printer Test \*\*\*\*\*

abcdefghijklmnopqrstuvwxy1234567890-|=|  
ABCDEFGHIJKLMNOPQRSTUVWXYZ!@#%^&\*()\_+?

abcdefghijklmnopqrstuvwxy1234567890-|=|  
ABCDEFGHIJKLMNOPQRSTUVWXYZ!@#%^&\*()\_+?

Current Instrument Setup


Data Entry Mode:                    Enabled  
Start Test Sequence:                DABACABA  
Display Prelim Rslt?                Yes  
Display Third Digit?                Yes  
Inhib Printer(Y/N)?                No  
Display Volume?                    No  
Disable On Memfull?                Yes  
# of Print Copies?                  1  
Select Std (D/W/I)?                Dry  
Standard Value?                    0.080  
Standard Lot #?                    02621080A1  
Standard Cyl #?                    25  
Standard Expiration?                03/05/2023  
Oper No?                            666666

Flow Cal. Date:                    06/13/2019  
Slope                                606  
Intercept                            -586035

IR Calibration Date:                06/13/2019

	3um	9um
0th Coef(*100):	-19986	-21891
1st Coef(*100):	263289	136002
2nd Coef(*100):	2309	1332
H2O adj(mg/l*10k):	544	502

\*\*\*\*\* Printer Test End \*\*\*\*\*

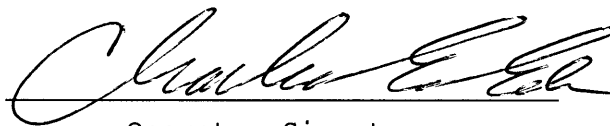

Operator Signature  
CHARLES EDER

Remarks:

PRINT TEST

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-004956  
Location = TOXL      8164.14.00 09/16  
05/05/2021      16:04

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	16:05
02 Std. Sol.	0.021	16:05
03 Room Air	0.000	16:06
04 Std. Sol.	0.022	16:07
05 Room Air	0.000	16:07
06 Std. Sol.	0.022	16:08
07 Room Air	0.000	16:08

08 Sim Temp = 34.0°C

Simul Ser No = MP3064  
Std Sol No = 20070  
County = 08      Oper No. = 666666

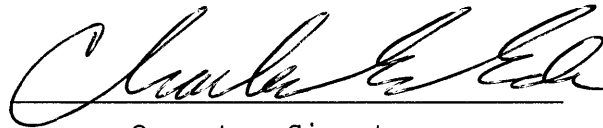

Operator Signature  
CHARLES EDER

Remarks:

Low AC  
0.020 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-004956  
Location = TOXL      8164.14.00 09/16  
05/05/2021      16:09

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	16:10
02 Std. Sol.	0.304	16:11
03 Room Air	0.000	16:11
04 Std. Sol.	0.306	16:12
05 Room Air	0.000	16:12
06 Std. Sol.	0.305	16:13
07 Room Air	0.000	16:14

08 Sim Temp = 34.0°C

Simul Ser No = MP5317  
Std Sol No = 202012A  
County = 08      Oper No. = 666666

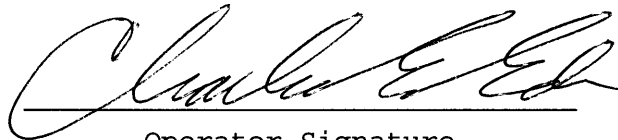

Operator Signature  
CHARLES EDER

Remarks: HIGH AC  
0.300 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-004956  
Location = TOXL                 8164.14.00 09/16  
05/05/2021                        16:15

Test	AC	Time
01 Room Air	0.000	16:16
02 *Subject Test	INT*	16:16
03 Room Air	0.000	16:17

\*Invalid Test  
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 01/08/1987  
Sub Sex = Male                    Weight = 150  
Test = DUI                        Cit = INTERFERENT  
Dr. Lic. = ND/DIS871456  
Lot No = 02621080A1  
Cyl No = 25  
Expiration Date = 03/05/2023  
County = 08                        Oper No. = 666666



Operator Signature  
CHARLES EDER

Remarks:

*Interferent Check*  
*0.10 AC + 0.05% ACETONE*

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-004956  
Location = TOXL                 8164.14.00 09/16  
05/05/2021                        16:17

Test	AC	Time
01 Diagnostic	OK	16:18
02 Room Air	0.000	16:18
03 *Subject Test	RFI*	16:19
04 Room Air	0.000	16:19

\*Invalid Test  
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 01/08/1987  
Sub Sex = Male                     Weight = 150  
Test = DUI                         Cit = RFI CHECK  
Dr. Lic. = ND/DIS871456  
Lot No = 02621080A1  
Cyl No = 25  
Expiration Date = 03/05/2023  
County = 08                        Oper No. = 666666

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature  
CHARLES EDER

Remarks:

RFI CHECK

Form 106-I8000


Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-004956  
Location = TOXL      8164.14.00 09/16  
05/05/2021      16:19

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	16:20
02 Std. Gas	0.080	16:20
03 Room Air	0.000	16:21
04 Std. Gas	0.082	16:21
05 Room Air	0.000	16:22
06 Std. Gas	0.081	16:22
07 Room Air	0.000	16:23

Lot No = 02621080A1  
Cyl No = 25  
Exp Date = 03/05/2023  
County = 08      Oper No. = 666666



Operator Signature  
CHARLES EDER

Remarks: *CALIBRATION CHECK*  
*0.080 AC*

Form 106-I8000


Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-004956  
Location = TOXL                    8164.14.00 09/16  
05/05/2021                            16:23

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	16:23
02 Std. Gas	0.082	16:24
03 Room Air	0.000	16:24
04 Std. Gas	0.081	16:24
05 Room Air	0.000	16:25
06 Std. Gas	0.081	16:25
07 Room Air	0.000	16:26

Lot No = 02621080A1  
Cyl No = 25  
Exp Date = 03/05/2023  
County = 08                            Oper No. = 666666

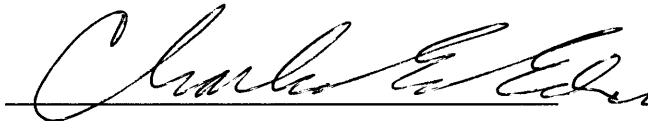  
Operator Signature  
CHARLES EDER

Remarks: *CALIBRATION CHECK*  
*0.080 AC*

Form 106-I8000



Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-004956  
Location = TOXL      8164.14.00 09/16  
05/05/2021      16:26

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	16:27
02 Std. Gas	0.082	16:27
03 Room Air	0.000	16:27
04 Std. Gas	0.081	16:28
05 Room Air	0.000	16:28
06 Std. Gas	0.081	16:29
07 Room Air	0.000	16:29

Lot No = 02621080A1  
Cyl No = 25  
Exp Date = 03/05/2023  
County = 08      Oper No. = 666666



Operator Signature  
CHARLES EDER

Remarks:

CALIBRATION CHECK  
0.080 AC

Form 106-I8000