

INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT

NORTH DAKOTA OFFICE OF ATTORNEY GENERAL CRIME LABORATORY DIVISION-TOXICOLOGY SECTION/BREATH ALCOHOL PROGRAM SFN 59281 (06/2018)

Serial Number 80-005358	Instrument Location Burleigh Morton Detention Center	
Reason for Install/Repair		
☐ Install After Receiving From Crime Laboratory ☐ Install After	er Location Change	
Other (Specify) Recieved from Crime Laboratory for annu	al check; Gas Cylinder already instal	led.
Check When Done:		
✓ 4. Enter Preliminary Data (i.e. Date, Time, DST (Y), and Locate	ion; Level 2, Function E).	
★ 6. Run Tests:		
⋈ B. ACA Test (Level 1, Function C).		
	r Level 1, Function B or C; Key Radio Du	ring Test).
▼ 7. Repair and/or Maintenance Performed (if any):		
	0496, Form 120-G) and Place it by the Ir	ntoxilyzer® for Use.
☑ 9. File Previous Intoxilyzer® Record (SFN504096, Form 120-0)	G) at the Intoxilyzer® Location at the Age	ncy.
10. Send the Following to the Crime Laboratory: Completed Int 104-G), Print Test, ACA Test, and RFI Test.	toxilyzer® 8000 Installation and Repair C	heckout (SFN59281, Form
Field Inspector Signature 4934		Date 05/07/2021
Crime Laboratory Use Only		
This installation has been reviewed and the instrument is approved to be the date the Field Inspector performed the installation. This record on County of Burleigh, North Dakota, is certified to be a true and correct of	file at the Office of Attorney General, Crir	mine alcohol concentration from ne Laboratory Division, in the
Reviewed/Certified By		Certified Date

Intoxilyzer Test Record and Checklist NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer
North Dakota Model 8000

Location = BMDC

05/07/2021

Alcohol Analyzer
SN 80-005358
8164.14.00 09/16
16:30

********* Printer Test ********

abcdefghijklmnopqrstuvwxyz1234567890-=| ABCDEFGHIJKLMNOPQRSTUVWXYZ!@#\$%^&*()_+?

abcdefghijklmnopqrstuvwxyz1234567890-=|
ABCDEFGHIJKLMNOPQRSTUVWXYZ!@#\$%^&*()_+?

Current Instrument Setup Enabled Data Entry Mode: DABACABA Start Test Sequence: Display Prelim Rslt? Yes Yes Display Third Digit? Inhib Printer(Y/N)? No No Display Volume? Yes Disable On Memfull? 1 # of Print Copies? Dry Select Std (D/W/I)? 0.080 Standard Value? 24119080A1 Standard Lot #? Standard Cyl #? 30 11/05/2021 Standard Expiration? 130315 Oper No?

Flow Cal. Date: 06/04/2019 Slope 570 Intercept 4079

IR Calibration Date: 06/04/2019
3um 9um

Oth Coef(*100): -19471 -24652
1st Coef(*100): 273237 143582
2nd Coef(*100): 2965 1293
H2O adj(mg/l*10k): 273 374

****** Printer Test End *******

Operator Signature
JARED LEMIEUX

Remarks:

Form 106-I8000

Intoxilyzer Test Record and Checklist NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer North Dakota Model 8000 Location = BMDC 05/07/2021

SN 80-005358 8164.14.00 09/16 16:31

DRY CAL CHECK

Test		AC	Time
01 Room 02 Std. 03 Room 04 Std. 05 Room 06 Std. 07 Room	Gas Air Gas Air Gas	0.000 0.081 0.000 0.081 0.000 0.082 0.000	16:31 16:32 16:32 16:33 16:33 16:34

Lot No = 24119080A1

Cyl No = 30

Exp Date = 11/05/2021

County = 08

Oper No. = 130315

Remarks:

Form 106-I8000

Operator Signatu JARED LEMIEUX

Intoxilyzer Test Record and Checklist NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer
North Dakota Model 8000
Location = BMDC
05/07/2021

Alcohol Analyzer SN 80-005358 8164.14.00 09/16 16:35

DRY CAL CHECK

Test	AC	Time
01 Room Air	RFI*	16:35
02 Room Air	RFI*	16:35

*Invalid Test Inhibited - RFI

Lot No = 24119080A1

Cyl No = 30

Exp Date = 11/05/2021

County = 08

Oper No. = 130315

Operator Signature JARED LEMIEUX

Remarks:

Form 106-I8000