

INTOXILYZER® 8000 INSPECTION

Intoxilyzer® 8000 Serial Number: 80-005937

Location: TOXL

A. General Setup and Checks:

1. Diagnostic Tests Pass and Instrument in Ready Mode
2. Breath tube heated
3. Date, time and location code (Level 2,E). Re-set if necessary.
4. Print test (Level 1,P). Sign and attach test record.
5. Tank monitor (Level 3,D,G). Display and Regulator within 50 psi.
 - a. Display: 1036 psi Regulator: 1000 psi

B. Tests (Sign and attach test records):

Configure simulator for the following tests (Level 1,S).

1. Wet Calibration Check
 - a. Low AC (Level 1,C): Known Value ≤ 0.03 AC: 0.030 AC
Sim. Ser #: MP5289 Lot #: 201911E Exp. Date: 11.19.21
 - b. High AC (Level 1,C): Known Value ≥ 0.25 AC: 0.400 AC
Sim. Ser #: MP3062 Lot #: 202103E Exp. Date: 3.24.23

Configure dry gas standard for the following tests (Level 1,S).

2. Interferent Check (Level 1,B): Known Value: 0.10 AC + 0.05% Acetone
Sim. Ser #: DR3847 Lot #: JCS 6 Exp. Date: NA
 Display reads "Interferent Detect"
3. RFI Check (CMS Mode) Display reads "RFI Detect"
4. Dry Calibration Check (Level 1,C): Known Value 0.080 AC
Gas Cyl Lot #: 24119080A1 Cyl #: 5 Exp. Date: 11.5.21

Test 1	<u>0.079</u>	Test 4	<u>0.080</u>	Test 7	<u>0.080</u>
Test 2	<u>0.080</u>	Test 5	<u>0.079</u>	Test 8	<u>0.079</u>
Test 3	<u>0.079</u>	Test 6	<u>0.079</u>	Test 9	<u>0.079</u>
Average	<u>0.079</u>				

C. Remarks/Maintenance: ANNUAL INSPECTION, REPLACED EXTERNAL BATTERY, REPLACED SIMULATOR RETURN O-RING, REPLACED GAS GAUGE O-RING.

Instrument is acceptable to be used in the field.

Charles E. Hill
Breath Analyst Signature

5/27/2021
Date

Kali J. Hill
Reviewed by

6/1/21
Date

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005937
Location = TOXL 8164.14.00 09/16
05/27/2021 11:43

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:44
02 Std. Sol.	0.030	11:44
03 Room Air	0.000	11:45
04 Std. Sol.	0.030	11:46
05 Room Air	0.000	11:46
06 Std. Sol.	0.031	11:47
07 Room Air	0.000	11:47

08 Sim Temp = 34.0°C

Simul Ser No = MP5289

Std Sol No = 201911E

County = 08

Oper No. = 666666



Operator Signature

CHARLES EDER

Remarks:

Low AC
0.030 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005937
Location = TOXL 8164.14.00 09/16
05/27/2021 11:48

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:49
02 Std. Sol.	0.398	11:50
03 Room Air	0.000	11:50
04 Std. Sol.	0.400	11:51
05 Room Air	0.000	11:52
06 Std. Sol.	0.400	11:52
07 Room Air	0.000	11:53

08 Sim Temp = 34.0°C

Simul Ser No = MP3062
Std Sol No = 202103E
County = 08 Oper No. = 666666

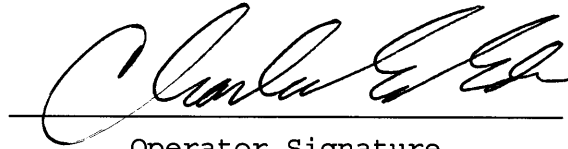

Operator Signature
CHARLES EDER

Remarks:

HIGH AC
0.400 AC

Form 106-I8000


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005937
Location = TOXL 8164.14.00 09/16
05/27/2021 11:56

Test	AC	Time
01 Room Air	0.000	11:57
02 *Subject Test	INT*	11:57
03 Room Air	0.000	11:58

*Invalid Test
Interferent Detected

Sub Name = TEST, DONOR2 NONE
Sub DOB = 07/25/1998
Sub Sex = Male Weight = NA
Test = DUI Cit = INTERFERENT
Dr. Lic. = ND/TES989643
Lot No = 24119080A1
Cyl No = 5
Expiration Date = 11/05/2021
County = 08 Oper No. = 666666

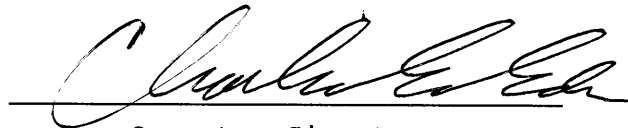

Operator Signature
CHARLES EDER

Remarks:

Interferent Check
0.10 AC + 0.05% ACETONE

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005937
Location = TOXL 8164.14.00 09/16
05/27/2021 11:59

Test	AC	Time
01 Diagnostic	OK	12:00
02 Room Air	0.000	12:00
03 *Subject Test	RFI*	12:00
04 Room Air	0.000	12:01

*Invalid Test
Inhibited - RFI

Sub Name = TEST, DONOR2 NONE
Sub DOB = 07/25/1998
Sub Sex = Male Weight = NA
Test = DUI Cit = RFI
Dr. Lic. = ND/CHECK
Lot No = 24119080A1
Cyl No = 5
Expiration Date = 11/05/2021
County = 08 Oper No. = 666666

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

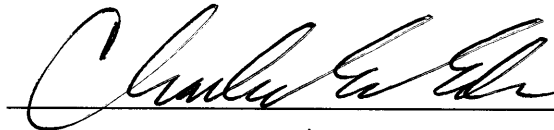

Operator Signature
CHARLES EDER

Remarks:

RFI CHECK

Form 106-I8000


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005937
Location = TOXL 8164.14.00 09/16
05/27/2021 12:01

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	12:02
02 Std. Gas	0.079	12:02
03 Room Air	0.000	12:03
04 Std. Gas	0.080	12:03
05 Room Air	0.000	12:03
06 Std. Gas	0.079	12:04
07 Room Air	0.000	12:04

Lot No = 24119080A1
Cyl No = 5
Exp Date = 11/05/2021
County = 08 Oper No. = 666666

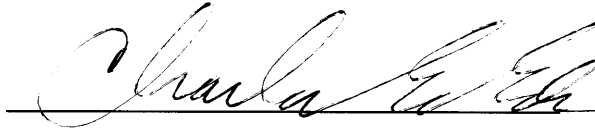

Operator Signature
CHARLES EDER

Remarks:

CALIBRATION CHECK
0.080 AC

Form 106-I8000


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005937
Location = TOXL 8164.14.00 09/16
05/27/2021 12:05

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	12:05
02 Std. Gas	0.080	12:06
03 Room Air	0.000	12:06
04 Std. Gas	0.079	12:07
05 Room Air	0.000	12:07
06 Std. Gas	0.079	12:07
07 Room Air	0.000	12:08

Lot No = 24119080A1
Cyl No = 5
Exp Date = 11/05/2021
County = 08 Oper No. = 666666

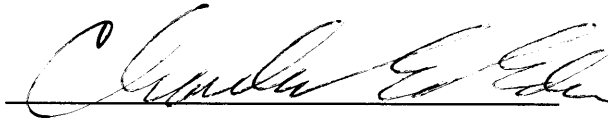

Operator Signature
CHARLES EDER

Remarks:

CALIBRATION CHECK
0.080 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005937
Location = TOXL 8164.14.00 09/16
05/27/2021 12:09

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	12:09
02 Std. Gas	0.080	12:10
03 Room Air	0.000	12:10
04 Std. Gas	0.079	12:10
05 Room Air	0.000	12:11
06 Std. Gas	0.079	12:11
07 Room Air	0.000	12:12

Lot No = 24119080A1
Cyl No = 5
Exp Date = 11/05/2021
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: CALIBRATION CHECK
0.080 AC

Form 106-I8000