Intoxilyzer® 8000 Inspection

## INTOXILYZER® 8000 INSPECTION

Intoxilvzer<sup>®</sup> 8000 Serial Number: 80-005943

Location: TOXL

- A. General Setup and Checks:
  - 1. Characteristic Tests Pass and Instrument in Ready Mode
  - 2. Breath tube heated
  - 3. X Date, time and location code (Level 2,E). Re-set if necessary.
  - 4. Print test (Level 1,P). Sign and attach test record.
  - 5. X Tank monitor (Level 3,D,G). Display and Regulator within 50 psi.
    - a. Display: <u>/98</u> psi Regulator: <u>225</u> psi
- B. Tests (Sign and attach test records):
  - Configure simulator for the following tests (Level 1,S).
  - 1. Wet Calibration Check
    - a. X Low AC (Level 1,C): Known Value  $\leq 0.03$  AC: 0.020 AC Sim. Ser #: <u>MP3069</u> Lot #: 20070 Exp. Date: 2.13-22
    - b. A High AC (Level 1,C): Known Value  $\geq 0.25$  AC: 0.300 AC Sim. Ser #: MP5317 Lot #: 2020/2A Exp. Date:  $/2 \cdot 8 \cdot 22$

Configure dry gas standard for the following tests (Level 1,S).

- 3. X RFI Check (CMS Mode) X Display reads "RFI Detect"
- C. Remarks/Maintenance: <u>ANNUAL INSPECTION, REPLACED</u> EXTELNAL BATTERY.

X Instrument is acceptable to be used in the field.

Breath Analyst Signature

Reviewed by

 $\frac{5/27/202}{Date}$ 

02/18 Issuing Authority - CEE

Page 1 of 1

SN 80-005943

Intoxilyzer Test Record and Checklist NDOAG Crime Lab. Div., Bismarck, ND 58501 CMI, Inc. Intoxilyzer Alcohol Analyzer North Dakota Model 8000 SN 80-005943 Location = TOXL 8164.14.00 09/16 05/27/2021 12:33 abcdefghijklmnopgrstuvwxyz1234567890-=| ABCDEFGHIJKLMNOPQRSTUVWXYZ!@#\$%^&\*() +? abcdefghijklmnopqrstuvwxyz1234567890-= ABCDEFGHIJKLMNOPQRSTUVWXYZ!@#\$%^&\*() +? Current Instrument Setup Data Entry Mode: Enabled Start Test Sequence: DABACABA Display Prelim Rslt? Yes Display Third Digit? Yes Inhib Printer(Y/N)? No Display Volume? No Disable On Memfull? Yes # of Print Copies? 1 Select Std (D/W/I)? Dry Standard Value? 0.080 Standard Lot #? 05620080A1 Standard Cyl #? 10 Standard Expiration? 04/05/2022 Oper No? 666666 Flow Cal. Date: 06/20/2013 Slope 686 Intercept -735246 IR Calibration Date: 08/26/2016 3um 9um \_\_\_\_\_ Oth Coef(\*100): -13589 1st Coef(\*100): 284566 2nd Coef(\*100): 2717 -19547 139614 1336 H2O adj(mq/l\*10k): 381404 

Operator Signature CHARLES EDER

RINT TEST

Form 106-18000

CMI, Inc. IntoxilyzerAlcohol AnalyzerNorth Dakota Model 8000SN 80-005943Location = TOXL8164.14.00 09/1605/27/202112:38

	WET CAL CHECK	
Test	AC	Time
01 Room Air	0.000	12:38
02 Std. Sol.	0.019	12:39
03 Room Air	0.000	12:40
04 Std. Sol.	0.019	12:40
05 Room Air	0.000	12:41
06 Std. Sol.	0.020	12:42
07 Room Air	0.000	12:42
08 Sim Temp =	34.0°C	

Simul Ser No = MP3064 Std Sol No = 20070 County = 08

Oper No. = 666666

le Car 4

Operator Signature CHARLES EDER

Low AC 0.020 AC

Form 106-18000

CMI, Inc. IntoxilyzerAlcohol AnalyzerNorth Dakota Model 8000SN 80-005943Location = TOXL8164.14.00 09/1605/27/202112:43

	WET CAL CHECK	
Test	AC	Time
01 Room Air	0.000	12:44
02 Std. Sol.	0.301	12:44
03 Room Air	0.000	12:45
04 Std. Sol.	0.301	12:46
05 Room Air	0.000	12:46
06 Std. Sol.	0.301	12:47
07 Room Air	0.000	12:48

 $08 \text{ Sim Temp} = 34.0^{\circ}\text{C}$ 

Simul Ser No = MP5317 Std Sol No = 202012A County = 08

Oper No. = 666666

Operator Signature CHARLES EDER HAGH AC

Remarks:

Form 106-18000

0.300A

Intoxilyzer Test Record and Checklist NDOAG Crime Lab. Div., Bismarck, ND 58501 CMI, Inc. Intoxilyzer Alcohol Analyzer North Dakota Model 8000 SN 80-005943 Location = TOXL 8164.14.00 09/16 05/27/2021 12:49 AC Time Test 12:50 01 Room Air 0.000 02 \*Subject Test 12:50 INT\* 03 Room Air 0.000 12:51 \*Invalid Test Interferent Detected Sub Name = TEST, DONOR2 NONE Sub DOB = 07/25/1998Weight = NA Sub Sex = Male Test = DUI Cit = INTERFERENT Dr. Lic. = ND/TES989643Lot No = 02621080A1Cyl No = 15Expiration Date = 03/05/2023County = 08Oper No. = 666666

Operator Signature Interferent Clieck 0.10AC + 0.05% ACETONE CHARLES, EDER Remarks:

Form 106-18000

Intoxilyzer Test Record and Checklist NDOAG Crime Lab. Div., Bismarck, ND 58501 CMI, Inc. Intoxilyzer Alcohol Analyzer North Dakota Model 8000 SN 80-005943 Location = TOXL 8164.14.00 09/16 05/27/2021 12:51 Test AC Time 12:52 01 Diagnostic OK 02 Room Air 0.000 12:53 03 \*Subject Test RFI\* 12:53 04 Room Air 0.000 12:53 \*Invalid Test Inhibited - RFI Sub Name = TEST, DONOR2 NONE Sub DOB = 07/25/1998Sub Sex = Male Weight = NA Cit = RFI CHECK Test = DUI Dr. Lic. = ND/TES989643Lot No = 02621080A1Cyl No = 15Expiration Date = 03/05/2023County = 08Oper No. = 666666

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

Operator Signature CHARLES EDER

Remarks:

MECK

Form 106-18000

Intoxilyzer Test Record and Checklist NDOAG Crime Lab. Div., Bismarck, ND 58501 CMI, Inc. Intoxilyzer Alcohol Analyzer North Dakota Model 8000 SN 80-005943 Location = TOXL 8164.14.00 09/16 05/27/2021 12:54 DRY CAL CHECK Test AC Time 01 Room Air 0.000 12:54 02 Std. Gas 0.080 12:55 03 Room Air 0.000 12:55 04 Std. Gas 0.081 12:55 05 Room Air 0.000 12:56 06 Std. Gas 0.080 12:56 07 Room Air 12:57 0.000 Lot No = 02621080A1Cyl No = 15Exp Date = 03/05/2023County = 08Oper No. = 666666Operator Signature CHARLES EDER ALIBRATION CHECK Remarks: 0.080AC Form 106-18000

CMI, Inc. Intoxilyzer Alcohol Analyzer North Dakota Model 8000 SN 80-005943 Location = TOXL 8164.14.00 09/16 05/27/2021 12:57

	DRY CAL CHECK	]
Test	AC	Time
01 Room Air 02 Std. Gas 03 Room Air 04 Std. Gas 05 Room Air 06 Std. Gas 07 Room Air	0.000 0.080 0.000 0.080 0.000 0.081 0.000	12:58 12:58 12:59 12:59 12:59 12:59 13:00 13:00

Lot No = 02621080A1Cyl No = 15Exp Date = 03/05/2023County = 08

Oper No. = 666666

Operator Signature

ALIBRATION CHECK 0.080A

Form 106-I8000

CMI, Inc. Intoxilyzer Alcohol Analyzer SN 80-005943 North Dakota Model 8000 Location = TOXL8164.14.00 09/16 05/27/2021 13:01

	DRY CAL CHECK	]
Test	AC	Time
01 Room Air 02 Std. Gas 03 Room Air 04 Std. Gas 05 Room Air 06 Std. Gas 07 Room Air	0.000 0.081 0.000 0.080 0.000 0.081 0.000	13:01 13:02 13:02 13:03 13:03 13:03 13:04

Lot No = 02621080A1Cyl No = 15Exp Date = 03/05/2023County = 08

Oper No. = 666666

Operator Signature CHARLES EDER

ator Signature HARLES EDER UBRAMON CHECK

Form 106-I8000