



ETHANOL GAS STANDARD CYLINDER REPORT
 NORTH DAKOTA OFFICE OF ATTORNEY GENERAL
 CRIME LABORATORY DIVISION-TOXICOLOGY SECTION/BREATH ALCOHOL PROGRAM
 SFN 59282 (08/2018)

Chemical Test Operator Name (Print) <i>Dumas, Anthony G.</i>		
Location <i>Grafton PD</i>	Intoxilyzer® Serial Number <i>80-006501</i>	
Gas Lot Number <i>0262108041</i>	Gas Cylinder Number <i>018</i>	Gas Expiration Date <i>3/5/2023</i>

Check When Done:

- 1. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 2. Perform an ACA Test (Level 1, Function C).
 Write in the ACA Test Results (Reported to 3 Digits, Ex. 0.081).
 A. *0.079* AC
 B. *0.079* AC
 C. *0.079* AC
- 3. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 4. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 5. Send the Following to the Crime Laboratory:
 - A. Ethanol Gas Standard Cylinder Report (SFN59282, Form 105-G).
 - B. ACA Test Record.

Chemical Test Operator Signature 	Date <i>27 MAY 2021</i>
Reviewed By (Crime Laboratory Use Only) 	Date <i>6-4-2021</i>

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006501
Location = GFTN 8164.14.00 09/16
05/27/2021 18:10

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	18:11
02 Std. Gas	0.079	18:11
03 Room Air	0.000	18:12
04 Std. Gas	0.079	18:12
05 Room Air	0.000	18:13
06 Std. Gas	0.079	18:13
07 Room Air	0.000	18:13

Lot No = 02621080A1
Cyl No = 18
Exp Date = 03/05/2023
County = 50 Oper No. = 020305



Operator Signature
ANTHONY DUMAS

Remarks: ACA Test after install
* Annual Inspection *
Form 106-I8000