

INTOXILYZER® 8000 INSPECTION

Intoxilyzer® 8000 Serial Number: 80-005946

Location: TOXL

A. General Setup and Checks:

1.  Diagnostic Tests Pass and Instrument in Ready Mode
2.  Breath tube heated
3.  Date, time and location code (Level 2,E). Re-set if necessary.
4.  Print test (Level 1,P). Sign and attach test record.
5.  Tank monitor (Level 3,D,G). Display and Regulator within 50 psi.
  - a. Display: 950 psi Regulator: 975 psi

B. Tests (Sign and attach test records):

Configure simulator for the following tests (Level 1,S).

1. Wet Calibration Check

- a.  Low AC (Level 1,C): Known Value  $\leq 0.03$  AC: 0.020 AC  
Sim. Ser #: MP3064 Lot #: 20070 Exp. Date: 2.13.22
- b.  High AC (Level 1,C): Known Value  $\geq 0.25$  AC: 0.300 AC  
Sim. Ser #: MP5317 Lot #: 2020124 Exp. Date: 12.8.22

Configure dry gas standard for the following tests (Level 1,S).

2.  Interferent Check (Level 1,B): Known Value: 0.10 AC + 0.05% Acetone  
Sim. Ser #: DR3847 Lot #: ICS6 Exp. Date: NA  
 Display reads "Interferent Detect"
3.  RFI Check (CMS Mode)  Display reads "RFI Detect"
4.  Dry Calibration Check (Level 1,C): Known Value 0.080 AC  
Gas Cyl Lot #: 02621080 A1 Cyl #: 26 Exp. Date: 3.5.23

Test 1	<u>0.080</u>	Test 4	<u>0.079</u>	Test 7	<u>0.080</u>
Test 2	<u>0.080</u>	Test 5	<u>0.080</u>	Test 8	<u>0.080</u>
Test 3	<u>0.079</u>	Test 6	<u>0.080</u>	Test 9	<u>0.080</u>
Average	<u>0.080</u>				

C. Remarks/Maintenance: ANNUAL INSPECTION

Instrument is acceptable to be used in the field.

[Signature]  
Breath Analyst Signature

6/2/21  
Date

[Signature]  
Reviewed by

6-7-21  
Date

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer                Alcohol Analyzer  
North Dakota Model 8000                SN 80-005946  
Location = TOXL                        8164.14.00 09/16  
06/02/2021                                11:30

\*\*\*\*\* Printer Test \*\*\*\*\*

abcdefghijklmnopqrstuvwxy z1234567890- = |  
ABCDEFGHIJKLMN OPQRSTUVWXYZ!@# \$%^ &\* ( ) \_ + ?

abcdefghijklmnopqrstuvwxy z1234567890- = |  
ABCDEFGHIJKLMN OPQRSTUVWXYZ!@# \$%^ &\* ( ) \_ + ?

Current Instrument Setup


Data Entry Mode:                        Enabled  
Start Test Sequence:                      DABACABA  
Display Prelim Rslt?                      Yes  
Display Third Digit?                       Yes  
Inhib Printer(Y/N)?                       No  
Display Volume?                           No  
Disable On Memfull?                       Yes  
# of Print Copies?                         1  
Select Std (D/W/I)?                       Dry  
Standard Value?                           0.080  
Standard Lot #?                           05620080A1  
Standard Cyl #?                           6  
Standard Expiration?                       04/05/2022  
Oper No?                                   666666

Flow Cal. Date:                           06/02/2021  
Slope                                       642  
Intercept                                   -600927

IR Calibration Date:                       06/02/2021

	3um	9um
0th Coef(*100):	-20165	-16483
1st Coef(*100):	290371	141330
2nd Coef(*100):	953	910
H2O adj(mg/l*10k):	496	404

\*\*\*\*\* Printer Test End \*\*\*\*\*

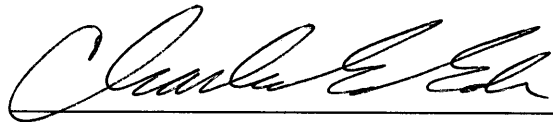

Operator Signature  
CHARLES EDER

Remarks:

PRINT TEST

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-005946  
Location = TOXL      8164.14.00 09/16  
06/02/2021      11:30

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:31
02 Std. Sol.	0.019	11:32
03 Room Air	0.000	11:32
04 Std. Sol.	0.020	11:33
05 Room Air	0.000	11:33
06 Std. Sol.	0.019	11:34
07 Room Air	0.000	11:35

08 Sim Temp = 34.0°C

Simul Ser No = MP3064

Std Sol No = 20070

County = 08

Oper No. = 666666



Operator Signature

CHARLES EDER

Remarks:

Low AC  
0.020 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-005946  
Location = TOXL      8164.14.00 09/16  
06/02/2021      11:35

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:36
02 Std. Sol.	0.300	11:36
03 Room Air	0.000	11:37
04 Std. Sol.	0.300	11:38
05 Room Air	0.000	11:38
06 Std. Sol.	0.301	11:39
07 Room Air	0.000	11:40

08 Sim Temp = 34.0°C

Simul Ser No = MP5317  
Std Sol No = 202012A  
County = 08      Oper No. = 666666

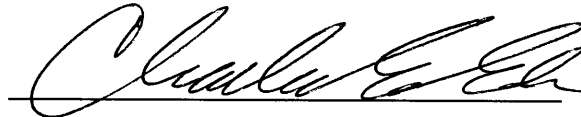

Operator Signature  
CHARLES EDER

Remarks:      HIGH AC  
                    0.300 AC

Form 106-I8000


Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-005946  
Location = TOXL      8164.14.00 09/16  
06/02/2021      11:40

Test	AC	Time
01 Room Air	0.000	11:41
02 *Subject Test	INT*	11:41
03 Room Air	0.000	11:42

\*Invalid Test  
Interferent Detected

Sub Name = TEST, DONOR2 NONE  
Sub DOB = 07/25/1998  
Sub Sex = Male      Weight = NA  
Test = DUI      Cit = INTERFERENT  
Dr. Lic. = ND/TES989643  
Lot No = 02621080A1  
Cyl No = 26  
Expiration Date = 03/05/2023  
County = 08      Oper No. = 666666



Operator Signature  
CHARLES EDER

Remarks:

~~RFT CHECK~~ CEE  
Interferent Check 0.10AC +0.05% ACETONE  
Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000            SN 80-005946  
Location = TOXL                    8164.14.00 09/16  
06/02/2021                            11:42

Test	AC	Time
01 Diagnostic	OK	11:43
02 Room Air	0.000	11:43
03 *Subject Test	RFI*	11:44
04 Room Air	0.000	11:44

\*Invalid Test  
Inhibited - RFI

Sub Name = TEST, DONOR2 NONE  
Sub DOB = 07/25/1998  
Sub Sex = Male                            Weight = NA  
Test = DUI                                Cit = RFI CHECK  
Dr. Lic. = ND/TES989643  
Lot No = 02621080A1  
Cyl No = 26  
Expiration Date = 03/05/2023  
County = 08                                Oper No. = 666666

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

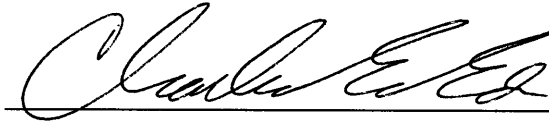

Operator Signature  
CHARLES EDER

Remarks: *RFI CHECK*

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-005946  
Location = TOXL      8164.14.00 09/16  
06/02/2021      11:44

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:45
02 Std. Gas	0.080	11:45
03 Room Air	0.000	11:46
04 Std. Gas	0.080	11:46
05 Room Air	0.000	11:47
06 Std. Gas	0.079	11:47
07 Room Air	0.000	11:48

Lot No = 02621080A1  
Cyl No = 26  
Exp Date = 03/05/2023  
County = 08      Oper No. = 666666



Operator Signature  
CHARLES EDER

Remarks: CALIBRATION CHECK  
0.080 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-005946  
Location = TOXL      8164.14.00 09/16  
06/02/2021      11:48

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:48
02 Std. Gas	0.079	11:49
03 Room Air	0.000	11:49
04 Std. Gas	0.080	11:50
05 Room Air	0.000	11:50
06 Std. Gas	0.080	11:50
07 Room Air	0.000	11:51

Lot No = 02621080A1  
Cyl No = 26  
Exp Date = 03/05/2023  
County = 08      Oper No. = 666666



Operator Signature  
CHARLES EDER

Remarks:

CALIBRATION CHECK  
0.080AC

Form 106-I8000




Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-005946  
Location = TOXL      8164.14.00 09/16  
06/02/2021      11:51

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:52
02 Std. Gas	0.080	11:52
03 Room Air	0.000	11:53
04 Std. Gas	0.080	11:53
05 Room Air	0.000	11:54
06 Std. Gas	0.080	11:54
07 Room Air	0.000	11:55

Lot No = 02621080A1  
Cyl No = 26  
Exp Date = 03/05/2023  
County = 08      Oper No. = 666666

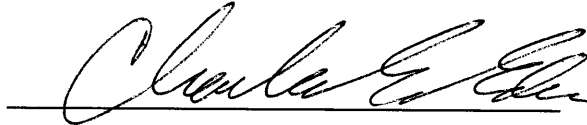

Operator Signature  
CHARLES EDER

Remarks:

CALIBRATION CHECK  
0.080 AC

Form 106-I8000