

INTOXILYZER® 8000 INSPECTION

Intoxilyzer® 8000 Serial Number: 80-005951 Location: TOXL

A. General Setup and Checks:

- 1. Diagnostic Tests Pass and Instrument in Ready Mode
- 2. Breath tube heated
- 3. Date, time and location code (Level 2,E). Re-set if necessary.
- 4. Print test (Level 1,P). Sign and attach test record.
- 5. Tank monitor (Level 3,D,G). Display and Regulator within 50 psi.
 - a. Display: 941 psi Regulator: 950 psi

B. Tests (Sign and attach test records):

Configure simulator for the following tests (Level 1,S).

- 1. Wet Calibration Check
 - a. Low AC (Level 1,C): Known Value \leq 0.03 AC: 0.030 AC
Sim. Ser #: MP5289 Lot #: 201911E Exp. Date: 11-19-21
 - b. High AC (Level 1,C): Known Value \geq 0.25 AC: 0.400 AC
Sim. Ser #: MP3062 Lot #: 202103E Exp. Date: 3-4-23

Configure dry gas standard for the following tests (Level 1,S).

- 2. Interferent Check (Level 1,B): Known Value: 0.10 AC + 0.05% Acetone
Sim. Ser #: DR3847 Lot #: ICS6 Exp. Date: NA
 Display reads "Interferent Detect"
- 3. RFI Check (CMS Mode) Display reads "RFI Detect"
- 4. Dry Calibration Check (Level 1,C): Known Value 0.080 AC
Gas Cyl Lot #: 02621080A1 Cyl #: 26 Exp. Date: 3-5-23

Test 1	<u>0.081</u>	Test 4	<u>0.081</u>	Test 7	<u>0.081</u>
Test 2	<u>0.081</u>	Test 5	<u>0.080</u>	Test 8	<u>0.081</u>
Test 3	<u>0.081</u>	Test 6	<u>0.081</u>	Test 9	<u>0.081</u>
Average	<u>0.081</u>				

C. Remarks/Maintenance: ANNUAL INSPECTION

REPLACED EXTERNAL BATTERY CEE 6-8-21

Instrument is acceptable to be used in the field.

Charles E. Cee
Breath Analyst Signature

6/8/21
Date

Kate J. Dietz
Reviewed by

6-8-21
Date

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005951
Location = TOXL 8164.14.00 09/16
06/07/2021 12:52

***** Printer Test *****

abcdefghijklmnopqrstuvwxyz1234567890- = |
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&^*()_+?

abcdefghijklmnopqrstuvwxyz1234567890- = |
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&^*()_+?

Current Instrument Setup

Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 24119080A1
Standard Cyl #? 1
Standard Expiration? 11/05/2021
Oper No? 666666

Flow Cal. Date: 06/21/2013
Slope 693
Intercept -721277

IR Calibration Date: 05/08/2017
 3um 9um

0th Coef(*100): -17201 -17496
1st Coef(*100): 267144 137813
2nd Coef(*100): 2948 1389
H2O adj(mg/l*10k): 438 428

***** Printer Test End *****



Operator Signature
CHARLES EDER

Remarks: PRINT TEST

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005951
Location = TOXL 8164.14.00 09/16
06/07/2021 12:54

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	12:55
02 Std. Sol.	0.030	12:56
03 Room Air	0.000	12:56
04 Std. Sol.	0.030	12:57
05 Room Air	0.000	12:58
06 Std. Sol.	0.030	12:58
07 Room Air	0.000	12:59

08 Sim Temp = 34.0°C

Simul Ser No = MP5289
Std Sol No = 201911E
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: *Low AC*
0.030 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005951
Location = TOXL 8164.14.00 09/16
06/07/2021 13:08

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	13:08
02 Std. Sol.	0.404	13:09
03 Room Air	0.000	13:10
04 Std. Sol.	0.404	13:10
05 Room Air	0.000	13:11
06 Std. Sol.	0.404	13:12
07 Room Air	0.000	13:12

08 Sim Temp = 34.0°C

Simul Ser No = MP3062
Std Sol No = 202103E
County = 08

Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks:

HIGH AC
0.400 AC

Form 106-I8000


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005951
Location = TOXL 8164.14.00 09/16
06/07/2021 13:15

Test	AC	Time
01 Room Air	0.000	13:15
02 *Subject Test	INT*	13:16
03 Room Air	0.000	13:16

*Invalid Test
Interferent Detected

Sub Name = TEST, DONOR2 NONE
Sub DOB = 07/25/1998
Sub Sex = Male Weight = NA
Test = DUI Cit = INTERFERENT
Dr. Lic. = ND/TES989643
Lot No = 02621080A1
Cyl No = 26
Expiration Date = 03/05/2023
County = 08 Oper No. = 666666

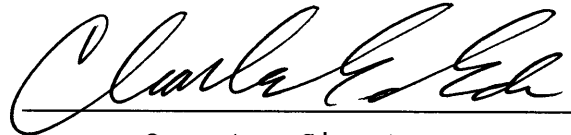

Operator Signature
CHARLES EDER

Remarks: *Interferent Check*

0.10 AC + 0.05% ACETONE

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005951
Location = TOXL 8164.14.00 09/16
06/07/2021 13:17

Test	AC	Time
01 Diagnostic	OK	13:17
02 Room Air	0.000	13:18
03 *Subject Test	RFI*	13:18
04 Room Air	0.000	13:18

*Invalid Test
Inhibited - RFI

Sub Name = TEST, DONOR2 NONE
Sub DOB = 07/25/1998
Sub Sex = Male Weight = NA
Test = DUI Cit = RFI CHECK
Dr. Lic. = ND/TES989643
Lot No = 02621080A1
Cyl No = 26
Expiration Date = 03/05/2023
County = 08 Oper No. = 666666

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

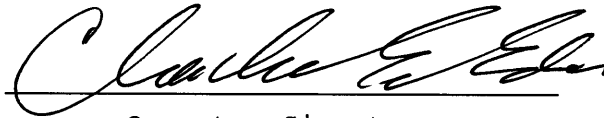

Operator Signature
CHARLES EDER

Remarks:

RFI CHECK

Form 106-I8000


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005951
Location = TOXL 8164.14.00 09/16
06/07/2021 13:19

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	13:19
02 Std. Gas	0.081	13:20
03 Room Air	0.000	13:20
04 Std. Gas	0.081	13:20
05 Room Air	0.000	13:21
06 Std. Gas	0.081	13:21
07 Room Air	0.000	13:22

Lot No = 02621080A1
Cyl No = 26
Exp Date = 03/05/2023
County = 08 Oper No. = 666666

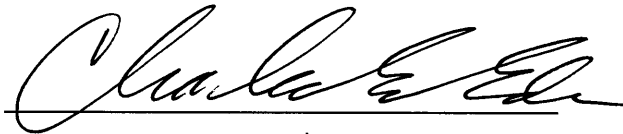

Operator Signature
CHARLES EDER

Remarks: CALIBRATION CHECK
0.080 AC

Form 106-I8000


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005951
Location = TOXL 8164.14.00 09/16
06/07/2021 13:22

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	13:23
02 Std. Gas	0.081	13:23
03 Room Air	0.000	13:24
04 Std. Gas	0.080	13:24
05 Room Air	0.000	13:24
06 Std. Gas	0.081	13:25
07 Room Air	0.000	13:25

Lot No = 02621080A1
Cyl No = 26
Exp Date = 03/05/2023
County = 08 Oper No. = 666666

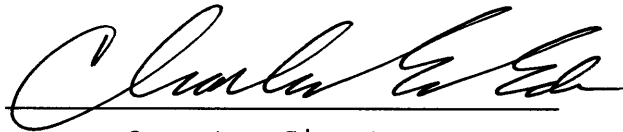

Operator Signature
CHARLES EDER

Remarks:

CALIBRATION CHECK
0.080 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005951
Location = TOXL 8164.14.00 09/16
06/07/2021 13:27

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	13:28
02 Std. Gas	0.081	13:28
03 Room Air	0.000	13:29
04 Std. Gas	0.081	13:29
05 Room Air	0.000	13:30
06 Std. Gas	0.081	13:30
07 Room Air	0.000	13:30

Lot No = 02621080A1
Cyl No = 26
Exp Date = 03/05/2023
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks:

CALIBRATION CHECK
0.080 AC

Form 106-I8000