

**INTOXILYZER® 8000 INSPECTION**

Intoxilyzer® 8000 Serial Number: 80-003044 Location: TOXL

**A. General Setup and Checks:**

- 1.  Diagnostic Tests Pass and Instrument in Ready Mode
- 2.  Breath tube heated
- 3.  Date, time and location code (Level 2,E). Re-set if necessary.
- 4.  Print test (Level 1,P). Sign and attach test record.
- 5.  Tank monitor (Level 3,D,G). Display and Regulator within 50 psi.
  - a. Display: 911 psi Regulator: 900 psi

**B. Tests (Sign and attach test records):**

- Configure simulator for the following tests (Level 1,S).
  - 1. Wet Calibration Check
    - a.  Low AC (Level 1,C): Known Value  $\leq$  0.03 AC: 0.015 AC  
Sim. Ser #: MP5319 Lot #: 201911C Exp. Date: 11-19-21
    - b.  High AC (Level 1,C): Known Value  $\geq$  0.25 AC: 0.460 AC  
Sim. Ser #: MP3060 Lot #: 1910B Exp. Date: 10-20-21

Configure dry gas standard for the following tests (Level 1,S).

- 2.  Interferent Check (Level 1,B): Known Value: 0.10 AC + 0.05% Acetone  
Sim. Ser #: DR7348 Lot #: 1C55 Exp. Date: N/A  
 Display reads "Interferent Detect"
- 3.  RFI Check (CMS Mode)  Display reads "RFI Detect"
- 4.  Dry Calibration Check (Level 1,C): Known Value 0.080 AC  
Gas Cyl Lot #: 24119080A1 Cyl #: 033 Exp. Date: 11-5-21

Test 1 <u>0.080</u>	Test 4 <u>0.080</u>	Test 7 <u>0.080</u>
Test 2 <u>0.081</u>	Test 5 <u>0.081</u>	Test 8 <u>0.080</u>
Test 3 <u>0.081</u>	Test 6 <u>0.079</u>	Test 9 <u>0.080</u>
Average <u>0.080</u>		

C. Remarks/Maintenance: Annual Inspection

Instrument is acceptable to be used in the field.

Kevin L. Aho  
Breath Analyst Signature

6-8-21  
Date

Charles E. E  
Reviewed by

6.8.21  
Date

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-003066  
Location = TOXL                8164.14.00 09/16  
06/08/2021                      14:10

\*\*\*\*\* Printer Test \*\*\*\*\*

abcdefghijklmnopqrstuvwxy1234567890-=  
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&\*()\_+?

abcdefghijklmnopqrstuvwxy1234567890-=  
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&\*()\_+?

Current Instrument Setup

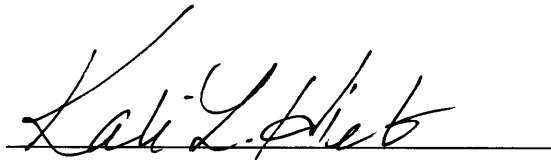
Data Entry Mode:            Enabled  
Start Test Sequence:        DABACABA  
Display Prelim Rslt?        Yes  
Display Third Digit?        Yes  
Inhib Printer(Y/N)?        No  
Display Volume?            No  
Disable On Memfull?        Yes  
# of Print Copies?         1  
Select Std (D/W/I)?        Dry  
Standard Value?            0.080  
Standard Lot #?            24119080A1  
Standard Cyl #?            9  
Standard Expiration?       11/05/2021  
Oper No?                    132598

Flow Cal. Date:            07/29/2010  
Slope                      806  
Intercept                 -895033

IR Calibration Date:       07/29/2010

	3um	9um
0th Coef(*100):	-14075	-5808
1st Coef(*100):	272113	138204
2nd Coef(*100):	1637	986
H2O adj(mg/l*10k):	289	148

\*\*\*\*\* Printer Test End \*\*\*\*\*



Operator Signature  
KALI HIEB

Remarks: *print test*

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

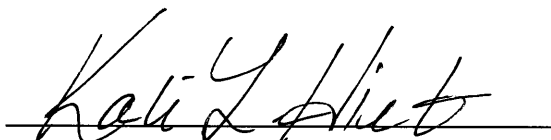
CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-003066  
Location = TOXL                    8164.14.00 09/16  
06/08/2021                            14:13

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:13
02 Std. Sol.	0.015	14:14
03 Room Air	0.000	14:15
04 Std. Sol.	0.015	14:15
05 Room Air	0.000	14:16
06 Std. Sol.	0.015	14:17
07 Room Air	0.000	14:17

08 Sim Temp = 34.0°C

Simul Ser No = MP5319  
Std Sol No = 201911C  
County = 08                            Oper No. = 132598



Operator Signature  
KALI HIEB

Remarks: *low cal check*  
*0.015 AC*

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

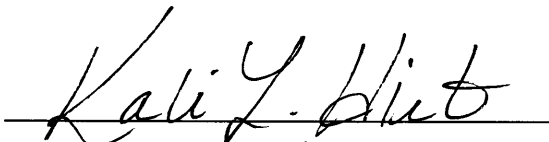
CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-003066  
Location = TOXL      8164.14.00 09/16  
06/08/2021      14:20

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:20
02 Std. Sol.	0.403	14:22
03 Room Air	0.000	14:22
04 Std. Sol.	0.407	14:23
05 Room Air	0.000	14:24
06 Std. Sol.	0.406	14:24
07 Room Air	0.000	14:25

08 Sim Temp = 33.9°C

Simul Ser No = MP3060  
Std Sol No = 19108  
County = 08      Oper No. = 132598

  
Operator Signature  
KALI HIEB

Remarks: *High cal check*  
*0.400 AC*  
Form 106-I8000

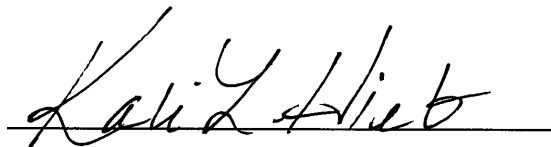
Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-003066  
Location = TOXL      8164.14.00 09/16  
06/08/2021      14:30

Test	AC	Time
01 Room Air	0.000	14:31
02 *Subject Test	INT*	14:32
03 Room Air	0.000	14:33

\*Invalid Test  
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 01/01/1982  
Sub Sex = Female      Weight = 150  
Test = DUI      Cit = NA  
Dr. Lic. = ND/DIS821456  
Lot No = 24119080A1  
Cyl No = 33  
Expiration Date = 11/05/2021  
County = 08      Oper No. = 132598



Operator Signature  
KALI HIEB

Remarks: 0.10 AC + 0.05g Acetone  
Interferent Check

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

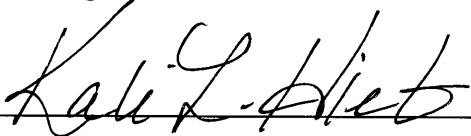
CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-003066  
Location = TOXL                    8164.14.00 09/16  
06/08/2021                            14:34

Test	AC	Time
01 Diagnostic	OK	14:35
02 Room Air	0.000	14:35
03 Reference	RFI*	14:35
04 Room Air	0.000	14:36

\*Invalid Test  
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 01/01/1982  
Sub Sex = Female                    Weight = 150  
Test = DUI                            Cit = NA  
Dr. Lic. = ND/DIS821456  
Lot No = 24119080A1  
Cyl No = 33  
Expiration Date = 11/05/2021  
County = 08                            Oper No. = 132598

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

  
Operator Signature  
KALI HIEB

Remarks: *RFI Check*

Form 106-I8000

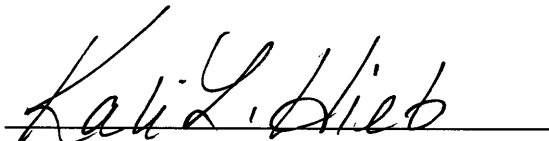
Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-003066  
Location = TOXL                    8164.14.00 09/16  
06/08/2021                            14:37

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:37
02 Std. Gas	0.080	14:38
03 Room Air	0.000	14:38
04 Std. Gas	0.081	14:39
05 Room Air	0.000	14:39
06 Std. Gas	0.081	14:40
07 Room Air	0.000	14:40

Lot No = 24119080A1  
Cyl No = 33  
Exp Date = 11/05/2021  
County = 08                            Oper No. = 132598

  
Operator Signature  
KALI HIEB

Remarks: *Dry cal check*  
*0.000 AC*  
Form 106-I8000

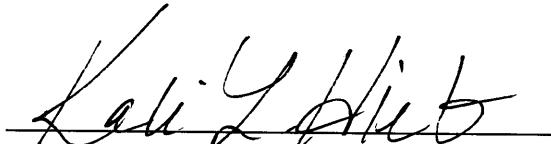
Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-003066  
Location = TOXL      8164.14.00 09/16  
06/08/2021      14:43

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:44
02 Std. Gas	0.080	14:44
03 Room Air	0.000	14:45
04 Std. Gas	0.081	14:45
05 Room Air	0.000	14:46
06 Std. Gas	0.079	14:46
07 Room Air	0.000	14:46

Lot No = 24119080A1  
Cyl No = 33  
Exp Date = 11/05/2021  
County = 08      Oper No. = 132598

  
Operator Signature  
KALI HIEB

Remarks: *Dry cal check*  
*0.080 AC*  
Form 106-I8000




Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-003066  
Location = TOXL      8164.14.00 09/16  
06/08/2021      14:47

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:48
02 Std. Gas	0.080	14:48
03 Room Air	0.000	14:49
04 Std. Gas	0.080	14:49
05 Room Air	0.000	14:50
06 Std. Gas	0.080	14:50
07 Room Air	0.000	14:51

Lot No = 24119080A1  
Cyl No = 33  
Exp Date = 11/05/2021  
County = 08      Oper No. = 132598

  
Operator Signature  
KALI HIEB

Remarks: *Dry cal check  
0.080 AC*

Form 106-I8000