

INTOXILYZER® 8000 INSPECTION

Intoxilyzer® 8000 Serial Number: 80-005357 Location: TOXL

A. General Setup and Checks:

1.  Diagnostic Tests Pass and Instrument in Ready Mode
2.  Breath tube heated
3.  Date, time and location code (Level 2,E). Re-set if necessary.
4.  Print test (Level 1,P). Sign and attach test record.
5.  Tank monitor (Level 3,D,G). Display and Regulator within 50 psi.
  - a. Display: 767 psi Regulator: 750 psi

B. Tests (Sign and attach test records):

Configure simulator for the following tests (Level 1,S).

1. Wet Calibration Check
  - a.  Low AC (Level 1,C): Known Value  $\leq$  0.03 AC: 0.030 AC  
Sim. Ser #: MPS289 Lot #: 201911E Exp. Date: 11-19-21
  - b.  High AC (Level 1,C): Known Value  $\geq$  0.25 AC: 0.400 AC  
Sim. Ser #: MP3062 Lot #: 202103E Exp. Date: 3-24-23

Configure dry gas standard for the following tests (Level 1,S).

2.  Interferent Check (Level 1,B): Known Value: 0.10 AC + 0.05% Acetone  
Sim. Ser #: DR3847 Lot #: IC56 Exp. Date: NA  
 Display reads "Interferent Detect"
3.  RFI Check (CMS Mode)  Display reads "RFI Detect"
4.  Dry Calibration Check (Level 1,C): Known Value 0.080 AC  
Gas Cyl Lot #: 2411908041 Cyl #: 31 Exp. Date: 11-5-21

Test 1	<u>0.082</u>	Test 4	<u>0.082</u>	Test 7	<u>0.082</u>
Test 2	<u>0.082</u>	Test 5	<u>0.082</u>	Test 8	<u>0.082</u>
Test 3	<u>0.083</u>	Test 6	<u>0.083</u>	Test 9	<u>0.082</u>
Average	<u>0.082</u>				

C. Remarks/Maintenance: ANNUAL INSPECTION, REPLACED EXTERNAL BATTERY, REPLACED SIMULATOR RETURN-O-RING

Instrument is acceptable to be used in the field.

Charles E. Ed  
Breath Analyst Signature

6/10/21  
Date

Kelli L. Hib  
Reviewed by

6-14-21  
Date

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-005357  
Location = TOXL      8164.14.00 09/16  
06/10/2021      12:16

\*\*\*\*\* Printer Test \*\*\*\*\*

abcdefghijklmnopqrstuvwxy1234567890-|=|  
ABCDEFGHIJKLMNopqrstuvwxyZ!@#%&\*()\_+?

abcdefghijklmnopqrstuvwxy1234567890-|=|  
ABCDEFGHIJKLMNopqrstuvwxyZ!@#%&\*()\_+?

Current Instrument Setup


Data Entry Mode:      Enabled  
Start Test Sequence:      DABACABA  
Display Prelim Rslt?      Yes  
Display Third Digit?      Yes  
Inhib Printer(Y/N)?      No  
Display Volume?      No  
Disable On Memfull?      Yes  
# of Print Copies?      1  
Select Std (D/W/I)?      Dry  
Standard Value?      0.080  
Standard Lot #?      05620080A1  
Standard Cyl #?      5  
Standard Expiration?      04/05/2022  
Oper No?      666666

Flow Cal. Date:      05/16/2018  
Slope      573  
Intercept      140460

IR Calibration Date:      06/05/2018  
                                    3um                      9um

-----  
0th Coef(\*100):      -15117              -26541  
1st Coef(\*100):      266971              144348  
2nd Coef(\*100):      2581                959  
H2O adj(mg/l\*10k):      293                383

\*\*\*\*\* Printer Test End \*\*\*\*\*

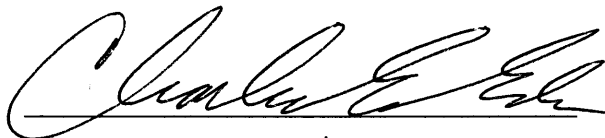

Operator Signature  
CHARLES EDER

Remarks:      PRINT TEST

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-005357  
Location = TOXL      8164.14.00 09/16  
06/10/2021      12:27

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	12:27
02 Std. Sol.	0.028	12:28
03 Room Air	0.000	12:29
04 Std. Sol.	0.029	12:29
05 Room Air	0.000	12:30
06 Std. Sol.	0.028	12:31
07 Room Air	0.000	12:31

08 Sim Temp = 34.0°C

Simul Ser No = MP5289  
Std Sol No = 201911E  
County = 08      Oper No. = 666666

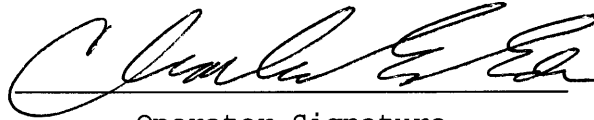

Operator Signature  
CHARLES EDER

Remarks:

Low AC  
0.03022

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-005357  
Location = TOXL      8164.14.00 09/16  
06/10/2021      12:34

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	12:35
02 Std. Sol.	0.398	12:36
03 Room Air	0.000	12:36
04 Std. Sol.	0.399	12:37
05 Room Air	0.000	12:38
06 Std. Sol.	0.399	12:38
07 Room Air	0.000	12:39

08 Sim Temp = 34.0°C

Simul Ser No = MP3062  
Std Sol No = 202103E  
County = 08      Oper No. = 666666



Operator Signature  
CHARLES EDER

Remarks:

HIGH AC  
0.400 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-005357  
Location = TOXL      8164.14.00 09/16  
06/10/2021      12:41

Test	AC	Time
01 Room Air	0.000	12:41
02 *Subject Test	INT*	12:42
03 Room Air	0.000	12:42

\*Invalid Test  
Interferent Detected

Sub Name = TEST, DONOR2 NONE  
Sub DOB = 07/25/1998  
Sub Sex = Male      Weight = NA  
Test = DUI      Cit = INTERFERENT  
Dr. Lic. = ND/TES989643  
Lot No = 24119080A1  
Cyl No = 31  
Expiration Date = 11/05/2021  
County = 08      Oper No. = 666666



Operator Signature  
CHARLES EDER

Remarks:

*Interferent Check*  
*0.10AC + 0.05% ACETONE*

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer                      Alcohol Analyzer  
North Dakota Model 8000                      SN 80-005357  
Location = TOXL                              8164.14.00 09/16  
06/10/2021    12:43

Test	AC	Time
01 Diagnostic	OK	12:44
02 Room Air	0.000	12:44
03 *Subject Test	RFI*	12:44
04 Room Air	0.000	12:45

\*Invalid Test  
Inhibited - RFI

Sub Name = TEST, DONOR2 NONE  
Sub DOB = 07/25/1998  
Sub Sex = Male                                      Weight = NA  
Test = DUI    Cit = RFI CHECK  
Dr. Lic. = ND/TES989643  
Lot No = 24119080A1  
Cyl No = 31  
Expiration Date = 11/05/2021  
County = 08    Oper No. = 666666

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

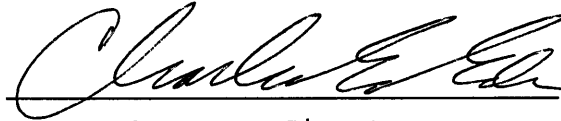

Operator Signature  
CHARLES EDER

Remarks:

RFI CHECK

Form 106-I8000


Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-005357  
Location = TOXL      8164.14.00 09/16  
06/10/2021      12:45

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	12:46
02 Std. Gas	0.082	12:46
03 Room Air	0.000	12:46
04 Std. Gas	0.082	12:47
05 Room Air	0.000	12:47
06 Std. Gas	0.083	12:48
07 Room Air	0.000	12:48

Lot No = 24119080A1  
Cyl No = 31  
Exp Date = 11/05/2021  
County = 08      Oper No. = 666666

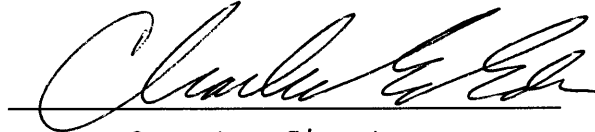

Operator Signature  
CHARLES EDER

Remarks:

CALIBRATION CHECK  
0.080 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-005357  
Location = TOXL      8164.14.00 09/16  
06/10/2021      12:49

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	12:49
02 Std. Gas	0.082	12:49
03 Room Air	0.000	12:50
04 Std. Gas	0.082	12:50
05 Room Air	0.000	12:51
06 Std. Gas	0.083	12:51
07 Room Air	0.000	12:52

Lot No = 24119080A1  
Cyl No = 31  
Exp Date = 11/05/2021  
County = 08      Oper No. = 666666



Operator Signature  
CHARLES EDER

Remarks: CALIBRATION CHECK  
0.080 AC

Form 106-I8000



Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-005357  
Location = TOXL      8164.14.00 09/16  
06/10/2021      12:52

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	12:53
02 Std. Gas	0.082	12:53
03 Room Air	0.000	12:54
04 Std. Gas	0.082	12:54
05 Room Air	0.000	12:54
06 Std. Gas	0.082	12:55
07 Room Air	0.000	12:55

Lot No = 24119080A1  
Cyl No = 31  
Exp Date = 11/05/2021  
County = 08      Oper No. = 666666



Operator Signature  
CHARLES EDER

Remarks: CALIBRATION CHECK  
0.080 AC

Form 106-I8000