Toxicology Section/Breath Alcohol Program Intoxilyzer® 8000 Inspection

# **INTOXILYZER® 8000 INSPECTION**

Intoxi	zer® 8000 Serial Number: 80 -006502 Location: ToxC
Α.	General Setup and Checks:  1.
В.	Tests (Sign and attach test records):  Configure simulator for the following tests (Level 1,S).  Wet Calibration Check  a.
C.	Remarks/Maintenance: AUNUAL INSPECTION
· · · · · ·	
	Analyst Signature  Lace Date  Lac
Revie	

02/18 Issuing Authority - CEE

Alcohol Analyzer CMI, Inc. Intoxilyzer North Dakota Model 8000 SN 80-006502 8164.14.00 09/16 Location = TOXL 10:55 06/17/2021

\*\*\*\*\*\*\* Printer Test \*\*\*\*\*\*\*\*

abcdefghijklmnopqrstuvwxyz1234567890-= ABCDEFGHIJKLMNOPQRSTUVWXYZ!@#\$%^&\*() +?

abcdefghijklmnopgrstuvwxyz1234567890-= ABCDEFGHIJKLMNOPQRSTUVWXYZ!@#\$%^&\*() +?

Current Instrument Setup

Data Entry Mode: Enabled Start Test Sequence: DABACABA Display Prelim Rslt? Yes Display Third Digit? Yes Inhib Printer(Y/N)? No Display Volume? No Disable On Memfull? Yes # of Print Copies? Select Std (D/W/I)? Dry 0.080 Standard Value? Standard Lot #? 05620080A1 Standard Cyl #? 04/05/2022 Standard Expiration? Oper No? 666666

08/19/2015 Flow Cal. Date: Slope 674 -654491

Intercept

Remarks:

IR Calibration Date: 08/19/2015

Oth Coef(\*100): -24446 1st Coef(\*100): 267787 2nd Coef(\*100): 3387 -18103 135417 1610 H2O adj(mg/l\*10k): 740 513

3um

\*\*\*\*\*\* Printer Test End \*\*\*\*\*\*\*\*

Operator Signature CHARLES EDER

CMI, Inc. Intoxilyzer
North Dakota Model 8000
Location = TOXL
06/17/2021

Alcohol Analyzer SN 80-006502 8164.14.00 09/16 10:58

## WET CAL CHECK

Test		AC	Time
01 Room	Air	0.000	10:58
02 Std.	Sol.	0.019	10:59
03 Room	Air	0.000	10:59
04 Std.	Sol.	0.019	11:00
05 Room	Air	0.000	11:01
06 Std.	Sol.	0.018	11:01
07 Room	Air	0.000	11:02

 $08 \text{ Sim Temp} = 34.0^{\circ}\text{C}$ 

Simul Ser No = MP3064 Std Sol No = 20070

County = 08

Oper No. = 666666

Operator Signature CHARLES EDER

Remarks:

Form 106-I8000

LOW AC

0.020AC

CMI, Inc. Intoxilyzer
North Dakota Model 8000
Location = TOXL
06/17/2021

Alcohol Analyzer SN 80-006502 8164.14.00 09/16 11:04

#### WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:04
02 Std. Sol.	0.300	11:05
03 Room Air	0.000	11:05
04 Std. Sol.	0.299	11:06
05 Room Air	0.000	11:07
06 Std. Sol.	0.298	11:07
07 Room Air	0.000	11:08

 $08 \text{ Sim Temp} = 34.0^{\circ}\text{C}$ 

Simul Ser No = MP5317 Std Sol No = 202012A

County = 08

Oper No. = 666666

Operator Signature CHARLES EDER

Remarks:

Form 106-I8000

0.300AC

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006502
Location = TOXL 8164.14.00 09/16
06/17/2021 11:12

Test	AC	Time
01 Room Air	0.000	11:13
02 *Subject Test	INT*	11:13
03 Room Air	0.000	11:14

<sup>\*</sup>Invalid Test

Interferent Detected

Sub Name = DISCOVER, THE SPIRIT

Sub DOB = 02/01/1992

Sub Sex = Female Weight = 150
Test = DUI Cit = INTERFERENT

Dr. Lic. = ND/DIS921456

Lot No = 24119080A1

Cyl No = 2

Expiration Date = 11/05/2021

County = 08 Oper No. = 666666

Operator Signature CHARLES EDER

s: Interperant the

0.10 AC + 0.05% ACETONE

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006502
Location = TOXL 8164.14.00 09/16
06/17/2021 11:14

Test	AC	Time
01 Diagnostic	OK	11:15
02 Room Air	0.000	11:16
03 *Subject Test	RFI*	11:16
04 Room Air	0.000	11:16

\*Invalid Test Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT

Sub DOB = 02/01/1992

Sub Sex = Female Weight = 150
Test = DUI Cit = RFI CHECK

Dr. Lic. = ND/DIS921456

Lot No = 24119080A1

Cyl No = 2

Expiration Date = 11/05/2021

County = 08 Oper No. = 666666

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

Operator Signature

CHARLES EDER

CMI, Inc. Intoxilyzer
North Dakota Model 8000
Location = TOXL
06/17/2021

SN 80-006502 8164.14.00 09/16 11:17

#### DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:18
02 Std. Gas	0.081	11:18
03 Room Air	0.000	11:19
04 Std. Gas	0.082	11:19
05 Room Air	0.000	11:20
06 Std. Gas	0.081	11:20
07 Room Air	0.000	11:20

Lot No = 24119080A1

Cyl No = 2

Exp Date = 11/05/2021

County = 08

Oper No. = 666666

ALIBRATION CHECK

Operator Signature CHARLES EDER

Remarks:

CMI, Inc. Intoxilyzer
North Dakota Model 8000
Location = TOXL
06/17/2021

Alcohol Analyzer SN 80-006502 8164.14.00 09/16 11:21

### DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:21
02 Std. Gas	0.082	11:22
03 Room Air	0.000	11:22
04 Std. Gas	0.082	11:22
05 Room Air	0.000	11:23
06 Std. Gas	0.082	11:23
07 Room Air	0.000	11:24

Lot No = 24119080A1

Cyl No = 2

Exp Date = 11/05/2021

County = 08

Oper No. = 666666

Operator Signature CHARLES EDER

ALIBRATION CHECK

0.080AC

Remarks:

CMI, Inc. Intoxilyzer
North Dakota Model 8000
Location = TOXL
06/17/2021

Alcohol Analyzer SN 80-006502 8164.14.00 09/16 11:24

## DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:25
02 Std. Gas	0.081	11:25
03 Room Air	0.000	11:25
04 Std. Gas	0.082	11:26
05 Room Air	0.000	11:26
06 Std. Gas	0.082	11:27
07 Room Air	0.000	11:27

Lot No = 24119080A1

Cyl No = 2

Exp Date = 11/05/2021

County = 08

Oper No. = 666666

Operator Signature CHARLES EDER

Remarks:

Form 106-I8000

0.080 AC