

INTOXILYZER® 8000 INITIAL INSPECTION

Intoxilyzer® 8000 Serial Number: 80-003072

A. GENERAL Setup and Checks:

1. Power On
 - 5 min. countdown during initial warm-up
 - 1 min. countdown to begin test after standby mode
2. Display Intensity (Ctrl + ↑ or ↓)
3. Password Security:
 - Level 1 (Operator) Level 2 (Field Inspector) Level 3 (Lab Personnel)
4. Printer Setup (Level 3,S,P)
5. Forms Load (CMI, Inc., CoBrA Software)

B. LEVEL 1 Setup and Checks:

1. Internal Printer Test and Paper Feed (Level 1,P). Attach record.
2. External Printer Test (Level 1,P). Attach record.
3. Barcode Reader (Level 1,B):
 - Operator ID ND License R key: Refusal. Attach record.

C. LEVEL 2 Setup and Checks:

1. Continuous Air Blank (Level 2,A)
2. Diagnostic (Level 2,D). Attach record.
3. Prelim Data Entry (Level 2,E)
4. Ser Number/Version Info (Level 2,V):
 - Inst Ser#: 80-003072 DSP Ver: 1362.14
 - Core Ver: 8164.14 CPLD Ver: 7

D. LEVEL 3 Setup and Checks:

1. DVM Monitor (Level 3,D,D): 3: 12491 s: 0 c: 47.00
9: 13096 s: 1 B: 45 0
(RFI indicator with keyed radio): 1
2. Function (F) Keys: : F5 – Tone F9 - Fan
 F6 – Green LED F10 – Pump
 F7 – Red LED F11 – Solenoid
3. Atmospheric Monitor (Level 3,D,A): Atmo Sensor, P: 957 CF: 1.05
4. Flow Monitor (Level 3,D,F): Flow Sensor, R: 124 F: 0.000 L/s

5. Temperature Monitor (Level 3,D,T): C: 47.0 A: 49.5 I: 0.0
B: 45.0 V: 0.0
6. Tank Monitor (Level 3,D,G): P (display): 178 psi P (gauge): 200 psi
7. General Setup (Level 3,S,G)
8. Configure Start Test (Level 3,S,T)
9. Network (Level 3,S,N)

E. TESTS (attach all test records):

Configure wet bath simulator standards before running tests (Level 1,S):

1. Zero AC (Level 1,C):
Sim ser #: MP5320 Lot #: MILL Q H₂O Exp date: NA
Known value: 0.000 AC
Results: 0.000 AC 0.000 AC 0.000 AC
2. Low AC (Level 1,C):
Sim ser #: MP5289 Lot #: 201911E Exp date: 11.19.21
Known value: 0.030 AC
Results: 0.028 AC 0.029 AC 0.029 AC
3. High AC (Level 1,C):
Sim ser #: MP3061 Lot #: 20101 Exp date: 2.13.22
Known value: 0.500 AC
Results: 0.501 AC 0.500 AC 0.500 AC

Configure dry gas standard before running tests (Level 1,S):

4. Ethanol + 0.05% Acetone (Level 1,B):
Sim ser #: DR5114 Lot #: ICS 6 Exp date: NA
Known value: 0.10 AC + 0.05% ACETONE
 Display reads "Interferent Detect"
5. Acetone (Level 1,B):
Sim ser #: DR5188 Lot #: ACE 1 Exp date: NA
Known value: 0.05% ACETONE
 Display reads "Interferent Detect"
6. Methanol (Level 1,B):
Sim ser #: DR5186 Lot #: MET 3 Exp date: NA
Known value: 0.10% METHANOL
 Display reads "Interferent Detect"
7. Isopropanol (Level 1,B):
Sim ser #: DR5189 Lot #: CEE IA IP A 2 Exp date: NA
Known value: 0.10% ISOPROPANOL
 Display reads "Interferent Detect"

- 8. Ambient Fail (Level 1,B): Display and test record read correct.
- 9. Invalid Sample (Level 1,B): Display and test record read correct.
- 10. Sequence Aborted (Level 1,B): Display and test record read correct.
- 11. ABA (Level 1,B): Test record reads correct.
- 12. Calibration Check Out of Tolerance (CMS mode): Display and test record read correct.
- 13. RFI Detected (B,C, or CMS mode): Display and test record read correct.
- 14. Tank Pressure Below Minimum (CMS mode): Display reads correct.
- 15. Deficient Samples (CMS mode): Display and test record read correct.
- 16. Improper Sample (CMS mode): Display and test record read correct.
- 17. Difference Too Great (CMS mode): Display and test record read correct.
- 18. CMS (CMS mode): Test record reads correct.
- 19. Stability Test (Level 3,T,S):

Ave: 0.0805 Std Dev: 0.0005 Rel Std Dev: 0.64%

Maintenance/Repairs/Notes: INITIAL INSPECTION DUE TO
OPENING UP INSTRUMENT TO REPLACE REAL TIME
CALC CLOCK BATTERY. TANK CAL SENSOR RESET.
REPLACED SIMULATOR RETURN O-RING.

Instrument is acceptable to be used in the field.

Charles E. Eh
Field Inspector's Signature

6/21/21
Date

Kali Y. Diets
Reviewed by

6-21-21
Date

Intoxilyzer Test Record and Checklist
NDDAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003072
Location = TOXL 8164.14.00 09/16
06/21/2021 13:43

***** Printer Test *****

12 char/line
abcdefghijkl
ABCDEFGHIJKL

16 char per line
abcdefghijklmnop
ABCDEFGHIJKLMNop

21 chars per line
abcdefghijklmnopqrstu
ABCDEFGHIJKLMNopqrstu

24 characters per line
abcdefghijklmnopqrstuwx
ABCDEFGHIJKLMNopqrstuwx

32 characters per line
abcdefghijklmnopqrstuwx123456
ABCDEFGHIJKLMNopqrstuwx123456

42 characters per line
abcdefghijklmnopqrstuwx1234567890abcdef
ABCDEFGHIJKLMNopqrstuwx1234567890abcdef


Current Instrument Setup
Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 34418080A2
Standard Cyl #? 5
Standard Expiration? 02/05/2021
Oper No? 666666

Flow Cal. Date: 03/09/2011
Slope 682
Intercept -620512

IR Calibration Date: 03/09/2011
3um 9um

0th Coef(*100): -24766 -17499
1st Coef(*100): 264559 138630
2nd Coef(*100): 1876 1114
H2O adj(mg/l*10k): 497 331

***** Test End *****

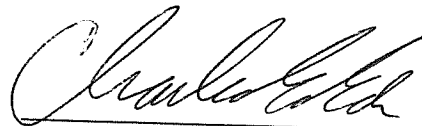

Operator Signature
CHARLES EDER

Remarks: INTERNAL

PRINT TEST

Form 106-18000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003072
Location = TOXL 8164.14.00 09/16
06/21/2021 13:44

***** Printer Test *****

abcdefghijklmnopqrstuvwxyz1234567890-|=|
ABCDEFGHIJKLMNopQRSTUVWXYZ!@#%^&*()_+?

abcdefghijklmnopqrstuvwxyz1234567890-|=|
ABCDEFGHIJKLMNopQRSTUVWXYZ!@#%^&*()_+?

Current Instrument Setup

Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 34418080A2
Standard Cyl #? 5
Standard Expiration? 02/05/2021
Oper No? 666666

Flow Cal. Date: 03/09/2011
Slope 682
Intercept -620512

IR Calibration Date: 03/09/2011
 3um 9um

0th Coef(*100): -24766 -17499
1st Coef(*100): 264559 138630
2nd Coef(*100): 1876 1114
H2O adj(mg/l*10k): 497 331

***** Printer Test End *****



Operator Signature
CHARLES EDER

Remarks:

EXTERNAL
PRINT TEST

Form 106-I8000


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003072
Location = TOXL 8164.14.00 09/16
06/21/2021 13:47

Test	AC	Time
01 Room Air	0.000	13:48
02 *Subject Test	REF*	13:48
03 Room Air	0.000	13:49

*Subject Test Refused

Sub Name = TEST, DONOR2 NONE
Sub DOB = 07/25/1998
Sub Sex = Male Weight = NA
Test = DUI Cit = R KEY REFUSAL
Dr. Lic. = ND/TES989643
Lot No = 34418080A2
Cyl No = 5
Expiration Date = 02/05/2021
County = 08 Oper No. = 666666

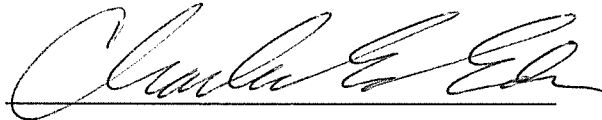

Operator Signature
CHARLES EDER

Remarks: "R" KEY REFUSAL

Form 106-I8000


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003072
Location = TOXL 8164.14.00 09/16
06/21/2021 13:53

DIAGNOSTIC

Voltage/Current Test	Pass
RAM Test	Pass
EEPROM Checksum Test	Pass
Real Time Clock Test	Pass
DSP Test	Pass
Analytical Stability Test	Pass
Modem Test	Pass
Temp Regulation Test	Pass

Oper No. = 666666

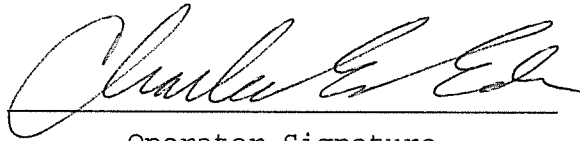

Operator Signature
CHARLES EDER

Remarks:

DIAGNOSTIC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003072
Location = TOXL 8164.14.00 09/16
06/21/2021 14:03

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:03
02 Std. Sol.	0.000	14:04
03 Room Air	0.000	14:05
04 Std. Sol.	0.000	14:05
05 Room Air	0.000	14:06
06 Std. Sol.	0.000	14:06
07 Room Air	0.000	14:07

08 Sim Temp = 34.0°C

Simul Ser No = MP5320
Std Sol No = MILLIQ H2O
County = 08 Oper No. = 666666

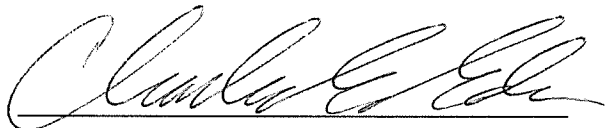

Operator Signature
CHARLES EDER

Remarks: ZERO AC
 0.000 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003072
Location = TOXL 8164.14.00 09/16
06/21/2021 14:11

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:11
02 Std. Sol.	0.028	14:12
03 Room Air	0.000	14:13
04 Std. Sol.	0.029	14:13
05 Room Air	0.000	14:14
06 Std. Sol.	0.029	14:15
07 Room Air	0.000	14:15

08 Sim Temp = 34.0°C

Simul Ser No = MP5289
Std Sol No = 201911E
County = 08 Oper No. = 666666

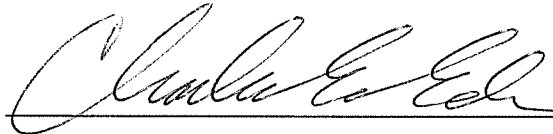

Operator Signature
CHARLES EDER

Remarks:

*Low AC
0.030 AC*

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003072
Location = TOXL 8164.14.00 09/16
06/21/2021 14:19

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:20
02 Std. Sol.	0.501	14:20
03 Room Air	0.000	14:21
04 Std. Sol.	0.500	14:22
05 Room Air	0.000	14:22
06 Std. Sol.	0.500	14:23
07 Room Air	0.000	14:23

08 Sim Temp = 34.0°C

Simul Ser No = MP3061
Std Sol No = 20101
County = 08 Oper No. = 666666

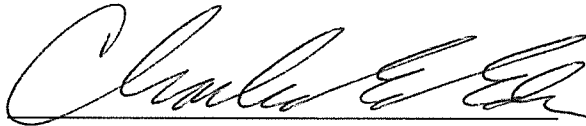

Operator Signature
CHARLES EDER

Remarks: HIGH AC
 0.500 AC

Form 106-I8000


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003072
Location = TOXL 8164.14.00 09/16
06/21/2021 14:47

Test	AC	Time
01 Room Air	0.000	14:48
02 *Subject Test	INT*	14:48
03 Room Air	0.000	14:49

*Invalid Test
Interferent Detected

Sub Name = TEST, DONOR2 NONE
Sub DOB = 07/25/1998
Sub Sex = Male Weight = NA
Test = DUI Cit = INTERFERENT
Dr. Lic. = ND/TES989643
Lot No = 24119080A1
Cyl No = 5
Expiration Date = 11/05/2021
County = 08 Oper No. = 666666

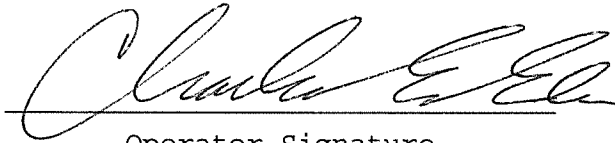

Operator Signature
CHARLES EDER

Remarks: *Interferent Check*
0.10 AC + 0.05% ACETONE

Form 106-I8000


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003072
Location = TOXL 8164.14.00 09/16
06/21/2021 14:50

Test	AC	Time
01 Room Air	0.000	14:51
02 *Subject Test	INT*	14:51
03 Room Air	0.000	14:52

*Invalid Test
Interferent Detected

Sub Name = TEST, DONOR2 NONE
Sub DOB = 07/25/1998
Sub Sex = Male Weight = NA
Test = DUI Cit = INTERFERENT
Dr. Lic. = ND/TES989643
Lot No = 24119080A1
Cyl No = 5
Expiration Date = 11/05/2021
County = 08 Oper No. = 666666

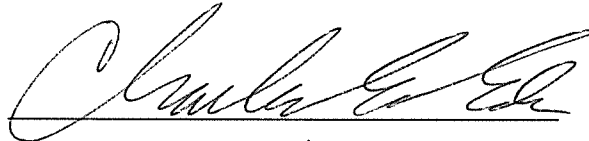

Operator Signature
CHARLES EDER

Remarks: *Interferent Check*
0.05% ACETONE

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003072
Location = TOXL 8164.14.00 09/16
06/21/2021 14:52

Test	AC	Time
01 Room Air	0.000	14:53
02 *Subject Test	INT*	14:53
03 Room Air	0.000	14:54

*Invalid Test
Interferent Detected

Sub Name = TEST, DONOR2 NONE
Sub DOB = 07/25/1998
Sub Sex = Male Weight = NA
Test = DUI Cit = INTERFERENT
Dr. Lic. = ND/TES989643
Lot No = 24119080A1
Cyl No = 5
Expiration Date = 11/05/2021
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: *Interferent Check*
0.10% METHANOL

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003072
Location = TOXL 8164.14.00 09/16
06/21/2021 14:55

Test	AC	Time
01 Room Air	0.000	14:56
02 *Subject Test	INT*	14:56
03 Room Air	0.000	14:57

*Invalid Test
Interferent Detected

Sub Name = TEST, DONOR2 NONE
Sub DOB = 07/25/1998
Sub Sex = Male Weight = NA
Test = DUI Cit = INTERFERENT
Dr. Lic. = ND/TES989643
Lot No = 24119080A1
Cyl No = 5
Expiration Date = 11/05/2021
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: *Interferent Check*
0.10% ISOPROPANOL
Form 106-I8000


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003072
Location = TOXL 8164.14.00 09/16
06/21/2021 14:59

Test	AC	Time
01 Room Air	AMB*	15:00
02 Room Air	0.000	15:00

*Invalid Test
Check Ambient Conditions

Sub Name = TEST, DONOR2 NONE
Sub DOB = 07/25/1998
Sub Sex = Male Weight = NA
Test = DUI Cit = AMBIENT FAIL
Dr. Lic. = ND/TES989643
Lot No = 24119080A1
Cyl No = 5
Expiration Date = 11/05/2021
County = 08 Oper No. = 666666

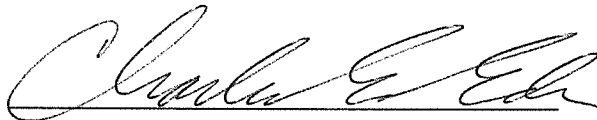

Operator Signature
CHARLES EDER

Remarks: *AMBIENT FAIL*

Form 106-I8000


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003072
Location = TOXL 8164.14.00 09/16
06/21/2021 15:01

Test	AC	Time
01 Room Air	0.000	15:01
02 Invalid Sample	X.XXX	15:02
03 Room Air	0.000	15:02

*Invalid Test - Mouth Alcohol

Sub Name = TEST, DONOR2 NONE
Sub DOB = 07/25/1998
Sub Sex = Male Weight = NA
Test = DUI Cit = INVALID SAMPLE
Dr. Lic. = ND/TES989643
Lot No = 24119080A1
Cyl No = 5
Expiration Date = 11/05/2021
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: *INVALID SAMPLE*

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003072
Location = TOXL 8164.14.00 09/16
06/21/2021 15:03

Test	AC	Time
01 Room Air	0.000	15:04
02 *Subject Test	ABT*	15:04
03 Room Air	0.000	15:05

*Sequence Aborted

Sub Name = TEST, DONOR2 NONE
Sub DOB = 07/25/1998
Sub Sex = Male Weight = NA
Test = DUI Cit = SEQUENCE ABORT
Dr. Lic. = ND/TES989643
Lot No = 24119080A1
Cyl No = 5
Expiration Date = 11/05/2021
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks:

SEQUENCE
ABORTED

Form 106-I8000


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003072
Location = TOXL 8164.14.00 09/16
06/21/2021 15:06

Test	AC	Time
01 Room Air	0.000	15:07
02 Subject Test 1	0.009	15:07
03 Room Air	0.000	15:08
04 Reported AC	0.009	15:07

No RFI Detected

Sub Name = TEST, DONOR2 NONE
Sub DOB = 07/25/1998
Sub Sex = Male Weight = NA
Test = DUI Cit = ABA MODE
Dr. Lic. = ND/TES989643
Lot No = 24119080A1
Cyl No = 5
Expiration Date = 11/05/2021
County = 08 Oper No. = 666666

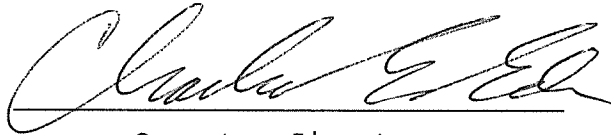

Operator Signature
CHARLES EDER

Remarks: *ABA MODE*
TEST

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003072
Location = TOXL 8164.14.00 09/16
06/21/2021 15:09

Test	AC	Time
01 Diagnostic	OK	15:10
02 Room Air	0.000	15:10
03 Subject Test 1	0.044	15:11
04 Room Air	0.000	15:13
05 Std. Gas	0.089*	15:14
06 Room Air	0.000	15:14

*Cal Check Out of Tolerance

Sub Name = TEST, DONOR2 NONE
Sub DOB = 07/25/1998
Sub Sex = Male Weight = NA
Test = DUI Cit = CAL CHK OUT TOL
Dr. Lic. = ND/TES989643
Lot No = 24119080A1
Cyl No = 5
Expiration Date = 11/05/2021
County = 08 Oper No. = 666666

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

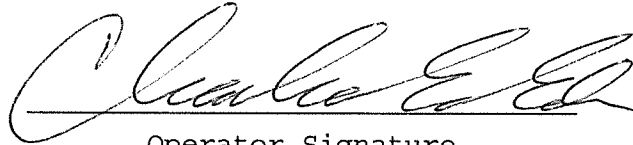

Operator Signature
CHARLES EDER

Remarks: CALIBRATION CHECK
OUT OF TOLERANCE

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003072
Location = TOXL 8164.14.00 09/16
06/21/2021 15:16

Test	AC	Time
01 Diagnostic	OK	15:17
02 Room Air	0.000	15:17
03 *Subject Test	RFI*	15:18
04 Room Air	0.000	15:18

*Invalid Test
Inhibited - RFI

Sub Name = TEST, DONOR2 NONE
Sub DOB = 07/25/1998
Sub Sex = Male Weight = NA
Test = DUI Cit = RFI CHECK
Dr. Lic. = ND/TES989643
Lot No = 24119080A1
Cyl No = 5
Expiration Date = 11/05/2021
County = 08 Oper No. = 666666

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

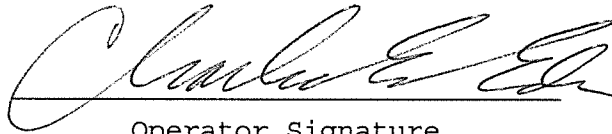

Operator Signature
CHARLES EDER

Remarks: RFI CHECK

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003072
Location = TOXL 8164.14.00 09/16
06/21/2021 15:18

Test	AC	Time
01 Diagnostic	OK	15:19
02 Room Air	0.000	15:20
03 *Subject Test	0.000*	15:23
04 Room Air	0.000	15:25
05 Std. Gas	0.080	15:26
06 Room Air	0.000	15:27
07 *Subject Test	0.000*	15:30
08 Room Air	0.000	15:31


No RFI Detected

*Deficient Sample - Value Printed was
Highest Obtained

Sub Name = TEST, DONOR2 NONE
Sub DOB = 07/25/1998
Sub Sex = Male Weight = NA
Test = DUI Cit = DEFICIENT SAMPL
Dr. Lic. = ND/TES989643
Lot No = 24119080A1
Cyl No = 5
Expiration Date = 11/05/2021
County = 08 Oper No. = 666666

20 minute waiting period ascertained? Y
Clean Mouthpiece used and disposed? Y
Clean Mouthpiece used and disposed? Y

I followed the Approved Method and the
instructions displayed by the Intoxilyzer
in conducting this test.

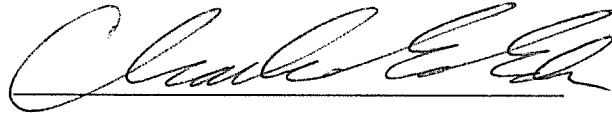

Operator Signature
CHARLES EDER

Remarks: DEFICIENT SAMPLES

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003072
Location = TOXL 8164.14.00 09/16
06/21/2021 15:32

Test	AC	Time
01 Diagnostic	OK	15:33
02 Room Air	0.000	15:34
03 Reference	IPS*	15:34
04 Room Air	0.000	15:34

*Improper Sample

Sub Name = TEST, DONOR2 NONE
Sub DOB = 07/25/1998
Sub Sex = Male Weight = NA
Test = DUI Cit = IMPROPER SAMPLE
Dr. Lic. = ND/TES989643
Lot No = 24119080A1
Cyl No = 5
Expiration Date = 11/05/2021
County = 08 Oper No. = 666666

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature
CHARLES EDER

Remarks: *IMPROPER SAMPLE*

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003072
Location = TOXL 8164.14.00 09/16
06/21/2021 15:51

Test	AC	Time
01 Diagnostic	OK	15:51
02 Room Air	0.000	15:52
03 Subject Test 1	0.046*	15:52
04 Room Air	0.000	15:54
05 Std. Gas	0.080	15:56
06 Room Air	0.000	15:57
07 Subject Test 2	0.186*	15:57
08 Room Air	0.000	15:58

*Difference Too Great

Sub Name = TEST, DONOR2 NONE
Sub DOB = 07/25/1998
Sub Sex = Male Weight = NA
Test = DUI Cit = DIFF TOO GREAT
Dr. Lic. = ND/TES989643
Lot No = 24119080A1
Cyl No = 5
Expiration Date = 11/05/2021
County = 08 Oper No. = 666666

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

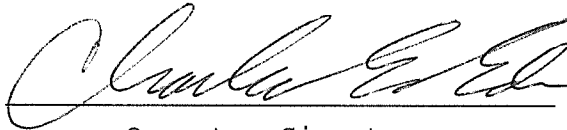

Operator Signature
CHARLES EDER

Remarks:

DIFFERENCE
TO GREAT

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003072
Location = TOXL 8164.14.00 09/16
06/21/2021 16:02

Test	AC	Time
01 Diagnostic	OK	16:03
02 Room Air	0.000	16:03
03 Subject Test 1	0.000	16:04
04 Room Air	0.000	16:06
05 Std. Gas	0.081	16:07
06 Room Air	0.000	16:08
07 Subject Test 2	0.000	16:08
08 Room Air	0.000	16:10
09 Reported AC	0.000	16:04

Difference OK
No RFI Detected

Sub Name = TEST, DONOR2 NONE
Sub DOB = 07/25/1998
Sub Sex = Male Weight = NA
Test = DUI Cit = CMS MODE TEST
Dr. Lic. = ND/TES989643
Lot No = 24119080A1
Cyl No = 5
Expiration Date = 11/05/2021
County = 08 Oper No. = 666666

20 minute waiting period ascertained? Y
Clean Mouthpiece used and disposed? Y
Clean Mouthpiece used and disposed? Y

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature
CHARLES EDER

Remarks: CMS MODE
TEST

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003072
Location = TOXL 8164.14.00 09/16
06/21/2021 16:11

Test	AC	Time
01 Room Air	0.000	16:11
02 Std. Gas	0.080	16:12
03 Room Air	0.000	16:12
04 Std. Gas	0.081	16:12
05 Room Air	0.000	16:13
06 Std. Gas	0.080	16:13
07 Room Air	0.000	16:14
08 Std. Gas	0.080	16:14
09 Room Air	0.000	16:15
10 Std. Gas	0.080	16:15
11 Room Air	0.000	16:15
12 Std. Gas	0.080	16:16
13 Room Air	0.000	16:16
14 Std. Gas	0.080	16:17
15 Room Air	0.000	16:17
16 Std. Gas	0.080	16:17
17 Room Air	0.000	16:18
18 Std. Gas	0.081	16:18
19 Room Air	0.000	16:19
20 Std. Gas	0.080	16:19
21 Room Air	0.000	16:20
22 Std. Gas	0.081	16:20
23 Room Air	0.000	16:21
24 Std. Gas	0.080	16:21
25 Room Air	0.000	16:21
26 Std. Gas	0.080	16:22
27 Room Air	0.000	16:22
28 Std. Gas	0.081	16:23
29 Room Air	0.000	16:23
30 Std. Gas	0.081	16:23
31 Room Air	0.000	16:24
32 Std. Gas	0.081	16:24
33 Room Air	0.000	16:25
34 Std. Gas	0.081	16:25
35 Room Air	0.000	16:26
36 Std. Gas	0.080	16:26
37 Room Air	0.000	16:26
38 Std. Gas	0.081	16:27
39 Room Air	0.000	16:27
40 Std. Gas	0.081	16:28
41 Room Air	0.000	16:28
42 Cal Check Stats:		
43 Average	0.0805	
44 Std Dev	0.0005	
45 Rel Std Dev	0.64%	



Operator Signature

CHARLES EDER

Remarks:

0.080 AC

STABILITY TEST

Form 106-I8000

Lot: 24119080 A1

CYL: 5

Exp: 11.5.2021