

INTOXILYZER® 8000 INITIAL INSPECTION

Intoxilyzer® 8000 Serial Number: 80-00 3067

A. GENERAL Setup and Checks:

- Power On
 - 5 min. countdown during initial warm-up
 - 1 min. countdown to begin test after standby mode
- Display Intensity (Ctrl + ↑ or ↓)
- Password Security:
 - Level 1 (Operator) Level 2 (Field Inspector) Level 3 (Lab Personnel)
- Printer Setup (Level 3,S,P)
- Forms Load (CMI, Inc., CoBrA Software)

B. LEVEL 1 Setup and Checks:

- Internal Printer Test and Paper Feed (Level 1,P). Attach record.
- External Printer Test (Level 1,P). Attach record.
- Barcode Reader (Level 1,B):
 - Operator ID ND License R key: Refusal. Attach record.

C. LEVEL 2 Setup and Checks:

- Continuous Air Blank (Level 2,A)
- Diagnostic (Level 2,D). Attach record.
- Prelim Data Entry (Level 2,E)
- Ser Number/Version Info (Level 2,V):
 - Inst Ser#: 80-003067 DSP Ver: 1362.14
 - Core Ver: 8164.14 CPLD Ver: 7

D. LEVEL 3 Setup and Checks:

- DVM Monitor (Level 3,D,D): 3: 12922 s: 1 c: 47.00
9: 12928 s: 1 B: 45 0
(RFI indicator with keyed radio): 1
- Function (F) Keys: : F5 – Tone F9 - Fan
 F6 – Green LED F10 – Pump
 F7 – Red LED F11 – Solenoid

- Atmospheric Monitor (Level 3,D,A): Atmo Sensor, P: 951 CF: 1.06
- Flow Monitor (Level 3,D,F): Flow Sensor, R: 105 F: 0.000 L/s

5. Temperature Monitor (Level 3,D,T): C: 47.0 A: 49.0 I: 0.0
B: 45.0 V: 0.0
6. Tank Monitor (Level 3,D,G): P (display): 176 psi P (gauge): 175 psi
7. General Setup (Level 3,S,G)
8. Configure Start Test (Level 3,S,T)
9. Network (Level 3,S,N)

E. TESTS (attach all test records):

Configure wet bath simulator standards before running tests (Level 1,S):

1. Zero AC (Level 1,C):
Sim ser #: MP5320 Lot #: MILLI Q H₂O Exp date: NA
Known value: 0.000 AC
Results: 0.000 AC 0.000 AC 0.000 AC
2. Low AC (Level 1,C):
Sim ser #: MP5289 Lot #: 201911E Exp date: 11.19.21
Known value: 0.030 AC
Results: 0.028 AC 0.028 AC 0.028 AC
3. High AC (Level 1,C):
Sim ser #: MP3061 Lot #: 20101 Exp date: 2.13.22
Known value: 0.500 AC
Results: 0.489 AC 0.491 AC 0.491 AC

Configure dry gas standard before running tests (Level 1,S):

4. Ethanol + 0.05% Acetone (Level 1,B):
Sim ser #: DR5114 Lot #: ICS 6 Exp date: NA
Known value: 0.10 AC + 0.05% ACETONE
 Display reads "Interferent Detect"
5. Acetone (Level 1,B):
Sim ser #: DR5188 Lot #: ACE 1 Exp date: NA
Known value: 0.05% ACETONE
 Display reads "Interferent Detect"
6. Methanol (Level 1,B):
Sim ser #: DR5186 Lot #: MET 3 Exp date: NA
Known value: 0.10% METHANOL
 Display reads "Interferent Detect"
7. Isopropanol (Level 1,B):
Sim ser #: DR5189 Lot #: IPA 2 Exp date: NA
Known value: 0.10% ISOPROPANOL
 Display reads "Interferent Detect"

- 8. Ambient Fail (Level 1,B): Display and test record read correct.
- 9. Invalid Sample (Level 1,B): Display and test record read correct.
- 10. Sequence Aborted (Level 1,B): Display and test record read correct.
- 11. ABA (Level 1,B): Test record reads correct.
- 12. Calibration Check Out of Tolerance (CMS mode): Display and test record read correct.
- 13. RFI Detected (B,C, or CMS mode): Display and test record read correct.
- 14. Tank Pressure Below Minimum (CMS mode): Display reads correct.
- 15. Deficient Samples (CMS mode): Display and test record read correct.
- 16. Improper Sample (CMS mode): Display and test record read correct.
- 17. Difference Too Great (CMS mode): Display and test record read correct.
- 18. CMS (CMS mode): Test record reads correct.

19. Stability Test (Level 3,T,S):

Ave: 0.0787 Std Dev: 0.0006 Rel Std Dev: 0.73%

Maintenance/Repairs/Notes: INITIAL INSPECTION DUE TO OPENING
UP INSTRUMENT TO REPLACE REAL TIME CLOCK
BATTERY.


Instrument is acceptable to be used in the field.



Field Inspector's Signature

6/22/21

Date

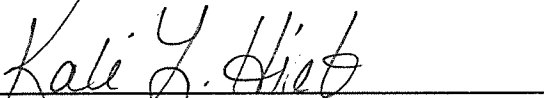

Reviewed by

6-22-21

Date

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003067
Location = TOXL 8164.14.00 09/16
06/22/2021 08:51

***** Printer Test *****

12 char/line
abcdefghijkl
ABCDEFGHIJKL

16 char per line
abcdefghijklmnop
ABCDEFGHIJKLMNPO

21 chars per line
abcdefghijklmnopqrstu
ABCDEFGHIJKLMNOPQRSTU

24 characters per line
abcdefghijklmnopqrstuwx
ABCDEFGHIJKLMNOPQRSTUWX

32 characters per line
abcdefghijklmnopqrstuwx123456
ABCDEFGHIJKLMNOPQRSTUWXYZ123456

42 characters per line
abcdefghijklmnopqrstuwx1234567890abcdef
ABCDEFGHIJKLMNOPQRSTUWXYZ1234567890ABCDEF


Current Instrument Setup
Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On MemFull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 34418080A2
Standard Cyl #? 5
Standard Expiration? 02/05/2021
Oper No? 666666

Flow Cal. Date: 04/17/2020
Slope 609
Intercept -236750

IR Calibration Date: 04/17/2020
3um 9um

0th Coef(*100):	-19307	-18350
1st Coef(*100):	282277	142540
2nd Coef(*100):	1257	744
H2O adj(mg/l)*10K):	302	214

***** Test End *****

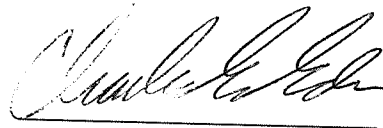

Operator Signature
CHARLES EDER

Remarks: *INTERNAL*
PRINT TEST

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003067
Location = TOXL 8164.14.00 09/16
06/22/2021 08:57

***** Printer Test *****

abcdefghijklmnopqrstuvwxyz1234567890-|=|
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&^*()_+?

abcdefghijklmnopqrstuvwxyz1234567890-|=|
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&^*()_+?

Current Instrument Setup


Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 34418080A2
Standard Cyl #? 5
Standard Expiration? 02/05/2021
Oper No? 666666

Flow Cal. Date: 04/17/2020
Slope 609
Intercept -236750

IR Calibration Date: 04/17/2020
 3um 9um

0th Coef(*100): -19307 -18350
1st Coef(*100): 282277 142540
2nd Coef(*100): 1257 744
H2O adj(mg/l*10k): 302 214

***** Printer Test End *****

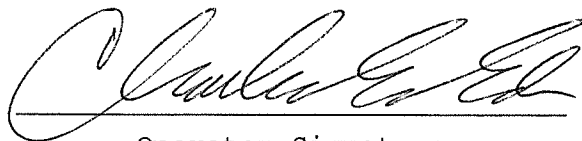

Operator Signature
CHARLES EDER

Remarks:

EXTERNAL
PRINT TEST

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003067
Location = TOXL 8164.14.00 09/16
06/22/2021 09:01

DIAGNOSTIC

Voltage/Current Test	Pass
RAM Test	Pass
EEPROM Checksum Test	Pass
Real Time Clock Test	Pass
DSP Test	Pass
Analytical Stability Test	Pass
Modem Test	Pass
Temp Regulation Test	Pass

Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks:

DIAGNOSTIC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003067
Location = TOXL 8164.14.00 09/16
06/22/2021 09:09

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	09:10
02 Std. Sol.	0.000	09:11
03 Room Air	0.000	09:11
04 Std. Sol.	0.000	09:12
05 Room Air	0.000	09:12
06 Std. Sol.	0.000	09:13
07 Room Air	0.000	09:13

08 Sim Temp = 34.0°C

Simul Ser No = MP5320
Std Sol No = MILLIQ H2O
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: ZERO AC
 0.000 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003067
Location = TOXL 8164.14.00 09/16
06/22/2021 09:16

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	09:17
02 Std. Sol.	0.028	09:17
03 Room Air	0.000	09:18
04 Std. Sol.	0.028	09:19
05 Room Air	0.000	09:19
06 Std. Sol.	0.028	09:20
07 Room Air	0.000	09:20

08 Sim Temp = 34.0°C

Simul Ser No = MP5289
Std Sol No = 201911E
County = 08 Oper No. = 666666

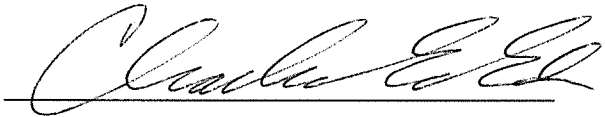

Operator Signature
CHARLES EDER

Remarks: *Low AC*
 0.030 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003067
Location = TOXL 8164.14.00 09/16
06/22/2021 09:22

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	09:22
02 Std. Sol.	0.489	09:23
03 Room Air	0.000	09:24
04 Std. Sol.	0.491	09:24
05 Room Air	0.000	09:25
06 Std. Sol.	0.491	09:26
07 Room Air	0.000	09:26

08 Sim Temp = 34.0°C

Simul Ser No = MP3061
Std Sol No = 20101
County = 08 Oper No. = 666666

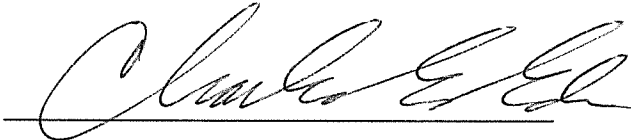

Operator Signature
CHARLES EDER

Remarks: HIGH AC
0.500 AC

Form 106-I8000


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003067
Location = TOXL 8164.14.00 09/16
06/22/2021 09:36

Test	AC	Time
01 Room Air	0.000	09:37
02 *Subject Test	INT*	09:37
03 Room Air	0.000	09:38

*Invalid Test
Interferent Detected

Sub Name = TEST, DONOR2 NONE
Sub DOB = 07/25/1998
Sub Sex = Male Weight = NA
Test = DUI Cit = INTERFERENT
Dr. Lic. = ND/TES989643
Lot No = 24119080A1
Cyl No = 5
Expiration Date = 11/05/2021
County = 08 Oper No. = 666666

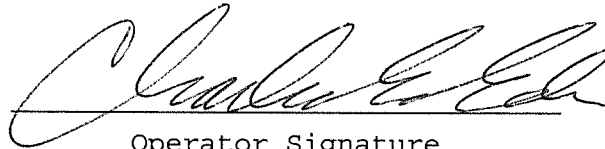

Operator Signature
CHARLES EDER

Remarks:

Interferent Check
0.10 AC + 0.05% ACETONE

Form 106-I8000


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003067
Location = TOXL 8164.14.00 09/16
06/22/2021 09:39

Test	AC	Time
01 Room Air	0.000	09:40
02 *Subject Test	INT*	09:41
03 Room Air	0.000	09:41

*Invalid Test
Interferent Detected

Sub Name = TEST, DONOR2 NONE
Sub DOB = 07/25/1998
Sub Sex = Male Weight = NA
Test = DUI Cit = INTERFERENT
Dr. Lic. = ND/TES989643
Lot No = 24119080A1
Cyl No = 5
Expiration Date = 11/05/2021
County = 08 Oper No. = 666666

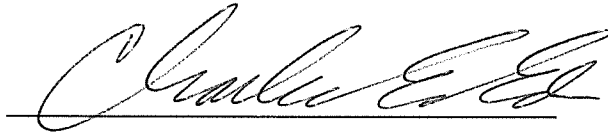

Operator Signature
CHARLES EDER

Remarks: *Interferent Check*
~~0.10 AC + 0.000 CEE~~

Form 106-I8000

0.05% ACETONE


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003067
Location = TOXL 8164.14.00 09/16
06/22/2021 09:43

Test	AC	Time
01 Room Air	0.000	09:44
02 *Subject Test	INT*	09:44
03 Room Air	0.000	09:45

*Invalid Test
Interferent Detected

Sub Name = TEST, DONOR2 NONE
Sub DOB = 07/25/1998
Sub Sex = Male Weight = NA
Test = DUI Cit = INTERFERENT
Dr. Lic. = ND/TES989643
Lot No = 24119080A1
Cyl No = 5
Expiration Date = 11/05/2021
County = 08 Oper No. = 666666

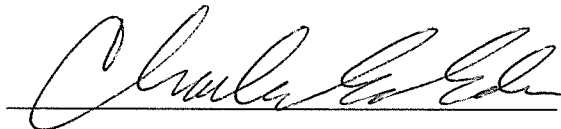

Operator Signature
CHARLES EDER

Remarks: *Interferent Check*
0.10% METHANOL
Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003067
Location = TOXL 8164.14.00 09/16
06/22/2021 09:45

Test	AC	Time
01 Room Air	0.000	09:46
02 *Subject Test	INT*	09:46
03 Room Air	0.000	09:47

*Invalid Test
Interferent Detected

Sub Name = TEST, DONOR2 NONE
Sub DOB = 07/25/1998
Sub Sex = Male Weight = NA
Test = DUI Cit = INTERFERENT
Dr. Lic. = ND/TES989643
Lot No = 24119080A1
Cyl No = 5
Expiration Date = 11/05/2021
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks:

Interferent Check
0.10% ISOPROPANOL

Form 106-I8000


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003067
Location = TOXL 8164.14.00 09/16
06/22/2021 09:48

Test	AC	Time
01 Room Air	AMB*	09:49
02 Room Air	0.000	09:50

*Invalid Test
Check Ambient Conditions

Sub Name = TEST, DONOR2 NONE
Sub DOB = 07/25/1998
Sub Sex = Male Weight = NA
Test = DUI Cit = AMBIENT FAIL
Dr. Lic. = ND/TES989643
Lot No = 24119080A1
Cyl No = 5
Expiration Date = 11/05/2021
County = 08 Oper No. = 666666

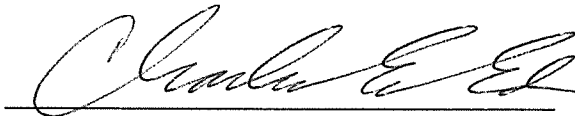

Operator Signature
CHARLES EDER

Remarks:

AMBIENT
FAIL

Form 106-I8000


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003067
Location = TOXL 8164.14.00 09/16
06/22/2021 09:50

Test	AC	Time
01 Room Air	0.000	09:51
02 Invalid Sample	X.XXX	09:51
03 Room Air	0.000	09:52

*Invalid Test - Mouth Alcohol

Sub Name = TEST, DONOR2 NONE
Sub DOB = 07/25/1998
Sub Sex = Male Weight = NA
Test = DUI Cit = INVALID SAMPLE
Dr. Lic. = ND/TES989643
Lot No = 24119080A1
Cyl No = 5
Expiration Date = 11/05/2021
County = 08 Oper No. = 666666

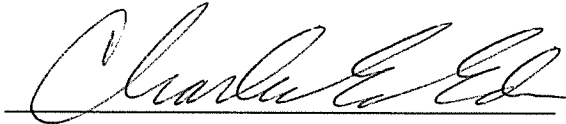

Operator Signature
CHARLES EDER

Remarks:

INVALID
SAMPLE

Form 106-I8000


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003067
Location = TOXL 8164.14.00 09/16
06/22/2021 09:52

Test	AC	Time
01 Room Air	0.000	09:53
02 *Subject Test	ABT*	09:53
03 Room Air	0.000	09:54

*Sequence Aborted

Sub Name = TEST, DONOR2 NONE
Sub DOB = 07/25/1998
Sub Sex = Male Weight = NA
Test = DUI Cit = SEQUENCE ABORT
Dr. Lic. = ND/TES989643
Lot No = 24119080A1
Cyl No = 5
Expiration Date = 11/05/2021
County = 08 Oper No. = 666666

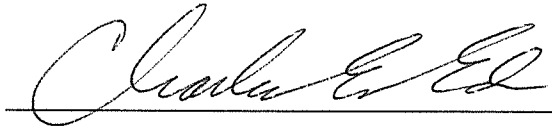

Operator Signature
CHARLES EDER

Remarks: SEQUENCE
ABORTED

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003067
Location = TOXL 8164.14.00 09/16
06/22/2021 09:59

Test	AC	Time
01 Diagnostic	OK	10:00
02 Room Air	0.000	10:00
03 Subject Test 1	0.187	10:01
04 Room Air	0.000	10:03
05 Std. Gas	0.087*	10:04
06 Room Air	0.000	10:05

*Cal Check Out of Tolerance

Sub Name = TEST, DONOR2 NONE
Sub DOB = 07/25/1998
Sub Sex = Male Weight = NA
Test = DUI Cit = CAL CHK OUT TOL
Dr. Lic. = ND/TES989643
Lot No = 24119080A1
Cyl No = 5
Expiration Date = 11/05/2021
County = 08 Oper No. = 666666

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature
CHARLES EDER

Remarks: CALIBRATION CHECK
OUT OF TOLERANCE

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003067
Location = TOXL 8164.14.00 09/16
06/22/2021 10:08

Test	AC	Time
01 Diagnostic	OK	10:09
02 Room Air	0.000	10:09
03 Subject Test 1	0.000	10:10
04 Room Air	0.000	10:12
05 Std. Gas	0.078	10:13
06 Room Air	0.000	10:14
07 *Subject Test	RFI*	10:14
08 Room Air	0.000	10:15

*Invalid Test
Inhibited - RFI

Sub Name = TEST, DONOR2 NONE
Sub DOB = 07/25/1998
Sub Sex = Male Weight = NA
Test = DUI Cit = RFI CHECK
Dr. Lic. = ND/TES989643
Lot No = 24119080A1
Cyl No = 5
Expiration Date = 11/05/2021
County = 08 Oper No. = 666666

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature
CHARLES EDER

Remarks: RFI CHECK

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003067
Location = TOXL 8164.14.00 09/16
06/22/2021 10:33

Test	AC	Time
01 Diagnostic	OK	10:33
02 Room Air	0.000	10:34
03 Reference	IPS*	10:34
04 Room Air	0.000	10:35

*Improper Sample

Sub Name = TEST, DONOR2 NONE
Sub DOB = 07/25/1998
Sub Sex = Male Weight = NA
Test = DUI Cit = IMPROPER SAMPLE
Dr. Lic. = ND/TES989643
Lot No = 24119080A1
Cyl No = 5
Expiration Date = 11/05/2021
County = 08 Oper No. = 666666

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature
CHARLES EDER

Remarks:

*IMPROPER
SAMPLE*

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003067
Location = TOXL 8164.14.00 09/16
06/22/2021 10:35

Test	AC	Time
01 Diagnostic	OK	10:36
02 Room Air	0.000	10:36
03 Subject Test 1	0.046*	10:36
04 Room Air	0.000	10:39
05 Std. Gas	0.078	10:40
06 Room Air	0.000	10:41
07 Subject Test 2	0.188*	10:42
08 Room Air	0.000	10:43

*Difference Too Great

Sub Name = TEST, DONOR2 NONE
Sub DOB = 07/25/1998
Sub Sex = Male Weight = NA
Test = DUI Cit = DIFF TOO GREAT
Dr. Lic. = ND/TES989643
Lot No = 24119080A1
Cyl No = 5
Expiration Date = 11/05/2021
County = 08 Oper No. = 666666

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature
CHARLES EDER

Remarks: DIFFERENCE TOO
GREAT
CEE

Form 106-18000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003067
Location = TOXL 8164.14.00 09/16
06/22/2021 10:44

Test	AC	Time
01 Diagnostic	OK	10:45
02 Room Air	0.000	10:46
03 Subject Test 1	0.189	10:46
04 Room Air	0.000	10:49
05 Std. Gas	0.079	10:50
06 Room Air	0.000	10:51
07 Subject Test 2	0.188	10:52
08 Room Air	0.000	10:53
09 Reported AC	0.188	10:52

Difference OK
No RFI Detected

Sub Name = TEST, DONOR2 NONE
Sub DOB = 07/25/1998
Sub Sex = Male Weight = NA
Test = DUI Cit = CMS MODE TEST
Dr. Lic. = ND/TES989643
Lot No = 24119080A1
Cyl No = 5
Expiration Date = 11/05/2021
County = 08 Oper No. = 666666

20 minute waiting period ascertained? Y
Clean Mouthpiece used and disposed? Y
Clean Mouthpiece used and disposed? Y

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature
CHARLES EDER

Remarks:

CMS TEST

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003067
Location = TOXL 8164.14.00 09/16
06/22/2021 10:54

Test	AC	Time
01 Room Air	0.000	10:55
02 Std. Gas	0.078	10:55
03 Room Air	0.000	10:56
04 Std. Gas	0.079	10:56
05 Room Air	0.000	10:57
06 Std. Gas	0.078	10:57
07 Room Air	0.000	10:57
08 Std. Gas	0.078	10:58
09 Room Air	0.000	10:58
10 Std. Gas	0.079	10:59
11 Room Air	0.000	10:59
12 Std. Gas	0.079	11:00
13 Room Air	0.000	11:00
14 Std. Gas	0.079	11:00
15 Room Air	0.000	11:01
16 Std. Gas	0.079	11:01
17 Room Air	0.000	11:02
18 Std. Gas	0.078	11:02
19 Room Air	0.000	11:03
20 Std. Gas	0.079	11:03
21 Room Air	0.000	11:04
22 Std. Gas	0.079	11:04
23 Room Air	0.000	11:04
24 Std. Gas	0.078	11:05
25 Room Air	0.000	11:05
26 Std. Gas	0.079	11:06
27 Room Air	0.000	11:06
28 Std. Gas	0.079	11:07
29 Room Air	0.000	11:07
30 Std. Gas	0.079	11:07
31 Room Air	0.000	11:08
32 Std. Gas	0.078	11:08
33 Room Air	0.000	11:09
34 Std. Gas	0.079	11:09
35 Room Air	0.000	11:10
36 Std. Gas	0.079	11:10
37 Room Air	0.000	11:11
38 Std. Gas	0.080	11:11
39 Room Air	0.000	11:11
40 Std. Gas	0.078	11:12
41 Room Air	0.000	11:12
42 Cal Check Stats:		
43 Average	0.0787	
44 Std Dev	0.0006	
45 Rel Std Dev	0.73%	



Operator Signature
CHARLES EDER

Remarks: 0.080Ac

STABILITY TEST

Form 106-I8000

Lot: 24119080A1

CYL: 5

EXP: 11/5/2021