



# ETHANOL GAS STANDARD CYLINDER REPORT

NORTH DAKOTA OFFICE OF ATTORNEY GENERAL

CRIME LABORATORY DIVISION-TOXICOLOGY SECTION/BREATH ALCOHOL PROGRAM

SFN 59282 (08/2018)

Chemical Test Operator Name (Print) <i>Beaver, Michael</i>		
Location <i>Harvey PD</i>	Intoxilyzer® Serial Number <i>80-006500</i>	
Gas Lot Number <i>02621080A1</i>	Gas Cylinder Number <i>14</i>	Gas Expiration Date <i>03/05/2023</i>

Check When Done:

- 1. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 2. Perform an ACA Test (Level 1, Function C).  
Write in the ACA Test Results (Reported to 3 Digits, Ex. 0.081).
  - A. *0.079* AC
  - B. *0.079* AC
  - C. *0.078* AC
- 3. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 4. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 5. Send the Following to the Crime Laboratory:
  - A. Ethanol Gas Standard Cylinder Report (SFN59282, Form 105-G).
  - B. ACA Test Record.

Chemical Test Operator Signature <i>[Signature]</i>	Date <i>10-Jul-21</i>
Reviewed By (Crime Laboratory Use Only) <i>[Signature]</i>	Date <i>7.20.2021</i>

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-006500  
Location = HARV      8164.14.00 09/16  
07/10/2021      16:35

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	16:36
02 Std. Gas	0.079	16:36
03 Room Air	0.000	16:36
04 Std. Gas	0.079	16:37
05 Room Air	0.000	16:37
06 Std. Gas	0.078	16:38
07 Room Air	0.000	16:38

Lot No = 02621080A1  
Cyl No = 14  
Exp Date = 03/05/2023  
County = 52      Oper No. = 131495



Operator Signature  
MICHAEL BEAVER

Remarks:

OK

Form 106-I8000