



**ETHANOL GAS STANDARD CYLINDER REPORT**  
 NORTH DAKOTA OFFICE OF ATTORNEY GENERAL  
 CRIME LABORATORY DIVISION-TOXICOLOGY SECTION/BREATH ALCOHOL PROGRAM  
 SFN 59282 (08/2018)

Chemical Test Operator Name (Print) <i>Rick D Czapiewski</i>		
Location <i>FINL</i>	Intoxilyzer® Serial Number <i>80-005946</i>	
Gas Lot Number <i>05620080A1</i>	Gas Cylinder Number <i>040</i>	Gas Expiration Date <i>4/5/2022</i>

Check When Done:

- 1. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 2. Perform an ACA Test (Level 1, Function C). *RECEIVED ACA TEST (DRY CAL CHECK) CEE 11.3.21*
- CEE* Write in the ACA Test Results (Reported to 3 Digits, Ex. 0.081).
  - A. *0.082* AC
  - B. *0.080* AC
  - C. *0.080* AC
- 3. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 4. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 5. Send the Following to the Crime Laboratory: *RECEIVED BOTH RECORDS CEE 11.3.21*
  - CEE* A. Ethanol Gas Standard Cylinder Report (SFN59282, Form 105-G).
  - B. ACA Test Record.

Chemical Test Operator Signature <i>Rick D Czapiewski</i>	Date <i>11/3/2021</i>
Reviewed By (Crime Laboratory Use Only) <i>Charles E. Ehr</i>	Date <i>11/3/2021</i>

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-005946  
Location = FINL      8164.14.00 09/16  
11/02/2021      20:07

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	20:08
02 Std. Gas	0.082	20:08
03 Room Air	0.000	20:09
04 Std. Gas	0.080	20:09
05 Room Air	0.000	20:10
06 Std. Gas	0.080	20:10
07 Room Air	0.000	20:11

Lot No = 05620080A1  
Cyl No = 40  
Exp Date = 04/05/2022  
County = 46      Oper No. = 130363



Operator Signature  
RICK CZAPIEWSKI

Remarks:

*ACT Test  
New Gas Cylinders*

Form 106-I8000