



ETHANOL GAS STANDARD CYLINDER REPORT
 NORTH DAKOTA OFFICE OF ATTORNEY GENERAL
 CRIME LABORATORY DIVISION-TOXICOLOGY SECTION/BREATH ALCOHOL PROGRAM
 SFN 59282 (08/2018)

Chemical Test Operator Name (Print) <i>Scott Austin</i>		
Location <i>NS</i>	Intoxilyzer® Serial Number <i>80-004940</i>	
Gas Lot Number <i>02621080A1</i>	Gas Cylinder Number <i>1</i>	Gas Expiration Date <i>03/05/2023</i>

Check When Done:

- 1. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 2. Perform an ACA Test (Level 1, Function C).
 Write in the ACA Test Results (Reported to 3 Digits, Ex. 0.081).
 A. *0.079* AC
 B. *0.080* AC
 C. *0.080* AC
- 3. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 4. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 5. Send the Following to the Crime Laboratory:
 A. Ethanol Gas Standard Cylinder Report (SFN59282, Form 105-G).
 B. ACA Test Record.

Chemical Test Operator Signature <i>Scott Austin</i>	Date <i>11-19-2021</i>
Reviewed By (Crime Laboratory Use Only) <i>Charles E. Edwards</i>	Date <i>11.30.21</i>

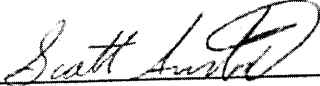
Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004940
Location = NS 8164.14.00 09/16
11/19/2021 02:54

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	02:56
02 Std. Gas	0.079	02:56
03 Room Air	0.000	02:57
04 Std. Gas	0.080	02:57
05 Room Air	0.000	02:57
06 Std. Gas	0.080	02:58
07 Room Air	0.000	02:58

Lot No = 02621080A1
Cyl No = 1
Exp Date = 03/05/2023
County = 30 Oper No. = 119901


Operator Signature
SCOTT AUSTIN

Remarks:

Form 106-I8000