



ETHANOL GAS STANDARD CYLINDER REPORT
 NORTH DAKOTA OFFICE OF ATTORNEY GENERAL
 CRIME LABORATORY DIVISION-TOXICOLOGY SECTION/BREATH ALCOHOL PROGRAM
 SFN 59282 (08/2018)

Chemical Test Operator Name (Print) <i>Darren D. Herdbieder</i>		
Location <i>Pierce County Jail</i>	Intoxilyzer® Serial Number <i>80-004953</i>	
Gas Lot Number <i>26021080 A1</i>	Gas Cylinder Number <i>008</i>	Gas Expiration Date <i>10/5/2023</i>

Check When Done:

- 1. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 2. Perform an ACA Test (Level 1, Function C).
 Write in the ACA Test Results (Reported to 3 Digits, Ex. 0.081).
 A. *0.083* AC
 B. *0.077* AC
 C. *0.082* AC
- 3. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 4. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 5. Send the Following to the Crime Laboratory:
 - A. Ethanol Gas Standard Cylinder Report (SFN59282, Form 105-G).
 - B. ACA Test Record.

*Sent to General N.D. gov
 3/10/22
 Dan Her*

Chemical Test Operator Signature <i>Darren Herdbieder / Darren Herdbieder</i>	Date <i>3/10/2022</i>
Reviewed By (Crime Laboratory Use Only) <i>Charles E. Ehr</i>	Date <i>3/11/2022</i>


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004953
Location = RUGB 8164.14.00 09/16
03/10/2022 15:51

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	15:52
02 Std. Gas	0.083	15:52
03 Room Air	0.000	15:52
04 Std. Gas	0.082	15:53
05 Room Air	0.000	15:53
06 Std. Gas	0.082	15:54
07 Room Air	0.000	15:54

Lot No = 26021080A1
Cyl No = 008
Exp Date = 10/05/2023
County = 35 Oper No. = 130392

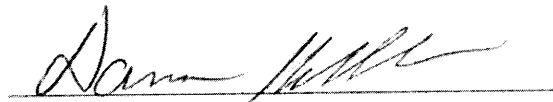

Operator Signature
DARREN HEIDBREDER

Remarks:

Form 106-I8000