



ETHANOL GAS STANDARD CYLINDER REPORT
 NORTH DAKOTA OFFICE OF ATTORNEY GENERAL
 CRIME LABORATORY DIVISION-TOXICOLOGY SECTION/BREATH ALCOHOL PROGRAM
 SFN 59282 (08/2018)

Chemical Test Operator Name (Print) <i>Rick L. Widicker</i>		
Location <i>Stutsman Co 50-downstairs (STUT)</i>		Intoxilyzer® Serial Number <i>80-006502</i>
Gas Lot Number <i>26021080A1</i>	Gas Cylinder Number <i>005</i>	Gas Expiration Date <i>10/05/23</i>

Check When Done:

- 1. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 2. Perform an ACA Test (Level 1, Function C).
 Write in the ACA Test Results (Reported to 3 Digits, Ex. 0.081).
 - A. *0.082* AC
 - B. *0.082* AC
 - C. *0.082* AC
- 3. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 4. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 5. Send the Following to the Crime Laboratory:
 - A. Ethanol Gas Standard Cylinder Report (SFN59282, Form 105-G).
 - B. ACA Test Record.

Chemical Test Operator Signature <i>[Signature]</i> *358 (010319)	Date <i>3/11/22</i>
Reviewed By (Crime Laboratory Use Only) <i>[Signature]</i>	Date <i>3/11/2022</i>

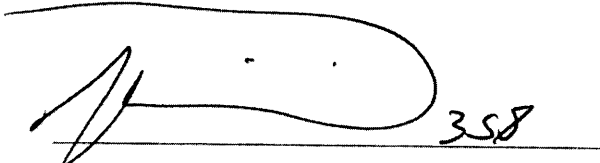
Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006502
Location = STUT 8164.14.00 09/16
03/11/2022 01:51

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	01:52
02 Std. Gas	0.082	01:52
03 Room Air	0.000	01:53
04 Std. Gas	0.082	01:53
05 Room Air	0.000	01:54
06 Std. Gas	0.082	01:54
07 Room Air	0.000	01:55

Lot No = 26021080A1
Cyl No = 5
Exp Date = 10/05/2023
County = 47 Oper No. = 010319


Operator Signature
RICK WIDICKER

Remarks: *ACA TEST - Good*

Form 106-I8000