



ETHANOL GAS STANDARD CYLINDER REPORT
 NORTH DAKOTA OFFICE OF ATTORNEY GENERAL
 CRIME LABORATORY DIVISION-TOXICOLOGY SECTION/BREATH ALCOHOL PROGRAM
 SFN 59282 (08/2018)

Chemical Test Operator Name (Print) <i>Joshua Fastnacht</i>		
Location <i>LaMoure SD</i>	Intoxilyzer® Serial Number <i>80-006491</i>	
Gas Lot Number <i>33321080A4</i>	Gas Cylinder Number <i>010</i>	Gas Expiration Date <i>1/5/2024</i>

Check When Done:

- 1. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 2. Perform an ACA Test (Level 1, Function C).
 Write in the ACA Test Results (Reported to 3 Digits, Ex. 0.081).
 A. *0.082* AC
 B. *0.082* AC
 C. *0.081* AC
- 3. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 4. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 5. Send the Following to the Crime Laboratory:
 A. Ethanol Gas Standard Cylinder Report (SFN59282, Form 105-G).
 B. ACA Test Record.

Chemical Test Operator Signature <i>[Signature]</i>	Date <i>3-18-22</i>
Reviewed By (Crime Laboratory Use Only) <i>[Signature]</i>	Date <i>3/18/2022</i>

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006491
Location = LAMO 8164.14.00 09/16
03/18/2022 12:10

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	12:10
02 Std. Gas	0.082	12:11
03 Room Air	0.000	12:11
04 Std. Gas	0.082	12:12
05 Room Air	0.000	12:12
06 Std. Gas	0.081	12:13
07 Room Air	0.000	12:13

Lot No = 33321080A4
Cyl No = 10
Exp Date = 01/05/2024
County = 23 Oper No. = 132539


Operator Signature
JOSHUA FASTNACHT

Remarks:

Form 106-I8000