

INTOXILYZER® 8000 INSPECTION

Intoxilyzer® 8000 Serial Number: 80-005955

Location: TUXL

A. General Setup and Checks:

1. Diagnostic Tests Pass and Instrument in Ready Mode
2. Breath tube heated
3. Date, time and location code (Level 2,E). Re-set if necessary.
4. Print test (Level 1,P). Sign and attach test record.
5. Tank monitor (Level 3,D,G). Display and Regulator within 50 psi.
 - a. Display: 645 psi Regulator: 650 psi

B. Tests (Sign and attach test records):

- Configure simulator for the following tests (Level 1,S).
1. Wet Calibration Check
 - a. Low AC (Level 1,C): Known Value ≤ 0.03 AC: 0.020 AC
Sim. Ser #: MP3070 Lot #: 22CP41184 Exp. Date: 3/1/24
 - b. High AC (Level 1,C): Known Value ≥ 0.25 AC: 0.400 AC
Sim. Ser #: MP3065 Lot #: 262103E Exp. Date: 3/24/23

Configure dry gas standard for the following tests (Level 1,S).

2. Interferent Check (Level 1,B): Known Value: 0.10 AC + 0.05% Acetone
Sim. Ser #: DR25147 Lot #: IC56 Exp. Date: NA
 Display reads "Interferent Detect"
3. RFI Check (CMS Mode) Display reads "RFI Detect"
4. Dry Calibration Check (Level 1,C): Known Value 0.080 AC
Gas Cyl Lot #: 02621080A1 Cyl #: 2 Exp. Date: 3/5/23

Test 1 <u>0.083</u>	Test 4 <u>0.083</u>	Test 7 <u>0.083</u>
Test 2 <u>0.083</u>	Test 5 <u>0.082</u>	Test 8 <u>0.083</u>
Test 3 <u>0.082</u>	Test 6 <u>0.083</u>	Test 9 <u>0.083</u>
Average <u>0.083</u>		

C. Remarks/Maintenance: Annual inspection.

Replaced breath tube o-ring. Replaced simulator return o-ring.
Cleaned breath hose. screen.

Instrument is acceptable to be used in the field.

Jesselle Portschiller
Breath Analyst Signature

21 Apr 22
Date

[Signature]
Reviewed by

042122
Date

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005955
Location = TOXL 8164.14.00 09/16
04/21/2022 10:02

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:03
02 Std. Sol.	0.021	10:03
03 Room Air	0.000	10:04
04 Std. Sol.	0.020	10:05
05 Room Air	0.000	10:05
06 Std. Sol.	0.020	10:06
07 Room Air	0.000	10:06

08 Sim Temp = 34.0°C

Simul Ser No = MP3070
Std Sol No = 22CP41184
County = 08

Oper No. = 132128

Janelle Portscheller

Operator Signature
JANELLE PORTSCHELLER

Remarks:

Low AC check 0.020

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005955
Location = TOXL 8164.14.00 09/16
04/21/2022 10:08

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:09
02 Std. Sol.	0.399	10:10
03 Room Air	0.000	10:10
04 Std. Sol.	0.400	10:11
05 Room Air	0.000	10:11
06 Std. Sol.	0.401	10:12
07 Room Air	0.000	10:13

08 Sim Temp = 34.0°C

Simul Ser No = MP3065

Std Sol No = 202103E

County = 08

Oper No. = 132128

Janelle Portscheller

Operator Signature
JANELLE PORTSCHELLER

Remarks: *High Ac Check 0.400*

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005955
Location = TOXL 8164.14.00 09/16
04/21/2022 10:17

Test	AC	Time
01 Room Air	0.000	10:18
02 *Subject Test	INT*	10:18
03 Room Air	0.000	10:19

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = INTERFERENT CK
Dr. Lic. = ND/DIS821456
Lot No = 02621080A1
Cyl No = 2
Expiration Date = 03/05/2023
County = 08 Oper No. = 132128

Janelle Portschiller

Operator Signature
JANELLE PORTSCHELLER

Remarks:

Inter ferent check

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005955
Location = TOXL 8164.14.00 09/16
04/21/2022 10:19

Test	AC	Time
01 Diagnostic	OK	10:20
02 Room Air	0.000	10:20
03 *Subject Test	RFI*	10:22
04 Room Air	0.000	10:22

*Invalid Test
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = RFI CK
Dr. Lic. = ND/DIS821456
Lot No = 02621080A1
Cyl No = 2
Expiration Date = 03/05/2023
County = 08 Oper No. = 132128

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature
JANELLE PORTSCHELLER

Remarks: *RFI check*

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005955
Location = TOXL 8164.14.00 09/16
04/21/2022 10:22

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:23
02 Std. Gas	0.083	10:23
03 Room Air	0.000	10:24
04 Std. Gas	0.083	10:24
05 Room Air	0.000	10:25
06 Std. Gas	0.082	10:25
07 Room Air	0.000	10:26

Lot No = 02621080A1
Cyl No = 2
Exp Date = 03/05/2023
County = 08 Oper No. = 132128

Janelle Portscheller

Operator Signature
JANELLE PORTSCHELLER

Remarks:

Dry Calibration Check 1-3

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005955
Location = TOXL 8164.14.00 09/16
04/21/2022 10:27

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:28
02 Std. Gas	0.083	10:28
03 Room Air	0.000	10:28
04 Std. Gas	0.082	10:29
05 Room Air	0.000	10:29
06 Std. Gas	0.083	10:30
07 Room Air	0.000	10:30


Lot No = 02621080A1

Cyl No = 2

Exp Date = 03/05/2023

County = 08

Oper No. = 132128

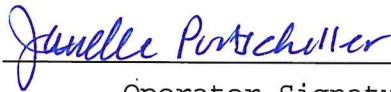

Operator Signature
JANELLE PORTSCHELLER

Remarks: *Dry Calibration Check #4-6*

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005955
Location = TOXL 8164.14.00 09/16
04/21/2022 10:31

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:31
02 Std. Gas	0.083	10:32
03 Room Air	0.000	10:32
04 Std. Gas	0.083	10:32
05 Room Air	0.000	10:33
06 Std. Gas	0.083	10:33
07 Room Air	0.000	10:34

Lot No = 02621080A1
Cyl No = 2
Exp Date = 03/05/2023
County = 08 Oper No. = 132128

Janelle Portscheller
Operator Signature
JANELLE PORTSCHELLER

Remarks: *Dry Calibration Check 7-9*