



ETHANOL GAS STANDARD CYLINDER REPORT
 NORTH DAKOTA OFFICE OF ATTORNEY GENERAL
 CRIME LABORATORY DIVISION-TOXICOLOGY SECTION/BREATH ALCOHOL PROGRAM
 SFN 59282 (08/2018)

Chemical Test Operator Name (Print) <i>Terry Dillon</i>		
Location <i>Grant County</i>	Intoxilyzer® Serial Number <i>80-004200</i>	
Gas Lot Number <i>26D21080A1</i>	Gas Cylinder Number <i>6</i>	Gas Expiration Date <i>10-05-2023</i>

Check When Done:

- 1. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 2. Perform an ACA Test (Level 1, Function C).
 Write in the ACA Test Results (Reported to 3 Digits, Ex. 0.081).
 A. 0. 082 AC
 B. 0. 082 AC
 C. 0. 081 AC
- 3. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 4. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 5. Send the Following to the Crime Laboratory:
 A. Ethanol Gas Standard Cylinder Report (SFN59282, Form 105-G).
 B. ACA Test Record.

Chemical Test Operator Signature <i>Terry Dillon</i>	Date <i>04/22/2022</i>
Reviewed By (Crime Laboratory Use Only) <i>Janelle Patschiller</i>	Date <i>06/27/2022</i>

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004200
Location = CARS 8164.14.00 09/16
04/22/2022 12:59

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	13:00
02 Std. Gas	0.082	13:01
03 Room Air	0.000	13:01
04 Std. Gas	0.082	13:02
05 Room Air	0.000	13:02
06 Std. Gas	0.081	13:02
07 Room Air	0.000	13:03

Lot No = 26021080A1

Cyl No = 6

Exp Date = 10/05/2023

County = 19

Oper No. = 132318



Operator Signature
TERRY DILLON

Remarks:

Form 106-I8000