



**INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT**  
 NORTH DAKOTA OFFICE OF ATTORNEY GENERAL  
 CRIME LABORATORY DIVISION-TOXICOLOGY SECTION/BREATH ALCOHOL PROGRAM  
 SFN 59281 (08/2018)

Serial Number <i>80-006503</i>	Instrument Location <i>Minot police dept</i>
Reason for Install/Repair	
<input checked="" type="checkbox"/> Install After Receiving From Crime Laboratory <input type="checkbox"/> Install After Location Change <input type="checkbox"/> Other (Specify) _____	

Check When Done:

- 1. Surge Protector Installed/Property Grounded.
- 2. Telephone Line Connected to Intoxilyzer® 8000.
- 3. Breath Tube Heated.
- 4. Enter Preliminary Data (i.e. Date, Time, DST (Y), and Location; Level 2, Function E).
- 5. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 6. Run Tests:
  - A. Print Test (Level 1, Function P).
  - B. ACA Test (Level 1, Function C).
  - C. Radio Frequency Interference (RFI) Test (CMS Mode or Level 1, Function B or C; Key Radio During Test).
- 7. Repair and/or Maintenance Performed (if any): \_\_\_\_\_
- 8. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 9. File Previous Intoxilyzer® Record (SFN504096, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 10. Send the Following to the Crime Laboratory: Completed Intoxilyzer® 8000 Installation and Repair Checkout (SFN59281, Form 104-G), Print Test, ACA Test, and RFI Test.

Field Inspector Signature <i>[Signature]</i>	Date <i>04-27-2022</i>
---	---------------------------

Crime Laboratory Use Only

This installation has been reviewed and the instrument is approved to be used for the analysis of breath to determine alcohol concentration from the date the Field Inspector performed the installation. This record on file at the Office of Attorney General, Crime Laboratory Division, in the County of Burleigh, North Dakota, is certified to be a true and correct copy of the documents received.

Reviewed/Certified By <i>Janelle Postschiller</i>	Certified Date <i>4/29/2022</i>
--	------------------------------------



Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-006503  
Location = MIPD      8164.14.00 09/16  
04/27/2022      15:52

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	15:53
02 Std. Gas	0.081	15:54
03 Room Air	0.000	15:54
04 Std. Gas	0.081	15:54
05 Room Air	0.000	15:55
06 Std. Gas	0.080	15:55
07 Room Air	0.000	15:56

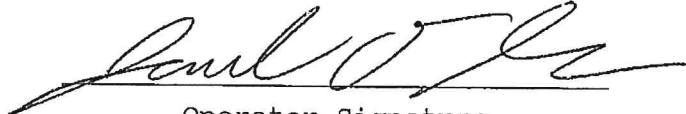
Lot No = 02621080A1

Cyl No = 10

Exp Date = 03/05/2023

County = 51

Oper No. = 130907

  
Operator Signature  
JACOB OLSON

41717893

Remarks:

Form 106-I8000

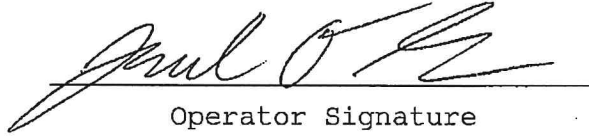
Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-006503  
Location = MIPD      8164.14.00 09/16  
04/27/2022      15:58

Test	AC	Time
01 Room Air	RFI*	15:59
02 Room Air	0.000	16:00

\*Invalid Test  
Inhibited - RFI

Sub Name = TEST, TEST TEST  
Sub DOB = 04/20/2022  
Sub Sex = Male      Weight = 222  
Test = DUI      Cit = 22-00000  
Dr. Lic. = ND/NA  
Lot No = 02621080A1  
Cyl No = 10  
Expiration Date = 03/05/2023  
County = 51      Oper No. = 130907

  
Operator Signature  
JACOB OLSON

41717893

Remarks: