




**ETHANOL GAS STANDARD CYLINDER REPORT**  
 NORTH DAKOTA OFFICE OF ATTORNEY GENERAL  
 CRIME LABORATORY DIVISION-TOXICOLOGY SECTION/BREATH ALCOHOL PROGRAM  
 SFN 59282 (08/2018)

Chemical Test Operator Name (Print) <i>Andrew P. Eisenschenk</i>		
Location <i>Watford City Police Department</i>		Intoxilyzer® Serial Number <i>SN 80-004956</i>
Gas Lot Number <i>33321080A4</i>	Gas Cylinder Number <i>11</i>	Gas Expiration Date <i>1/5/2024</i>

Check When Done:

- 1. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 2. Perform an ACA Test (Level 1, Function C).  
 Write in the ACA Test Results (Reported to 3 Digits, Ex. 0.081).
  - A. *0.081* AC
  - B. *0.082* AC
  - C. *0.082* AC
- 3. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 4. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 5. Send the Following to the Crime Laboratory:
  - A. Ethanol Gas Standard Cylinder Report (SFN59282, Form 105-G).
  - B. ACA Test Record.

Chemical Test Operator Signature 	Date <i>5-4-22</i>
Reviewed By (Crime Laboratory Use Only) <i>Janelle Potchiller</i>	Date <i>11 May 20</i>

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-004956  
Location = WCPD      8164.14.00 09/16  
05/04/2022      12:19

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	12:19
02 Std. Gas	0.081	12:20
03 Room Air	0.000	12:20
04 Std. Gas	0.082	12:21
05 Room Air	0.000	12:21
06 Std. Gas	0.082	12:22
07 Room Air	0.000	12:22


Lot No = 33321080A4

Cyl No = 11

Exp Date = 01/05/2024

County = 27

Oper No. = 130409

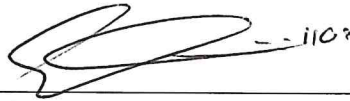

Operator Signature  
ANDREW EISENSCHENK

Remarks:

ACA for tank setup

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-004956  
Location = WCPD                8164.14.00 09/16  
05/04/2022                        12:14

DIAGNOSTIC

Voltage/Current Test	Pass
RAM Test	Pass
EEPROM Checksum Test	Pass
Real Time Clock Test	Pass
DSP Test	Pass
Analytical Stability Test	Pass
Modem Test	Pass
Temp Regulation Test	Pass

Oper No. = 130409



Operator Signature  
ANDREW EISENSCHENK

Remarks:

*Print*

Form 106-I8000