

INTOXILYZER® 8000 INSPECTION

Intoxilyzer® 8000 Serial Number: 80-005945

Location: TOXL

A. General Setup and Checks:

1. Diagnostic Tests Pass and Instrument in Ready Mode
2. Breath tube heated
3. Date, time and location code (Level 2,E). Re-set if necessary.
4. Print test (Level 1,P). Sign and attach test record.
5. Tank monitor (Level 3,D,G). Display and Regulator within 50 psi.
 - a. Display: 292 psi Regulator: 300 psi

B. Tests (Sign and attach test records):

Configure simulator for the following tests (Level 1,S).

1. Wet Calibration Check

- a. Low AC (Level 1,C): Known Value ≤ 0.03 AC: 0.020 AC
Sim. Ser #: MP3060 Lot #: 202201A Exp. Date: 1/11/24
- b. High AC (Level 1,C): Known Value ≥ 0.25 AC: 0.300 AC
Sim. Ser #: MP3064 Lot #: 202012A Exp. Date: 12/8/22

Configure dry gas standard for the following tests (Level 1,S).

2. Interferent Check (Level 1,B): Known Value: 0.10 AC + 0.05% Acetone
Sim. Ser #: 12598 Lot #: 1256 Exp. Date: NA

025114 ←

Display reads "Interferent Detect"

3. RFI Check (CMS Mode) Display reads "RFI Detect"

4. Dry Calibration Check (Level 1,C): Known Value 0.080 AC
Gas Cyl Lot #: 02621080A Cyl #: 4 Exp. Date: 3/5/2023

Test 1	<u>0.079</u>	Test 4	<u>0.079</u>	Test 7	<u>0.079</u>
Test 2	<u>0.078</u>	Test 5	<u>0.079</u>	Test 8	<u>0.079</u>
Test 3	<u>0.078</u>	Test 6	<u>0.079</u>	Test 9	<u>0.079</u>
Average	<u>0.079</u>				

C. Remarks/Maintenance: Annual inspection.

Replaced breath tube o-ring and simulator return o-ring.

Replaced electrical tape on outside of the breath tube hose.

Instrument is acceptable to be used in the field.

Janelle Pertscheller
Breath Analyst Signature

06/09/22
Date

[Signature]
Reviewed by

06/02/22
Date

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005945
Location = ~~BSPD~~ ^{MP} ~~TOXL~~ 8164.14.00 09/16
06/09/2022 12:09

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	12:10
02 Std. Sol.	0.306	12:10
03 Room Air	0.000	12:11
04 Std. Sol.	0.307	12:12
05 Room Air	0.000	12:12
06 Std. Sol.	0.309	12:13
07 Room Air	0.000	12:14

08 Sim Temp = 34.0°C

Simul Ser No = MP3064
Std Sol No = ~~202112A~~ ^{MP} 202012A
County = 08 Oper No. = 132128

Janelle Portscheller

Operator Signature
JANELLE PORTSCHELLER

Remarks: *High AC check 0.300*

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005945
Location = ~~BSPD~~ *for Tux* 8164.14.00 09/16
06/09/2022 12:34

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	12:35
02 Std. Gas	0.079	12:35
03 Room Air	0.000	12:36
04 Std. Gas	0.079	12:36
05 Room Air	0.000	12:37
06 Std. Gas	0.079	12:37
07 Room Air	0.000	12:38

Lot No = 02621080A1

Cyl No = 4

Exp Date = 03/05/2023

County = 08

Oper No. = 132128

Janelle Portscheller

Operator Signature
JANELLE PORTSCHELLER

Remarks: *Dry Calibration check #7-9*

Form 106-I8000