



**ETHANOL GAS STANDARD CYLINDER REPORT**  
 NORTH DAKOTA OFFICE OF ATTORNEY GENERAL  
 CRIME LABORATORY DIVISION-TOXICOLOGY SECTION/BREATH ALCOHOL PROGRAM  
 SFN 59282 (08/2018)

Chemical Test Operator Name (Print) <i>Tim Bleth</i>		
Location <i>BSPD</i>	Intoxilyzer® Serial Number <i>80-005945</i>	
Gas Lot Number <i>34921080A1</i>	Gas Cylinder Number <i>51</i>	Gas Expiration Date <i>2/5/2024</i>

Check When Done:

- 1. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 2. Perform an ACA Test (Level 1, Function C).  
 Write in the ACA Test Results (Reported to 3 Digits, Ex. 0.081).  
 A. *0.028* AC  
 B. *0.029* AC  
 C. *0.029* AC
- 3. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 4. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 5. Send the Following to the Crime Laboratory:
  - A. Ethanol Gas Standard Cylinder Report (SFN59282, Form 105-G).
  - B. ACA Test Record.

Chemical Test Operator Signature <i>Tim Bleth</i>	Date <i>6-10-22</i>
Reviewed By (Crime Laboratory Use Only) <i>Janelle Pertscheller</i>	Date <i>10 June 22</i>

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-005945  
Location = BSPD      8164.14.00 09/16  
06/10/2022      13:56

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	13:57
02 Std. Gas	0.078	13:58
03 Room Air	0.000	13:58
04 Std. Gas	0.079	13:59
05 Room Air	0.000	13:59
06 Std. Gas	0.079	14:00
07 Room Air	0.000	14:00

Lot No = 34921080A1  
Cyl No = 51  
Exp Date = 02/05/2024  
County = 08      Oper No. = 069823

  
\_\_\_\_\_  
Operator Signature  
TIMOTHY BLETH

Remarks:

*new Install of Cylinder - Annual Inspection*