

INTOXILYZER® 8000 INSPECTION

Intoxilyzer® 8000 Serial Number: 80-006502

Location: TOXL

A. General Setup and Checks:

1. Diagnostic Tests Pass and Instrument in Ready Mode
2. Breath tube heated
3. Date, time and location code (Level 2,E). Re-set if necessary.
4. Print test (Level 1,P). Sign and attach test record.
5. Tank monitor (Level 3,D,G). Display and Regulator within 50 psi.
 - a. Display: 1006 psi Regulator: 1000 psi

B. Tests (Sign and attach test records):

Configure simulator for the following tests (Level 1,S).

1. Wet Calibration Check

- a. Low AC (Level 1,C): Known Value ≤ 0.03 AC: 0.020 AC
Sim. Ser #: MP6036 Lot #: 21340 Exp. Date: 8/24/23
- b. High AC (Level 1,C): Known Value ≥ 0.25 AC: 0.300 AC
Sim. Ser #: MP3068 Lot #: 202201F Exp. Date: 1/18/24

Configure dry gas standard for the following tests (Level 1,S).

2. Interferent Check (Level 1,B): Known Value: 0.1 AC + 0.05% Acetone
Sim. Ser #: DR3847 Lot #: 1C56 Exp. Date: NA
 Display reads "Interferent Detect"
3. RFI Check (CMS Mode) Display reads "RFI Detect"
4. Dry Calibration Check (Level 1,C): Known Value 0.080 AC
Gas Cyl Lot #: 02621086A1 Cyl #: 2 Exp. Date: 3/5/23

| | | |
|----------------------|---------------------|---------------------|
| Test 1 <u>0.081</u> | Test 4 <u>0.081</u> | Test 7 <u>0.081</u> |
| Test 2 <u>0.082</u> | Test 5 <u>0.081</u> | Test 8 <u>0.081</u> |
| Test 3 <u>0.081</u> | Test 6 <u>0.081</u> | Test 9 <u>0.081</u> |
| Average <u>0.081</u> | | |

C. Remarks/Maintenance: Annual Inspection.

Instrument is acceptable to be used in the field.

Janelle Pertscheller
Breath Analyst Signature

6/14/22
Date

[Signature]
Reviewed by

06/16/22
Date

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006502
Location = TOXL 8164.14.00 09/16
06/14/2022 14:23

***** Printer Test *****

abcdefghijklmnopqrstuvwxy1234567890-=-|
ABCDEFGHIJKLMNPOQRSTUVWXYZ!@#%&^*()_+?

abcdefghijklmnopqrstuvwxy1234567890-=-|
ABCDEFGHIJKLMNPOQRSTUVWXYZ!@#%&^*()_+?

Current Instrument Setup


Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer (Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 26021080A1
Standard Cyl #? 5
Standard Expiration? 10/05/2023
Oper No? 132128

Flow Cal. Date: 08/19/2015
Slope 674
Intercept -654491

IR Calibration Date: 08/19/2015
 3um 9um

0th Coef(*100): -24446 -18103
1st Coef(*100): 267787 135417
2nd Coef(*100): 3387 1610
H2O adj(mg/l*10k): 740 513

***** Printer Test End *****

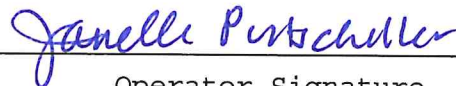

Operator Signature
JANELLE PORTSCHELLER

Remarks: *Print test*

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006502
Location = TOXL 8164.14.00 09/16
06/14/2022 14:26

WET CAL CHECK

| Test | AC | Time |
|--------------|-------|-------|
| 01 Room Air | 0.000 | 14:27 |
| 02 Std. Sol. | 0.019 | 14:28 |
| 03 Room Air | 0.000 | 14:28 |
| 04 Std. Sol. | 0.019 | 14:29 |
| 05 Room Air | 0.000 | 14:30 |
| 06 Std. Sol. | 0.019 | 14:30 |
| 07 Room Air | 0.000 | 14:31 |

08 Sim Temp = 34.0°C

Simul Ser No = MP6036
Std Sol No = 21340
County = 08 Oper No. = 132128



Operator Signature
JANELLE PORTSCHELLER

Remarks: *Low AC check 0.020*

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006502
Location = TOXL 8164.14.00 09/16
06/14/2022 14:33

WET CAL CHECK

| Test | AC | Time |
|--------------|-------|-------|
| 01 Room Air | 0.000 | 14:33 |
| 02 Std. Sol. | 0.296 | 14:40 |
| 03 Room Air | 0.000 | 14:41 |
| 04 Std. Sol. | 0.301 | 14:42 |
| 05 Room Air | 0.000 | 14:42 |
| 06 Std. Sol. | 0.302 | 14:43 |
| 07 Room Air | 0.000 | 14:43 |

08 Sim Temp = 34.0°C

Simul Ser No = MP3068
Std Sol No = 202201F
County = 08 Oper No. = 132128

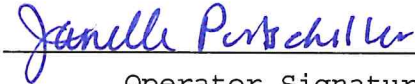

Operator Signature
JANELLE PORTSCHELLER

Remarks: *High AC check 0.300*

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006502
Location = TOXL 8164.14.00 09/16
06/14/2022 14:51

| Test | AC | Time |
|------------------|-------|-------|
| 01 Diagnostic | OK | 14:52 |
| 02 Room Air | 0.000 | 14:52 |
| 03 *Subject Test | RFI* | 14:53 |
| 04 Room Air | 0.000 | 14:54 |

*Invalid Test
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 02/01/1992
Sub Sex = Female Weight = 150
Test = DUI Cit = RFI CHK
Dr. Lic. = ND/DIS921456
Lot No = 02621080A1
Cyl No = 2
Expiration Date = 03/05/2023
County = 08 Oper No. = 132128

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

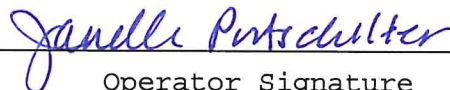

Operator Signature
JANELLE PORTSCHELLER

Remarks: *RFI check*


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006502
Location = TOXL 8164.14.00 09/16
06/14/2022 14:59

DRY CAL CHECK

| Test | AC | Time |
|-------------|-------|-------|
| 01 Room Air | 0.000 | 14:59 |
| 02 Std. Gas | 0.081 | 14:59 |
| 03 Room Air | 0.000 | 15:00 |
| 04 Std. Gas | 0.082 | 15:00 |
| 05 Room Air | 0.000 | 15:01 |
| 06 Std. Gas | 0.081 | 15:01 |
| 07 Room Air | 0.000 | 15:02 |

Lot No = 02621080A1
Cyl No = 2
Exp Date = 03/05/2023
County = 08 Oper No. = 132128

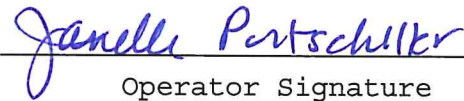

Operator Signature
JANELLE PORTSCHELLER

Remarks: *Dry Calibration check # 1-3*

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006502
Location = TOXL 8164.14.00 09/16
06/14/2022 15:02

DRY CAL CHECK

| Test | AC | Time |
|-------------|-------|-------|
| 01 Room Air | 0.000 | 15:03 |
| 02 Std. Gas | 0.081 | 15:03 |
| 03 Room Air | 0.000 | 15:04 |
| 04 Std. Gas | 0.081 | 15:04 |
| 05 Room Air | 0.000 | 15:04 |
| 06 Std. Gas | 0.081 | 15:05 |
| 07 Room Air | 0.000 | 15:05 |


Lot No = 02621080A1

Cyl No = 2

Exp Date = 03/05/2023

County = 08

Oper No. = 132128

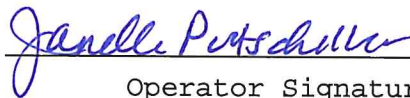

Operator Signature
JANELLE PORTSCHELLER

Remarks: *dry calibration check # 4-6*

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006502
Location = TOXL 8164.14.00 09/16
06/14/2022 15:06

DRY CAL CHECK

| Test | AC | Time |
|-------------|-------|-------|
| 01 Room Air | 0.000 | 15:07 |
| 02 Std. Gas | 0.081 | 15:07 |
| 03 Room Air | 0.000 | 15:07 |
| 04 Std. Gas | 0.081 | 15:08 |
| 05 Room Air | 0.000 | 15:08 |
| 06 Std. Gas | 0.081 | 15:09 |
| 07 Room Air | 0.000 | 15:09 |

Lot No = 02621080A1
Cyl No = 2
Exp Date = 03/05/2023
County = 08 Oper No. = 132128



Operator Signature
JANELLE PORTSCHELLER

Remarks: *Dry calibration check #7-9*