




INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT
 NORTH DAKOTA OFFICE OF ATTORNEY GENERAL
 CRIME LABORATORY DIVISION-TOXICOLOGY SECTION/BREATH ALCOHOL PROGRAM
 SFN 59281 (06/2018)

Serial Number 80-004936	Instrument Location Bottineau Co. Sheriff Dept.
Reason for Install/Repair	
<input checked="" type="checkbox"/> Install After Receiving From Crime Laboratory <input type="checkbox"/> Install After Location Change <input type="checkbox"/> Other (Specify) _____	

Check When Done:

- 1. Surge Protector Installed/Property Grounded.
- 2. Telephone Line Connected to Intoxilyzer® 8000.
- 3. Breath Tube Heated.
- 4. Enter Preliminary Data (i.e. Date, Time, DST (Y), and Location; Level 2, Function E).
- 5. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 6. Run Tests:
 - A. Print Test (Level 1, Function P).
 - B. ACA Test (Level 1, Function C).
 - C. Radio Frequency Interference (RFI) Test (CMS Mode or Level 1, Function B or C; Key Radio During Test).
- 7. Repair and/or Maintenance Performed (if any): None
- 8. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 9. File Previous Intoxilyzer® Record (SFN504096, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 10. Send the Following to the Crime Laboratory: Completed Intoxilyzer® 8000 Installation and Repair Checkout (SFN59281, Form 104-G), Print Test, ACA Test, and RFI Test.

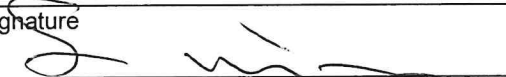Field Inspector Signature 	Date 06-14-2022
--	---------------------------

Crime Laboratory Use Only

This installation has been reviewed and the instrument is approved to be used for the analysis of breath to determine alcohol concentration from the date the Field Inspector performed the installation. This record on file at the Office of Attorney General, Crime Laboratory Division, in the County of Burleigh, North Dakota, is certified to be a true and correct copy of the documents received.

Reviewed/Certified By Janelle Portscheller	Certified Date 06/23/22
--	-----------------------------------


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004936
Location = BOTT 8164.14.00 09/16
06/14/2022 17:33

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	17:33
02 Std. Gas	0.078	17:34
03 Room Air	0.000	17:34
04 Std. Gas	0.079	17:35
05 Room Air	0.000	17:35
06 Std. Gas	0.079	17:36
07 Room Air	0.000	17:36

Lot No = 02621080A1
Cyl No = 11
Exp Date = 03/05/2023
County = 05 Oper No. = 039117


Operator Signature
SEAN MURPHY

Remarks:

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004936
Location = BOTT 8164.14.00 09/16
06/14/2022 17:38

Test	AC	Time
01 Diagnostic	OK	17:40
02 Room Air	0.000	17:40
03 *Subject Test	RFI*	17:40
04 Room Air	0.000	17:41

*Invalid Test
Inhibited - RFI

Sub Name = TEST, SUBJECT T
Sub DOB = 01/16/1965
Sub Sex = Male Weight = 200
Test = DUI Cit = 0000001
Dr. Lic. = ND/NA
Lot No = 02621080A1
Cyl No = 11
Expiration Date = 03/05/2023
County = 05 Oper No. = 039117

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

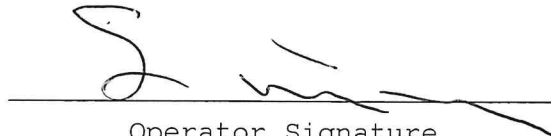

Operator Signature
SEAN MURPHY

Remarks: