

INTOXILYZER® 8000 INSPECTION

Intoxilyzer® 8000 Serial Number: 80-005361

Location: TUXL

A. General Setup and Checks:

1. Diagnostic Tests Pass and Instrument in Ready Mode
2. Breath tube heated
3. Date, time and location code (Level 2,E). Re-set if necessary.
4. Print test (Level 1,P). Sign and attach test record.
5. Tank monitor (Level 3,D,G). Display and Regulator within 50 psi.
 - a. Display: 980 psi Regulator: 1000 psi

B. Tests (Sign and attach test records):

- Configure simulator for the following tests (Level 1,S).
1. Wet Calibration Check
 - a. Low AC (Level 1,C): Known Value \leq 0.03 AC: 0.020 AC
Sim. Ser #: MP3060 Lot #: 202201A Exp. Date: 1/11/24
 - b. High AC (Level 1,C): Known Value \geq 0.25 AC: 0.301 AC
Sim. Ser #: MP3064 Lot #: 202790 Exp. Date: 12/9/22
2020 12A
- Configure dry gas standard for the following tests (Level 1,S).
2. Interferent Check (Level 1,B): Known Value: 0.10 AC + 0.05% Acetone
Sim. Ser #: DR5114 Lot #: ICS C Exp. Date: NA
 Display reads "Interferent Detect"
 3. RFI Check (CMS Mode) Display reads "RFI Detect"
 4. Dry Calibration Check (Level 1,C): Known Value 0.080 AC
Gas Cyl Lot #: 0262108A1 Cyl #: 31 Exp. Date: 3/5/23

Test 1 <u>0.079</u>	Test 4 <u>0.080</u>	Test 7 <u>0.079</u>
Test 2 <u>0.079</u>	Test 5 <u>0.079</u>	Test 8 <u>0.080</u>
Test 3 <u>0.080</u>	Test 6 <u>0.080</u>	Test 9 <u>0.079</u>
Average <u>0.079</u>		

C. Remarks/Maintenance: Annual inspection.

Replace a breath tube o-ring and simulator return o-ring.

Instrument is acceptable to be used in the field.

Janelle Pertschiller
Breath Analyst Signature

6/15/22
Date

[Signature]
Reviewed by

06/16/22
Date

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005361
Location = TOXL 8164.14.00 09/16
06/15/2022 10:09

***** Printer Test *****

abcdefghijklmnopqrstuvwxy1234567890-|=|
ABCDEFGHIJKLMNOPQRSTUVWXYZ!@#%&*()_+?

abcdefghijklmnopqrstuvwxy1234567890-|=|
ABCDEFGHIJKLMNOPQRSTUVWXYZ!@#%&*()_+?

Current Instrument Setup


Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 02621080A1
Standard Cyl #? 23
Standard Expiration? 03/05/2023
Oper No? 132128

Flow Cal. Date: 05/15/2020
Slope 634
Intercept -488053

IR Calibration Date: 05/15/2020
3um 9um

0th Coef(*100): -25383 -27323
1st Coef(*100): 288661 141732
2nd Coef(*100): 458 822
H2O adj(mg/l*10k): 335 405

***** Printer Test End *****

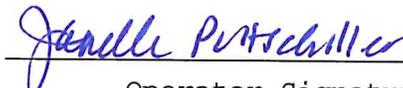
Operator Signature
JANELLE PORTSCHELLER

Remarks: *print test*

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005361
Location = TOXL 8164.14.00 09/16
06/15/2022 10:15

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:15
02 Std. Sol.	0.019	10:16
03 Room Air	0.000	10:17
04 Std. Sol.	0.020	10:17
05 Room Air	0.000	10:18
06 Std. Sol.	0.020	10:19
07 Room Air	0.000	10:19

08 Sim Temp = 34.0°C

Simul Ser No = MP3060
Std Sol No = 202201A
County = 08 Oper No. = 132128

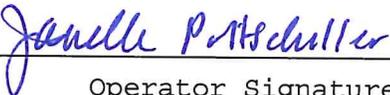

Operator Signature
JANELLE PORTSCHELLER

Remarks: *Low AC check 0.020*

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005361
Location = TOXL 8164.14.00 09/16
06/15/2022 10:23

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:23
02 Std. Sol.	0.301	10:25
03 Room Air	0.000	10:25
04 Std. Sol.	0.301	10:26
05 Room Air	0.000	10:27
06 Std. Sol.	0.300	10:27
07 Room Air	0.000	10:28

08 Sim Temp = 34.0°C

Simul Ser No = MP3064
Std Sol No = 202012A
County = 08 Oper No. = 132128

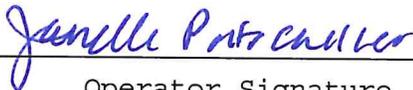

Operator Signature
JANELLE PORTSCHELLER

Remarks: *High Ac Check 0.300*


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005361
Location = TOXL 8164.14.00 09/16
06/15/2022 10:32

Test	AC	Time
01 Room Air	0.000	10:33
02 *Subject Test	INT*	10:34
03 Room Air	0.000	10:35

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = INTERFERENT CK
Dr. Lic. = ND/DIS821456
Lot No = 02621080A1
Cyl No = 31
Expiration Date = 03/05/2023
County = 08 Oper No. = 132128

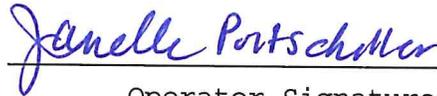

Operator Signature
JANELLE PORTSCHELLER

Remarks: *Interferent Check*

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005361
Location = TOXL 8164.14.00 09/16
06/15/2022 10:35

Test	AC	Time
01 Diagnostic	OK	10:36
02 Room Air	0.000	10:36
03 *Subject Test	RFI*	10:37
04 Room Air	0.000	10:37

*Invalid Test
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = RFI CHK
Dr. Lic. = ND/DIS821456
Lot No = 02621080A1
Cyl No = 31
Expiration Date = 03/05/2023
County = 08 Oper No. = 132128

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

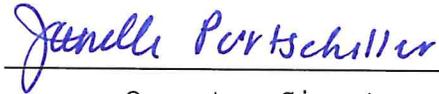

Operator Signature
JANELLE PORTSCHELLER

Remarks: *RFI Check*

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005361
Location = TOXL 8164.14.00 09/16
06/15/2022 10:39

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:39
02 Std. Gas	0.079	10:40
03 Room Air	0.000	10:40
04 Std. Gas	0.079	10:40
05 Room Air	0.000	10:41
06 Std. Gas	0.080	10:41
07 Room Air	0.000	10:42

Lot No = 02621080A1
Cyl No = 31
Exp Date = 03/05/2023
County = 08 Oper No. = 132128

Janelle Portscheller
Operator Signature
JANELLE PORTSCHELLER

Remarks: *Dry Calibration Check #1-3*

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005361
Location = TOXL 8164.14.00 09/16
06/15/2022 10:42

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:43
02 Std. Gas	0.080	10:43
03 Room Air	0.000	10:43
04 Std. Gas	0.079	10:44
05 Room Air	0.000	10:44
06 Std. Gas	0.080	10:45
07 Room Air	0.000	10:45


Lot No = 02621080A1

Cyl No = 31

Exp Date = 03/05/2023

County = 08

Oper No. = 132128

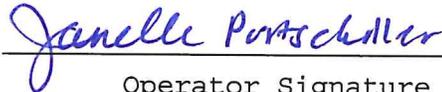

Operator Signature
JANELLE PORTSCHELLER

Remarks: *Dry Calibration check #4-6*

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005361
Location = TOXL 8164.14.00 09/16
06/15/2022 10:45

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:46
02 Std. Gas	0.079	10:46
03 Room Air	0.000	10:47
04 Std. Gas	0.080	10:47
05 Room Air	0.000	10:48
06 Std. Gas	0.079	10:48
07 Room Air	0.000	10:49

Lot No = 02621080A1
Cyl No = 31
Exp Date = 03/05/2023
County = 08 Oper No. = 132128

Janelle Portscheller
Operator Signature
JANELLE PORTSCHELLER

Remarks: *dry calibration check #7-9*